

Foldable Stretcher

Model: SKB1A02

User's Manual



Contents

1. Manual introduction.....	2
2. Applicable Scope.....	2
3. Features.....	2
4. Technical Parameters.....	2
5. Installing guide.....	2
6. Installing and using steps.....	3
7. Caution.....	4
8. Warning	4
9. Storage.....	4
10. Transport.....	4

Note: The appearance of products be described in this manual may be a little different; this is not effect product performance. Please understand if take you some inconvenience on the operation.

1. Manual Introduction

Please read the instructions carefully as they provide important information on SAFETY DURING USE AND MAINTENANCE PROCEDURES descriptions and images contained in the publication are not binding; we has the right to modify any information for enhancement purposes without any obligation to update this manual. Graphics and images contained in this manual are only examples and may improvement of products.

2. Applicable Scope

Used in hospital, force field, stadium emergency, for transporting, immobilizing, lifting patient in the pre-hospital care environment. Non-sterile.

3. Features

- The folding stretcher with outrigger is made of high strength aluminum alloy material and the waterproof leather materials
- This stretcher can be folded in length and width, easy for transportation and storage
- The fabric is fire proof, water proof, anti-cracking and easy to clean
- With safety belts, stirup feet and hinged spread bar
- Product size (L x W x H): 208 x 55 x 13cm
- Folded size (L x W x H): 104 x 17 x 9cm
- Load bearing: $\leq 159\text{kg}$

4. Technical parameters

Model	SKB1A02
Unfolded Size (L*W*H)	208*55*13CM
Folded Size	104*17*9CM
Loading bearing	159kg

5. Installing guide


For saving the packing volume and safe transportation, the scoop stretcher you received are not attached the body fixation belts. You need to fasten the safety belts on the stretcher.

Installing guide comes with the users' manual. Before installing the product, please read this guide carefully and strictly follow the process.

Note: Try the foldable stretcher after installing before putting it into use.

6. Installing and using steps



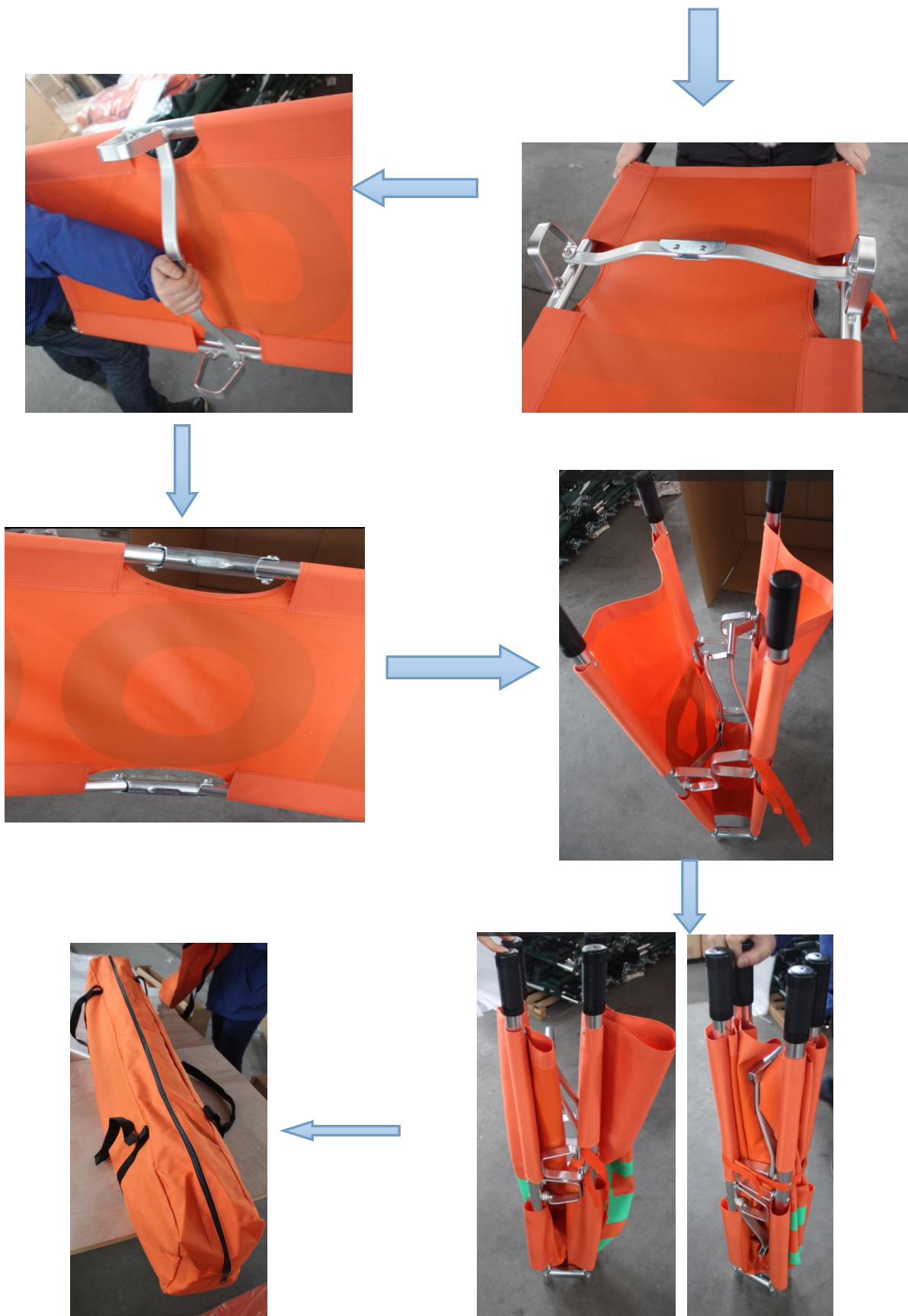

7. Caution

7.1 Operators must read this manual carefully, follow the instructions and familiarize with the correct operating and maintenance procedures of the trolley, otherwise the company is not responsible for the danger occur.

7.2 Check the parts if well installed in right positions before operating.

7.3 Prohibited illegal and barbaric action and overload operation, to avoid improper use damaging the bed.

7.4 When raising the backrest, there must no any obstructers on the way of raise.

7.5 After each use, the table should be put at lowest position to keep the lifting mechanism in minimum use, auxiliary units should be put back to original place.

8. Warning

⚠ DO NOT ALLOW SMALL CHILDREN ON OR NEAR BED DURING OPERATION.

⚠ DO NOT ALLOW THIS DEVICE TO BE OPERATED BY SMALL CHILDREN.

⚠ WHEN OPERATING THE FUNCTION OF THE BED, ALWAYS ENSURE THAT THE INDIVIDUAL CONFINED TO THE BED IS POSITIONED PROPERLY WITHIN THE CONFINES OF THE BED. DO NOT LET ANY EXTREMITIES PROTRUDE OVER THE SIDE OR BETWEEN THE BED RAILS WHEN PERFORMING ANY FUNCTIONS.

⚠ WARNING/CAUTION LABELS APPLIED TO THE BED OUTLINE HAZARDS OR UNSAFE PRACTICES THAT COULD RESULT IN PERSONAL INJURY AND/OR PROPERTY DAMAGE.

⚠ WHEN USING NASAL OR MASKED TYPE ADMINISTERING EQUIPMENT, OXYGEN OR AIR TUBING MUST BE ROUTED AND SECURED PROPERLY TO ENSURE THAT TUBING DOES NOT BECOME ENTANGLED AND EVENTUALLY SEVERED DURING NORMAL OPERATION OF BED.

⚠ KEEP ALL MOVING PARTS FREE OF OBSTRUCTIONS (I.E. BLANKETS/SHEETS, HEATING BLANKETS/PADS, TUBING, WIRING, AND OTHER TYPES OF PRODUCTS)

9. Storage

11.1 Store this stretcher in the place of damp proof and non-corrosion environment

10. Transport

11.1 Be transferred by common transporting tools: ship, plane, train, truck.

11.2 Hospital bed can be manually handled.

11.3 No throwing or dropping when transshipment.

