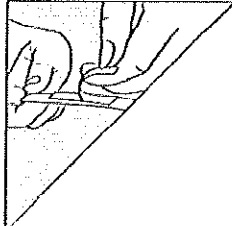
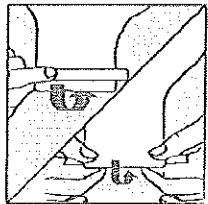
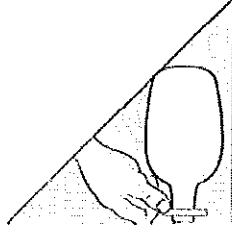
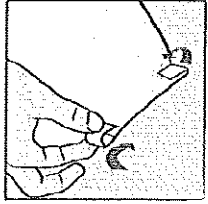
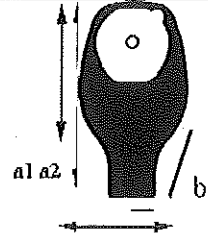


Modalitati de utilizare a clemelor flexibile si rigide:

How to use the soft clamp?

	1 - Remove the release paper from the adhesive strip.		3 - Fold the outlet backwards four times using the clamp.
	2 - Center and stick the soft clamp at the bottom of the opening (on the outer side of the pouch, not on the skin side). Leave a small space between the clamp and the end of the opening. Its size should approximately be the width of the clamp.		4 - Squeeze the extremities of the clamp firmly to complete closure.

2.4 Referinte

Coduri referinta	Diametrul in mm		Capacitate in conformitate cu ISO 8670-2	Culoare	Ambalare
55410A	10 → 70 mm	a1= 235 mm a2= 295 mm b= 150 mm	650 ml	Bej	Cutii cu 10, 20 sau 30 bucati cu 30 cleme moi
55510A	10 → 70 mm			Transparent	Cutii cu 10 sau 30 bucati cu 30 cleme moi

2.5 Specificatii tehnice ale produsului

Fara latex si PVC

Componente	Materiale
Ambalare flansa	Hartie siliconata cu margini suprapuse
Stratul adeziv al flansci	Adeziv hidrocoloidic (compus din polisobutilen, carboximeticleluloza si pectina) Forma hexagonala
Punga colectoare	<ul style="list-style-type: none"> • bej sau transparenta pe fata de la piele: film multistrat si material netesut din microfibre de polietilena care protejeaza impotriva mirosurilor • pe fata externa : film alb multistrat, material alb netesut din microfibre de polietilena
Cleme flexibile	Material alb, flexibil, dublu adeziv 4 fire de metal acoperite cu un film alb din polietilena Dimensiuni : lungime 140 +/- 3mm, latime 8 +/- 0.5 mm

