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BIBLIOGRAFIE

1. Wu, A.H.B., Tietz Clinical guide to laboratory test, 4th Ed., (W.B. Saunders eds.), (2006), 66.
2. Dufour, R., The liver: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
3. Johnson, A.M., Amino Acids and Proteins, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 286.
4. Doumas, B., *et al.*, Clin. Chim. Acta., (1971), **31**, 87.
5. Berth, M. & Delanghe, J., Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg., (2004), **59**, 263.
6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed, AACC Press (1997).
7. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed, AACC Press (1995).

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BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzyme, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (Saunders), (2008), 317.
2. Wu, A.H.B., *Tietz Clinical guide to laboratory test*, 4th Ed., (W.B. Saunders Company), (2006), 64
3. Dufour, R., *The liver: Function and chemical Pathology, Clinical Chemistry: Theory Analysis. Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby, Inc.), (2010), 586, appendix
4. Schumann, G., et al. *Clin Chem Lab Med.*, (2002), **40**, 718.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D. S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young, D. S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995)

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BIBLIOGRAFIE

1. Panteghini, M. & Biais, R., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Dufour, D.R., *The Pancreas: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 651 and appendix.
3. Winn-Deen, E.S., *et al*, *Clin. Chem.*, (1988), **34**, 2005
4. Guder W.G, *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma, and serum samples*, (2002). WHO/DIL/LAB/99.1 Rev.2.
5. Schumann, G., *et al*. *Clin. Chem. Lab Med.*, (2006), **44**, 1146.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D. S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young, D. S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzyme*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (Saunders), (2008), 317.
2. Wu, A.H.B., Tietz Clinical guide to laboratory test, 4th Ed., (W.B. Saunders Company), (2006), 64
3. Dufour, R., *The liver: Function and chemical Pathology*, Clinical Chemistry: Theory Analysis. Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby, Inc.), (2010), 586, appendix
4. Schumann, G., et al. Clin Chem Lab Med., (2002), **40**, 718.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D. S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young, D. S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995)

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BIBLIOGRAFIE

1. Higgins, T., *et al.*, *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
7. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Higgins, T., *et al.*, Hemoglobin, Iron, and Bilirubin, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Itani, O., Tsang, R.C., *Bone disease, Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 614 and appendix.
2. Endres, D.B., Rude, R.K., *Disorders of Bone. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 711.
3. Foley, K. F., Boccuzzi, L., *Urine Calcium: Laboratory Measurement and Clinical Utility, Labmedicine* (2010), **41**, 683.
4. Bauer, P. J., *Anal. Biochem.*, (1981), **110**, 61.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 202.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg.*, (2004), **59**, 263.
8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
9. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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
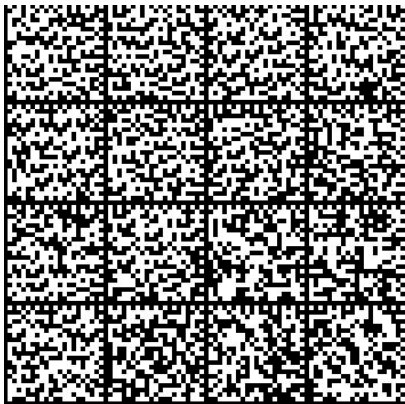
Traceability legend :
SRM: Standard Reference Material
ERM: European Reference Material
ID-MS: Isotope Dilution – Mass Spectrometry
ID-GC-MS: Isotope Dilution – Gas Chromatography –Mass Spectrometry
IFCC: International Federation of Clinical Chemistry and Laboratory Medicine

selectra
PRO

selectra
MACH

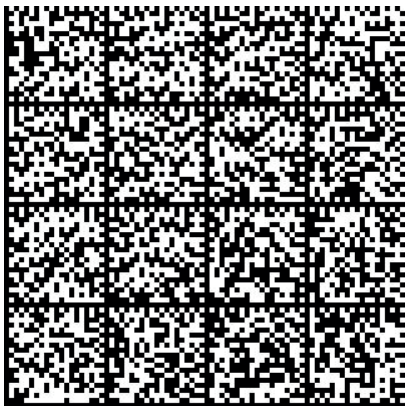
ELICAL 2

REF CALI-0550 LOT 03-3280 2027-01 

FV-CALI2-v39 (01/2025)


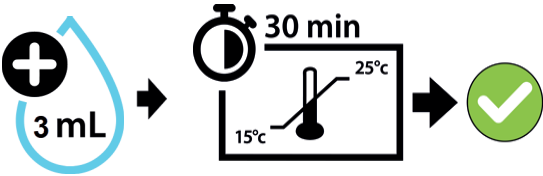
Preparation

Precautions

 8 hours horas heures stunden 15°C → 25°C	 2 days días jours tage 2°C → 8°C	 4 weeks semanas semaines wochen -25°C → -15°C	
--	--	---	---

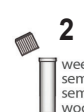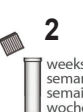Exceptions

	Total bilirubin	Direct bilirubin
 15°C → 25°C	 6 hours horas heures stunden	 3 hours horas heures stunden
 2°C → 8°C	 1 days días jours tage	 8 hours horas heures stunden
 -25°C → -15°C	 2 weeks semanas semaines wochen	 2 weeks semanas semaines wochen

 www.vitalscientific.com

ELICAL 2

LOT 03-3280

2027-01

FOR SELECTRA PRO ANALYZERS*

Code	REAGENT NAME	METHOD	Traceability	Value	Units	Value	Units
ALBU-0XXX	ALBUMIN	Bromocresol green (BCG)	ERM-DA 470k	3.51	g/dL	35.1	g/L
PASL-0XXX	ALP (DEA) SL	DGKC	Manual measurement	455	U/L	7.58	μkat/L
ALPI-0XXX	ALP IFCC	IFCC	IFCC	247	U/L	4.12	μkat/L
ALSL-0XXX	ALT / GPT 4+1 SL	IFCC without pyridoxal-phosphate	IFCC	98.3	U/L	1.64	μkat/L
AMSL-0XXX	AMYLASE SL	CNP-G3 / Standardization against IFCC method	IFCC	234	U/L	3.89	μkat/L
ASSL-0XXX	AST / GOT 4+1 SL	IFCC without pyridoxal-phosphate	IFCC	110.5	U/L	1.84	μkat/L
BITO-0XXX	TOTAL BILIRUBIN 4+1	Modified Malloy-Evelyn	SRM 916a	4.14	mg/dL	70.8	μmol/L
BIDI-0XXX	DIRECT BILIRUBIN 4+1	Modified Malloy-Evelyn	SRM 916a	2.31	mg/dL	39.5	μmol/L
CALA-0XXX	CALCIUM ARSENAZO	Arsenazo III	SRM 956d	11.20	mg/dL	2.79	mmol/L
CHSL-0XXX	CHOLESTEROL SL	Enzymatic / PAP	ID-GC-MS	157	mg/dL	4.06	mmol/L
CHEB-0250	CHOLINESTERASE	DGKC	DGKC Manual measurement	5270	U/L	87.83	μkat/L
CKSL-0XXX	CK NAC SL	IFCC	IFCC	364	U/L	6.07	μkat/L
CRCO-0XXX	CREATININE JAFFE IMPROVED APPLICATION	Jaffe	ID-MS	4.06	mg/dL	359	μmol/L
CRSL-0XXX	CREATININE PAP SL	Enzymatic / PAP	ID-MS	3.81	mg/dL	337	μmol/L
TIBC-0XXX	Direct TIBC	Chromazurol B	ERM-DA 470k (Transferrin value converted to TIBC)	388	μg/dL	69.5	μmol/L
GISL-0XXX	GAMMA GT PLUS SL	IFCC	IFCC	101.0	U/L	1.69	μkat/L
GPSL-0XXX	GLUCOSE PAP SL	Enzymatic / PAP	ID-MS	192.2	mg/dL	10.67	mmol/L
GHSL-0XXX	GLUCOSE HK SL	Hexokinase	ID-MS	198.0	mg/dL	10.99	mmol/L
FEFE-0XXX	IRON FERENE	Ferene	SRM 937	190	μg/dL	34.1	μmol/L
IRON-0XXX	IRON FERENE NEW PRODUCT	Ferene	SRM 937	180	μg/dL	32.2	μmol/L
LACI-0250	LACTATE	Enzymatic / PAP	Primary reference material (weighed in purified material)	30.2	mg/dL	3.35	mmol/L
LLSL-0XXX	LDH-L SL	IFCC	IFCC	263	U/L	4.38	μkat/L
LPSL-0XXX	LIPASE SL	DGGR	Manual measurement	99	U/L	1.66	μkat/L
LITH-0XXX	LITHIUM	Substituted porphyrin	Atomic absorption	1.41	mmol/L	-	-
MGXB-0XXX	MAGNESIUM XB	Xylidyl Blue	Atomic absorption	2.65	mg/dL	1.09	mmol/L
PHOS-0XXX	PHOSPHORUS	Phosphomolybdate	SRM 956d	5.49	mg/dL	1.77	mmol/L
PROB-0XXX	TOTAL PROTEIN PLUS	Biuret	SRM 909c	5.07	g/dL	50.7	g/L
TGML-0XXX	TRIGLYCERIDES SL TRIGLYCERIDES MONO SL	Enzymatic / PAP	ID-GC-MS	135	mg/dL	1.53	mmol/L
URSL-0XXX	UREA UV SL	Urease / GIDH	ID-MS	99.1	mg/dL	16.50	mmol/L
AUML-0XXX	URIC ACID MONO SL	Enzymatic / PAP	ID-MS	5.34	mg/dL	318	μmol/L
AUSL-0XXX	URIC ACID SL	Enzymatic / PAP	ID-MS	5.34	mg/dL	318	μmol/L

* and earlier Selectra analyzers / et automates Selectra antérieurs
Elical 2 is a multiparametric calibrator intended for the calibration of VitalScientific reagents.

ELICAL 2

LOT 03-3280

2027-01

FOR SELECTRA MACH5 ANALYZERS

CODE	REAGENT NAME	METHOD	Traceability	Value	Units	Value	Units
ALBU-M830	ALBUMIN	Bromocresol green (BCG)	ERM-DA 470k	3.51	g/dL	35.1	g/L
ALPI-0230	ALP IFCC	IFCC	IFCC	247	U/L	4.12	μkat/L
ALSL-M490	ALT / GPT	IFCC without pyridoxal-phosphate	IFCC	98.3	U/L	1.64	μkat/L
AMSL-M430	AMYLASE	CNP-G3 / Standardization against IFCC method	IFCC	234	U/L	3.89	μkat/L
ASSL-M490	AST / GOT	IFCC without pyridoxal-phosphate	IFCC	110.5	U/L	1.84	μkat/L
BITO-M430	TOTAL BILIRUBIN	Modified Malloy-Evelyn	SRM 916a	4.14	mg/dL	70.8	μmol/L
BIDI-M430	DIRECT BILIRUBIN	Modified Malloy-Evelyn	SRM 916a	2.31	mg/dL	39.5	μmol/L
CALA-M430	CALCIUM ARSENAZO	Arsenazo III	SRM 956d	11.20	mg/dL	2.79	mmol/L
CHSL-M690	CHOLESTEROL	Enzymatic / PAP	ID-GC-MS	157	mg/dL	4.06	mmol/L
CHEB-0250	CHOLINESTERASE	DGKC	DGKC Manual measurement	5270	U/L	87.83	μkat/L
CKSL-M230	CK NAC	IFCC	IFCC	364	U/L	6.07	μkat/L
CRSL-M490	CREATININE PAP	Enzymatic / PAP	ID-MS	3.81	mg/dL	337	μmol/L
TIBC-M130	Direct TIBC	Chromazurol B	ERM-DA 470k (Transferrin value converted to TIBC)	388	μg/dL	69.5	μmol/L
GISL-M230	GAMMA-GT	IFCC	IFCC	101.0	U/L	1.69	μkat/L
GPSL-M690	GLUCOSE PAP	Enzymatic / PAP	ID-MS	192.2	mg/dL	10.67	mmol/L
GHSL-M490	GLUCOSE HK	Hexokinase	ID-MS	198.0	mg/dL	10.99	mmol/L
FEFE-M230	IRON FERENE	Ferene	SRM 937	190	μg/dL	34.1	μmol/L
IRON-M230	IRON FERENE NEW PRODUCT	Ferene	SRM 937	180	μg/dL	32.2	μmol/L
LACI-0250	LACTATE	Enzymatic / PAP	Primary reference material (weighed in purified material)	30.2	mg/dL	3.35	mmol/L
LLSL-M230	LDH IFCC	IFCC	IFCC	263	U/L	4.38	μkat/L
LPSL-0250	LIPASE	DGGR	Manual measurement	99	U/L	1.66	μkat/L
LITH-0230	LITHIUM	Substituted porphyrin	Atomic absorption	1.41	mmol/L	-	-
MGXB-M430	MAGNESIUM XB	Xylidyl Blue	Atomic absorption	2.65	mg/dL	1.09	mmol/L
PHOS-M430	PHOSPHORUS	Phosphomolybdate	SRM 956d	5.49	mg/dL	1.77	mmol/L
PROB-M830	TOTAL PROTEIN	Biuret	SRM 909c	5.07	g/dL	50.7	g/L
TGML-M690	TRIGLYCERIDES	Enzymatic / PAP	ID-GC-MS	135	mg/dL	1.53	mmol/L
URSL-M830	UREA	Urease / GIDH	ID-MS	99.1	mg/dL	16.50	mmol/L
AUML-M830	URIC ACID	Enzymatic / PAP	ID-MS	5.34	mg/dL	318	μmol/L

Elical 2 is a multiparametric calibrator intended for the calibration of VitalScientific reagents.

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), JAMA, (2001), **285**, 2486.
4. Ahmadraji, T. & Killard, A., Analytical Methods, (2013), **5**, 3612.
5. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 564.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, *Clin. Chem. Lab. Med.*, (2020), **58**, 496.
4. Allain, C.C., *et al.*, *Clin. Chem.*, (1974), **20**, 470.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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English - EN

INTENDED USE

This *in vitro* diagnostic device is intended for the quality control of the performances of reagents, used for CK-MB quantitative determination.

This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum and human CK-MB.
- Sodium azide < 0.1 % (w/w)
- CK-MB concentrations are lot-specific.

MATERIALS REQUIRED

BUT NOT PROVIDED

- VitalScientific reagent, used for CK-MB quantitative determination.
- General Laboratory equipment.

TRACEABILITY

The traceability is indicated on the instructions for use of the reagent used in combination with the recommended calibration factor.

PRECAUTIONS FOR USE AND WARNINGS

- Harmful to aquatic life with long lasting effects.
- Avoid release into the environment.
- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of this product was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA-approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilizate.
- Add exactly 3 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

PRODUCT DETERIORATION

- The product after reconstitution should be clear. Cloudiness would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

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After reconstitution :

- These products are stable 5 days at 2-8 °C.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

PROCEDURE

To use CK-MB CONTROL, follow the procedure described in the instructions for use of the reagent used.

LIMITATIONS

Using CK-MB CONTROL has been validated with the VitalScientific systems (Analyzers and reagents used for CK-MB quantitative determination).

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DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European Union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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BIBLIOGRAFIE

1. Wu, A. H. B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 306
2. Sanhai, W.R. *et al.*, Cardiac and Muscle Disease. Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 677 and appendix.
3. Apple, F. S. & Jaffe, A. S., Cardiovascular Disease, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 614.
4. Boot, E., de Haan, L., Psychopharmacology (Berl), (2000), **150**, 347.
5. Schumann, G., *et al.*, Clin. Chem. Lab. Med., (2002), **40**, 635.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995)

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), JAMA, (2001), **285**, 2486.
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 684.
5. Ahmadraji, T. & Killard, A., Analytical Methods, (2013), **5**, 3612.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Wu, A. H. B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 306
2. Sanhai, W.R. *et al.*, Cardiac and Muscle Disease. Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 677 and appendix.
3. Apple, F. S. & Jaffe, A. S., Cardiovascular Disease, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 614.
4. Schumann, G., *et al.*, Clin. Chem. Lab. Med., (2002), **40**, 635.
5. Neumeier, D., *et al.*, Clin.Chim.Acta., (1976), **73**, 445.
6. Klein, G., *et al.*, Clin. Chem., (2001), **47**, Suppl. A30.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995)

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English - EN

INTENDED USE

These *in vitro* diagnostic devices are intended for the quality control of the performances of quantitative tests listed in the value sheet.

These *in vitro* diagnostic devices are for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Concentrations for each analyte to test are lot-specific.

MATERIALS REQUIRED BUT NOT PROVIDED

- VitalScientific reagents listed in the value sheet.
- General Laboratory equipment.

TRACEABILITY

The traceability is indicated on the instructions for use of the reagent used in combination with the recommended calibrator.

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PREPARATION

- Gently tap the vial so that the control material can be at the bottom of the vial.
- Carefully open the vial, avoiding the loss of lyophilizate.
- Add exactly 5 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

Important: To reactivate the alkaline phosphatase, allow the reconstituted control serum to stand for one hour at 15-25 °C.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particules would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Store at 2-8 °C and protect from light. Do not freeze.
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.
- Stability of the components:
Between 2 and 8 °C : 5 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light) :
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of direct bilirubin (when stored protected from light) :
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of alkaline phosphatase:
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 4 weeks (when frozen once)

Note:

- Store the vials tightly capped and protected from light after reconstitution.
- Mix content thoroughly prior to use.
- Thawing frozen specimen at 37°C, and vigorous mixing (vortex), should be avoided as this may denature ferritin.

PROCEDURE

To use ELITROL I & II, follow the procedure described in the instructions for the use of the reagent used.

LIMITATIONS

Using ELITROL I & II has been validated with the VitalScientific systems (Analyzers and reagents listed in the value sheet).

Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit.

Individual laboratories may obtain values different from those announced. Technique, equipment and experimental error may produce slightly different values. Each laboratory should determine their own mean values.

For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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English - EN

INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of reagents, used for C-reactive protein (CRP) quantitative determination.

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COMPOSITION

- Liquid product prepared from human plasma spiked with Human recombinant CRP.
- Sodium azide < 0.1 % (w/w)
- C-reactive protein (CRP) concentrations are lot-specific.

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- VitalScientific reagent, used for C-reactive protein (CRP) quantitative determination (Ref. ICRP-XXXX).
- Normal saline solution (NaCl 9 g/L)
- General Laboratory equipment.

TRACEABILITY

Calibrators from CRP IP CALIBRATOR SET are traceable to ERM-DA474/IFCC reference material.

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PROCEDURE

To use CRP IP CALIBRATOR SET, follow the procedure described in the instructions for use of the reagent used.

Add a zero point (NaCl 9 g/L solution).

LIMITATIONS

Using CRP IP CALIBRATOR SET has been validated with the VitalScientific systems (Analyzers and reagents used for C-reactive protein (CRP) quantitative determination (Ref. ICRP-XXXX)).

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English - EN**INTENDED USE**

This *in vitro* diagnostic device is intended for the quality control of the performances of reagents (Ref. ICRP-0400), used for C-reactive protein (CRP) quantitative determination. This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Liquid product prepared from human plasma spiked with human recombinant CRP.
- Sodium azide < 0.1 % (w/w)
- C-reactive protein (CRP) concentrations are lot-specific.

MATERIALS REQUIRED**BUT NOT PROVIDED**

- VitalScientific reagent, used for C-reactive protein (CRP) quantitative determination (Ref. ICRP-0400).
- General Laboratory equipment.

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PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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These products are ready to use.

PRODUCT DETERIORATION

- The product should be clear. Cloudiness would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

- Store at 2-8 °C and protect from light. Do not freeze.
- Do not use after the expiry date stated on the vial label.
- These products are stable 6 weeks at 2-8°C after the first opening.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

PROCEDURE

To use CRP IP CONTROL I & II, follow the procedure described in the instructions for use of the reagent used.

LIMITATIONS

Using CRP IP CONTROL I & II has been validated with the VitalScientific systems (Analyzers and reagents used for C-reactive protein (CRP) quantitative determination (Ref. ICRP-0400)). Using any other system should be validated by the laboratory.

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BIBLIOGRAFIE

1. Kaplan, J.M., First, M.R., *Renal Function. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 567 and appendix.
2. Delaney, M.P., Price, C.P., Lamb, E.J., *Kidney Function and Disease. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 631.
3. Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 316.
4. Fossati, P., Prencipe, L., Berti, G., *Clin. Chem.*, (1983), **29**, 1494.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Ceriotti, F. *et al.*, *Reference Intervals for Serum Creatinine Concentrations: Assessment of Available Data for Global Application*, *Clin. Chem.*, (2008), **54**, 559.
7. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), 59, 263.
8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press (1997).
9. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press (1995).

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BIBLIOGRAFIE

1. Schreiber, W.E., *Iron and Porphyrin Metabolism*, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 755 and appendix.
2. Higgins, T., Beutler, E. and Dumas, B.T., *Hemoglobin, Iron and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
3. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 634.
4. Higgins, T., Clin. Chem., (1981), **27**, 1619.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzymes*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Wu, A.H.B., Tietz Clinical guide to laboratory test, 4th Ed., (W.B. Saunders eds.), (2006), 470.
3. Dufour, R., *The Liver: Function and Chemical Pathology*, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
4. Schumann, G., *et al.*, Clin Chem Lab Med., (2002), **40**, 734
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations*, WHO/DIL/LAB/99.1 Rev.2, (2002).
6. Kytzia, H.J., *Reference intervals for GGT according to the new IFCC 37°C reference procedure*, Clin. Chem. Lab. Med., (2005), **43**, A69.
7. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg. (2004), **59**, 263.
8. Young, D. S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D. S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995)

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BIBLIOGRAFIE

1. Sacks, D.B., *Carbohydrates. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 373.
2. Dods, R.F., *Diabetes Mellitus. Clinical Chemistry : Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 729 and appendix
3. Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 444.
4. World Health Organization (WHO), *Definition and Diagnosis of Diabetes Mellitus and Intermediate Hyperglycemia*, (2006).
5. Trinder, P., *Ann. Clin. Biochem.*, (1969), 6, 24.
6. Guder, W.G., *et al., Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
9. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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FTNA-HBACCa-v8 (04/2024)

CAUTION: Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner (Rx ONLY)

INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of HbA1c reagent, used for HbA1c quantitative determination on Selectra Pro Series Analyzers according to the values indicated in the value sheet included in the kit.

COMPOSITION

Calibrators are lyophilised hemolysates prepared from human erythrocytes. Stabilizers are added to maintain hemoglobin in the reduced state for the accurate calibration of the hemoglobin A1c procedure.

TRACEABILITY

Values for HbA1c Calibrator Set have been standardized from NGSP/DCCT (%) system. These values have been converted by calculation to traceable values from IFCC reference method (mmol/mol)⁽¹⁾.

REQUIRED MATERIAL BUT NOT PROVIDED

- VitalScientific HbA1c reagent
- Selectra Pro Series Analyzers
- Distilled or deionized water
- General Laboratory equipment.

PRECAUTIONS AND WARNING

- This *in vitro* diagnostic device is for professional use only.
- Take normal precautions and adhere to good laboratory practice.
- All human material should be considered as potentially infectious. All products derived from blood are prepared exclusively from the blood of donors tested individually by FDA - approved methods and found to be negative for HbsAg and antibodies to HCV and HIV1/HIV2. However as no test method can rule out the potential risk of infection with absolute certainty, handle cautiously as potentially infectious.
- Use clean or single use laboratory equipment only to avoid contamination.
- For more information, material Safety Data Sheet (SDS) is available on request for the professional user.

DISPOSAL OF WASTE

Disposal of all waste material should be in accordance with local, state and Federal regulatory requirements.

PREPARATION

Reconstitute each calibrator with 0.5 mL of deionized water. Gently mix for 10 minutes, or until all material has dissolved.

Note : As all samples, these calibrators should be hemolysed before use.

REAGENT DETERIORATION

Reconstituted calibrators which show any evidence of contamination should be discarded.

STABILITY AND STORAGE

Prior to reconstitution : When stored at -2-8 °C and protected from light, the calibrators are stable until the expiry date stated on the label.

After reconstitution : Calibrators are stable for 30 days when stored at 2-8 °C.

- After opening, the vials should be kept correctly and tightly capped to prevent contamination and evaporation.
- Do not freeze.

PROCEDURE

To calibrate with HbA1c Calibrator Set follow the procedure described in the instructions for use of the HbA1c reagent. HbA1c results for samples and controls are determined using the calibration curve obtained with 4 levels of calibrators.

CALIBRATION VALUES

The concentrations indicated in the value sheet enclosed in the kit are lot-specific. The values were determined and defined by VitalScientific on Selectra Analyzers using HbA1c reagent.

For users with Selectra TouchPro software, use the values included in the barcode available on the value sheet.

BIBLIOGRAPHY

1. Weykamp, C. et al., *Clinical Chemistry*. (2008), **54**, 240.
2. Occupational Safety and Health Standards : bloodborne pathogens. (29 CFR Part 1910.1030). Fed.Register. July 1, 2001; 17:260-273.
3. Directive 2000/54/EC. Official Journal of the European Communities N° L262 from October 17, 2000.

SYMBOLS

	Batch code
	Consult instruction for use
	<i>In vitro</i> diagnostic medical device
	Manufacturer
	Temperature limitation
	Use by
	Catalogue number
	Content
	Reconstitute with x mL
	Calibrator 1
	Calibrator 2
	Calibrator 3
	Calibrator 4
	Add exactly 0.5 mL of distilled or deionised water
	Keep at room temperature for 10 minutes before use
	These products are stable x days at x °C
	These products should be immediately and tightly capped to prevent contamination and evaporation
	Modification from previous version

HbA1c CONTROL L+H

FTNA-HBACCn-v9 (04/2024)

References :
HBAC-0049

Kit composition:
Control L : 2 x 0.5 mL Control H : 2 x 0.5 mL

CAUTION: Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner (Rx ONLY)

INTENDED USE

This *in vitro* diagnostic device is intended for accuracy control of the quantitative HbA1c reagent on Selectra Pro Series Analyzers according to the values indicated in the value sheet included in the kit.

COMPOSITION

Controls are lyophilised hemolysates prepared from human erythrocytes. Stabilizers are added to maintain hemoglobin in the reduced state providing complete control of HbA1c procedure.

TRACEABILITY

Values for HbA1c Control L + H have been standardized from NGSP/DCCT(%) system. These values have been converted by calculation to traceable values from IFCC reference method (mmol/mol)⁽¹⁾.

REQUIRED MATERIAL BUT NOT PROVIDED

- VitalScientific HbA1c reagent
- Selectra Pro Series Analyzers
- Distilled or deionized water
- General Laboratory equipment

PRECAUTIONS AND WARNING

- This *in vitro* diagnostic device is for professional use only.
- Take normal precautions and adhere to good laboratory practice.
- All human material should be considered as potentially infectious. All products derived from blood are prepared exclusively from the blood of donors tested individually by FDA - approved methods and found to be negative for HbsAg and antibodies to HCV and HIV1/HIV2. However as no test method can rule out the potential risk of infection with absolute certainty, handle cautiously as potentially infectious.
- Use clean or single use laboratory equipment only to avoid contamination.
- For more information, Material Safety Data Sheet (SDS) is available on request for the professional user.

DISPOSAL OF WASTE

Disposal of all waste material should be in accordance with local, state and Federal regulatory requirements.

PREPARATION

Reconstitute each control with 0.5 mL deionized water. Gently mix for 10 minutes, or until all material has dissolved.

Note : As all samples, these controls should be hemolysed before use.

REAGENT DETERIORATION

Reconstituted controls which show any evidence of contamination should be discarded.

STABILITY AND STORAGE

Prior to reconstitution : when stored at -2-8 °C and protected from light, the controls are stable until the expiry date stated on the label.

After reconstitution : The controls are stable for 30 days when stored at 2-8 °C.

- After opening, the vials should be kept correctly and tightly capped to prevent contamination and evaporation.
- Do not freeze.

PROCEDURE

HbA1c Control L + H are handled as samples according to the instructions for use of HbA1c reagent. These controls should be assayed together with patient samples, at least once a day and after each calibration. The control frequency should be adapted to Quality Control procedures of each laboratory and the regulatory requirements. Results should be within the defined ranges. If values fall outside of the defined ranges, each laboratory should take corrective measures. Quality control materials should be used in accordance with local, state, and/or federal guidelines.

CONTROL VALUES

The concentrations indicated in the value sheet enclosed in the kit are lot-specific.

These values have been adjusted to be in the normal range (HbA1c Control L) or in the pathological range (HbA1c Control H). The target and range values were determined under strictly standardized conditions on Selectra Pro Series Analyzers using the HbA1c reagent and HbA1c Calibrator Set.

For users with Selectra TouchPro software, use the values included in the barcode available on the value sheet.

BIBLIOGRAPHY

- 1 - Weykamp, C. *et al.*, *Clinical Chemistry*, (2008), **54**, 240.
2. Occupational Safety and Health Standards : bloodborne pathogens. (29 CFR Part 1910.1030). Fed.Register. July 1, 2001; 17:260-273.
3. Directive 2000/54/EC. Official Journal of the European Communities N° L262 from October 17, 2000.

SYMBOLS

	Batch code
	Consult instruction for use
	<i>In vitro</i> diagnostic medical device
	Manufacturer
	Temperature limitation
	Use by
	Catalogue number
	Content
	Reconstitute with x mL
	Control Low
	Control High
	Add exactly 0.5 mL of distilled or deionised water
	Keep at room temperature for 10 minutes before use
	These products are stable x days at x °C
	These products should be immediately and tightly capped to prevent contamination and evaporation
	Modification from previous version

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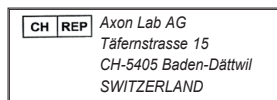
English - EN

❖INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of VitalScientific reagents, used for HDL or LDL cholesterol quantitative determination. This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum.
- The HDL and LDL concentrations of this calibrator are lot-specific.

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- VitalScientific reagent, used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)
- General Laboratory equipment.

TRACEABILITY

HDL LDL CALIBRATOR is traceable to the reference method approved by the CDC (Centers for Disease Control and Prevention).

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved or CE compliant. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilized.
- Add exactly 1 mL of distilled or deionised water.
- Carefully close the vial and dissolve the contents completely by occasional gentle stirring avoiding the formation of foam.
- Keep at room temperature for 30 minutes before use.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particulates would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

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After reconstitution :

- These products are stable 2 weeks at 2-8 °C or 10 weeks at -20°C.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

❖PROCEDURE

To use HDL LDL CALIBRATOR, follow the procedure described in the instructions for use of the VitalScientific reagent used.

❖LIMITATIONS

Using HDL LDL CALIBRATOR has been validated with the VitalScientific systems (Analyzers and reagents used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)). Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit. For users of Selectra instruments allowing data import for tests, calibrators and controls, use the corresponding barcode available on the value sheet.

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BIBLIOGRAFIE

1. Johnson, A.M., *Amino Acids and Proteins*, Tietz Fundamentals of Clinical Chemistry, 6th Ed, Burtis, C.A. & Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds), (2008), 286.
2. Sanhai, W.R., *et al.*, *Cardiac and Muscle Disease*, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby, Inc. eds.), (2010), 677 and appendix.
3. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 190.
4. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
5. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
7. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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English - EN

INTENDED USE

These *in vitro* diagnostic devices are intended for the quality control of the performances of quantitative tests listed in the value sheet. These *in vitro* diagnostic devices are for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum.
- Sodium azide < 0.1 % (w/w)
- Concentrations for each analyte to test are lot specific.

➤MATERIALS REQUIRED BUT NOT PROVIDED

- For rheumatoid factor quantitative determination : IRFA-0230/RHFA-M130.
- For anti-streptolysine-O quantitative determination : ASLO-0250.
- For C-reactive protein quantitative determination : ICRP-M230/ICRP-0400.
- General Laboratory equipment.

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PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbAg, HCV and HIV1/2. The methods used were FDA approved or CE compliant. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilizate.
- Add **exactly 1 mL of distilled or deionised water.**
- Carefully close the vial and dissolve the contents completely by occasional gentle stirring avoiding the formation of foam.
- Keep at room temperature for 10 minutes before use.

PRODUCT DETERIORATION

The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particulates would indicate deterioration. Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution). Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- **Store at 2-8 °C and protect from light. Do not freeze.**
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products are stable 10 days at 2-8 °C and 3 months at -20°C.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

➤PROCEDURE

To use RHEUMATOLOGY CONTROL I & II, follow the procedure described in the instructions for use of the VitalScientific reagent used.

➤LIMITATIONS

Using RHEUMATOLOGY CONTROL I & II has been validated with the VitalScientific systems (Analyzers and reagents listed in § MATERIALS REQUIRED BUT NOT PROVIDED). Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit. Individual laboratories may obtain values different from those announced. Technique, equipment and experimental error may produce slightly different values. Each laboratory should determine their own mean values. For tests which allow it, use corresponding barcode, available on the value sheet.

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English - EN

INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of reagents used for rheumatoid factors quantitative determination.

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COMPOSITION

- Lyophilized product prepared from human serum.
- Rheumatoid factors concentrations are lot-specific.

MATERIALS REQUIRED

BUT NOT PROVIDED

- VitalScientific reagent RHEUMATOID FACTOR, used for rheumatoid factors quantitative determination (IRFA-0230).
- Normal saline solution (NaCl 9 g/L).
- General Laboratory equipment.

TRACEABILITY

RF CALIBRATOR is traceable to NIBSC 64/002 reference material.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilizate.
- Add exactly 2 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely by occasional gentle stirring avoiding the formation of foam.
- Keep at room temperature for 10 minutes before use.

PRODUCT DETERIORATION

- The product should be clear (after reconstitution). Cloudiness would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

- Prior to reconstitution :
 - Store at 2-8 °C and protect from light. Do not freeze.
 - Do not use after the expiry date stated on the vial label.
- After reconstitution :
 - These products are stable 1 month at 2-8 °C or 3 months at -20 °C.
 - These products should be immediately and tightly capped to prevent contamination and evaporation.

PROCEDURE

To use RF CALIBRATOR, follow the procedure described in the instructions for use of the VitalScientific reagent used.

Prepare a dilution range as following:

Dilution	1	2	3	4	5	6
RF CALIBRATOR (µL)	--	25	50	100	200	400
NaCl 9 g/L (µL)	400	375	350	300	200	-
Factor	0	1/16	1/8	1/4	1/2	1

LIMITATIONS

Using RF CALIBRATOR has been validated with the VitalScientific systems (Analyzers and reagents used for rheumatoid factors quantitative determination (IRFA-0230)).

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DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements.

By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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BIBLIOGRAFIE

1. Wu, A.H.B., Clinical guide to laboratory test, 4th Ed., (W.B. Saunders eds.), (2006), 648.
2. Sanhai W. R., *Protein Isoforms : Isoenzymes and Isoforms, Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 302 and appendix.
3. Panteghini, M. D., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 317.
4. Schumann, G. *et al.*, Clin. Chem. Lab. Med., (2002), **40**, 643.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D. S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young, D. S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Dufour, D.R., *The Pancreas: Function and Chemical Pathology*, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 651 and appendix.
2. Panteghini, M. & Bais, R., *Enzymes*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
3. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 676.
5. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
7. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

- 1.-Endres, D. B. & Rude, R.K., *Disorders of Bone*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 711.
- 2.- Itani O. & Tsang, C., *Bone Disease*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 614 and appendix.
- 3.-Swaminathan, R., Clin. Biochem. Rev., (2003), **24**, 47.
- 4.- Mann, C.K., Yoe, J.H., *Spectrophotometric determination of magnesium with sodium 1-azo-2-hydroxy-3-(2,4-dimethylcarboxanilido)-naphthalene-1'-(2-hydroxybenzene-5-sulfonate)*, Anal. Chem., (1956), **28**, 202.
- 5.- Guder W.G, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
- 6.- Wu, A.H.B. Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds. Philadelphia USA), (2006), 706.
- 7.- Liamis, G. *et al.*, Am. J. Nephrol., (2013), **38**, 50.
8. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
9. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
10. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Dufour D. R., *The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
2. Wu, A.H.B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 916.
3. Dumas, B.T., *et al.*, *Clin. Chem.*, (1981), 27, 1642.
4. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*, (2002). WHO/DIL/LAB/99.1 Rev.2.
5. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), 59, 263.
6. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed, AACC Press (1997).
7. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed, AACC Press (1995).

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid Metabolism. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), *Executive Summary of the Third Report of the National Cholesterol Education Program (NCEP)*, JAMA, (2001), **285**, 2486.
4. Fossati, P. & Prencipe, L., Clin. Chem., (1982), **28**, 2077.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press (1997).
9. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press (1995).

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