


4.15 Editing Radiography Program

The radiography program in the protocol can be edited. Detailed control of the radiography condition provides the more appropriate image acquisition, and improves the operability.

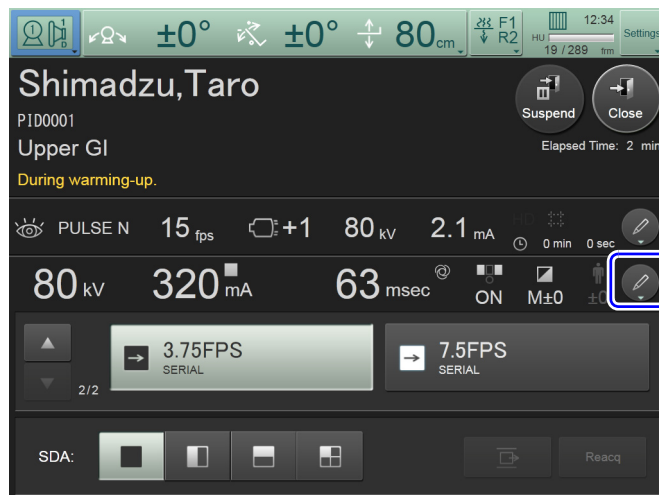
4.15.1 Outline

The radiography program edit provide a new creation, overwriting, rename, display edit, move, and delete of the program through the touch panel.

 **Hint** The DR techniques cannot be selected outside of the examination. Start a study to edit the radiography program of DR technique.



How to Edit

- 1 Press the lower  icon on the touch panel.



The radiography program is displayed.

- 2 Select the desired radiography program.

To display a radiography program that is not shown, press  or .

