


# 4.15 Editing Radiography Program

The radiography program in the protocol can be edited. Detailed control of the radiography condition provides the more appropriate image acquisition, and improves the operability.

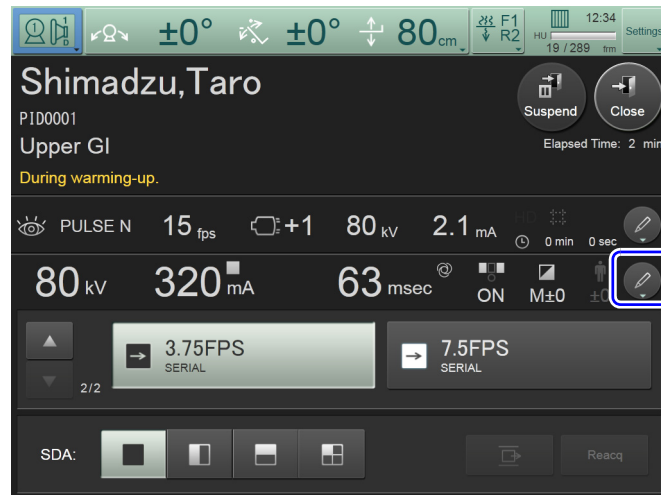
## 4.15.1 Outline

The radiography program edit provide a new creation, overwriting, rename, display edit, move, and delete of the program through the touch panel.

 **Hint** The DR techniques cannot be selected outside of the examination. Start a study to edit the radiography program of DR technique.



### How to Edit

- 1 Press the lower  icon on the touch panel.



The radiography program is displayed.

- 2 Select the desired radiography program.

To display a radiography program that is not shown, press  or .

