


## 2.2 Operation Panel (R-300)

### 2.2.1 When Combined with General Radiography Equipment

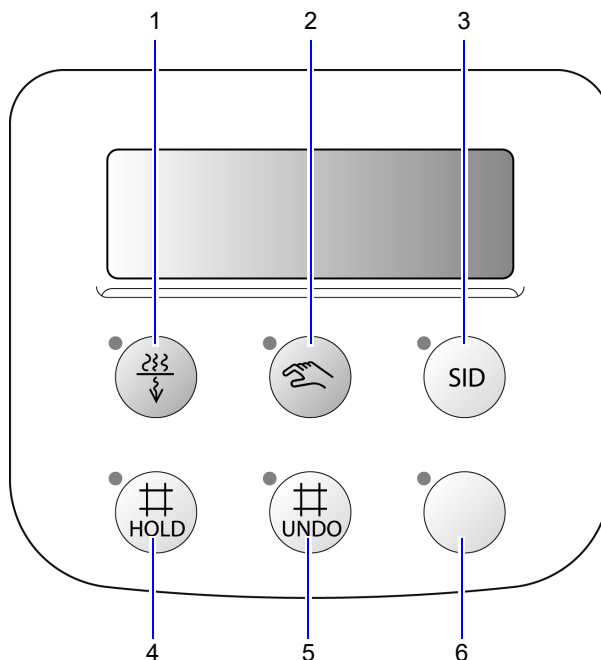

No.	Name	Function	See
1	Filter selection button	Enables you to select from among three types of X-ray filter.	P.35
2	Manual operation button	On pressing this button its LED lamp lights and the interlock with the cassette size featured on the radiography stand or radiography table is released, allowing you to set the irradiation field with this equipment.	P.23
3	SID change button	Changes the SID taken as the basis for the irradiation field in general radiography or manual operation. Successive actuations of this button cycle the SID through the sequence: 1000 → 1500 → 1800 → 2000 → 1000. The irradiation field will be calculated based on the set SID, and will be shown in the display panel.	-