

5.1.2 Searching Scheduled

Check the Scheduled search condition

- You can search the **Scheduled** items by using the search tools.
- Six search conditions are provided for searching studies.
- AND** arithmetic operation is applied to the input search condition.

Search item	Description	Input method	Partial search	Allowed characters
Patient ID	Patient ID	Entry	Possible	(,) , + , - , _ , /
Patient Name	Patient name	Entry	Possible	(,) , + , - , _ , @ , / , ' , `
Accession Number	Prescription no.	Entry	Possible	(,) , + , - , _ , @ , / , ' , ` , ~ , ! , ? , , # , ; , : , , , ,
Station Name	Exposure site	Check and input	Possible	None
Modality	Device code	Check and select one	Impossible	None
Scheduled Date	Appointment date	Select date	Impossible	None

Patient ID / Patient Name / Accession Number

- You can search one of the three items (Patient Name, Patient ID, Accession Number) or all items by selecting 'ID, Name, Acc No'.
- Input information you want to search.
 - You can input or search the items partly.
 - If you choose all three items, the search work is applied for each item.

Station Name

- Choose search condition of **Station Name** by checking the box.
- The items of search condition can be added or deleted from **Setting → Registration → Scheduled**.

Modality

- You can select search condition of **Modality** by checking the box.
- Select one of the three options - **DX, CR, DR**.

Station AE Title

- You can select search condition of **Station AE Title** by checking the box.
- The items of search condition can be added or deleted from **Setting → Registration → Scheduled**.



- Go to **Setting → DICOM → General** to set the default value of **Station Name** and search condition of **Station AE Title**.