

Products not unpacked shall be stored in the range -30 °C to +40 °C and the relative humidity shall not be higher than 80%. The term of validity is 2 years from the date of production. Disposal Guidance: Throw away.

Executive standard: EN149:2001+A1:2009 Production Date: See the certificate
AFŞARLAR İŞ ELBİSELERİ İŞ GÜV. EKİP. SAN. VE TİC. LTD. ŞTİ.

Address: Yavuztürk Mah. Hoşgörü Sok. 1B Üsküdar / İSTANBUL Service Hotline: +90216 318 86 58 www.securex.com.tr



CE mark and Notified
Body No



See the information provided
by the manufacturer



The temperature range of
storage conditions



Maximum relative humidity of
storage conditions



Keep out of direct sunlight



Single use

Importer: Mopptex GmbH, www.mopptex.com



4. Pull the straps over the ears and adjust the mask until the

3. Pull the straps on both sides and hang on the ears, and adjust till you feel comfortable.

2. Hold the mask on your face and against you chin.

EN - 1. Face the mask without the nose clip, and the nose clip should be on top.

5. Dichtsitz der Maske prüfen, bei Luftaustritt erneut anpassen.

4. Seitliche Bänder optimal einkelemen

3. Nasenclip mit den Fingern gut auf Nase andrücken und verschließen

2. Mit dem Kinn einsteigen, danach Clip-Seite über die Nase ziehen.

DE - 1. Maske aus der Verpackung nehmen, Nasenclip muss über die Nase angelegt werden.

TR

ENG

