

# RENO-S130D

## PM (Preventive Maintenance) KIT LIST

2022 year



(#14452) 2nd-5th Floor, 54, Samjak-ro 133beon-gil,  
Bucheon-si, Gyeonggi-do, Republic of Korea  
T. 032-222-0140 / F. 032-222-0110 / E-mail. cs@renosem.com

**RENO-S130D** Sterilizer requires regular maintenance activities based on number of cycles and duration specified on each PM Kit 1, PK Kit 2, PM Kit 3. PM Maintenance is performed by field service engineers of RENOSEM or Authorized distributors who have received RENOSEM-approved training and holding its certificate. PM Kit 1 is required every 6 months or 600 cycles, PM Kit 2 is required in every 12 months or 1200 cycles, PM Kit 3 is required in every 24 months or 2,400 cycles whichever comes faster. RE S130D also shows PM alarm when it is required.

**Example** 2 year PM Kits ( PM Kit 1- 4sets, PM Kit 2- 2sets, PM Kit 3- 1set)

No	PM KIT Number Description	Image	Qty	Unit	Cycle	Year	
<b>S130D PM KIT-1(Every 600 cycles)</b>							
PM06-0010-00 , S130D PM KIT-1-1 (LEYBOLD, Option 1)							
PM06-0020-00 , S130D PM KIT-1-2 (WOOSUNG, Option 2)							
1-1	Option 1 Rotary Pump Oil 1 L (Leybold)		1	Bottle	600	0.5	
1-2	Rotary Pump Oil 1 L (WooSung)		3	bottle	600	0.5	
2	Activated Carbon Filter		1	set	600	0.5	
3-1	Option 1 O-ring (Inlet & Outlet) (Leybold)		1	set	600	0.5	※ <b>Remark</b> Replace PM KIT-1 every 6 month or 600 cycles, whichever comes first.
3-2	Option 2 O-ring (Inlet&Outlet) (WooSung)		1	set	600	0.5	
4	Oil Paper & Glove		1	pair	-	-	※ <b>Cleaning</b> Injection port hole cleaning should be performed every 600 cycles.
5	3L Oil container		1	ea	-	-	
6	Funnel		1	ea	-	-	
7	Garbage Vinyl		1	ea	-	-	
<b>S130D PM KIT-2(Every 1,200 cycles)</b>							
PM06-0030-00 , S130D PM KIT-2-1 (LEYBOLD, Option 1)							
PM06-0040-00 , S130D PM KIT-2-2 (WOOSUNG, Option 2)							
PM06-0060-00 , S130D PM KIT-2-3 (WOOSUNG, Option 3)							
1-1	Option 1 Oil Mist Trap (Leybold)		1	set	1200	1.0	