





## **endoCleaner**

endoscope washer & disinfectant

- Effective cleaning and disinfection with Full traceability
- Environmental Friendly and Cost Efficiency (10 liter disinfectant & less water)
- Compatible with Olympus, Pentax, Fujinon, Storz endoscope
- Compliant with ISO 15883-1 and 15883-4.

**Process**

**RFID &  
Traceability**

**Touch  
Screen**

**Leak Test**

**Disinfectant  
and  
Enzymatic**

**EndoMonitoring**

**Details**






Ability to follow the whole Process;

- Prewash
- ↓
- Washing with enzymatic
- ↓
- Rinsing enzymatic
- ↓
- Disinfection
- ↓
- Rinsing disinfection
- ↓
- Air + alcohol (optional)



## Full traceability of endoscopes

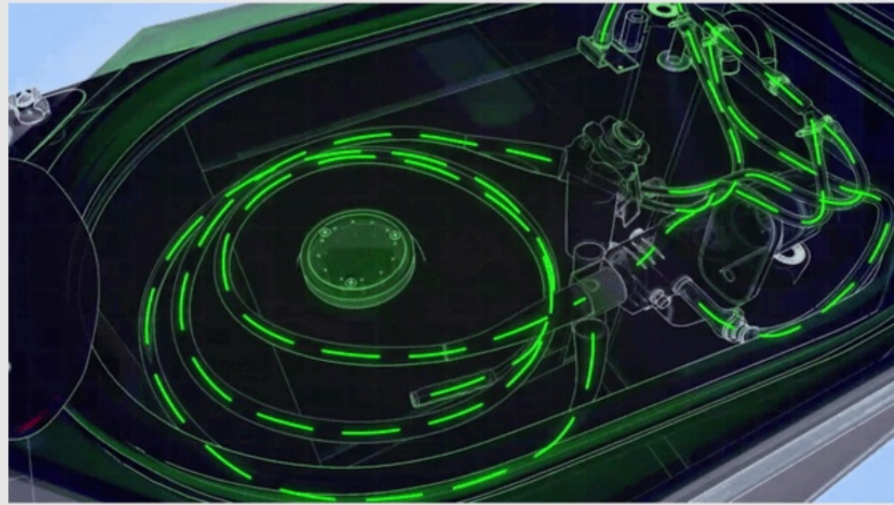

- Reads RFID tag for endoscopy
- Reads RFID card for user
- Disinfection traceability
- Traced in detail, with a report sent to an internal printer



RFID Tag



Personel Card  
RFID



Printer




## Touch screen LCD



- Color touch screen.
- Easy to use interface.
- Follow the whole washing and disinfection process.



## Main Menu Definitions

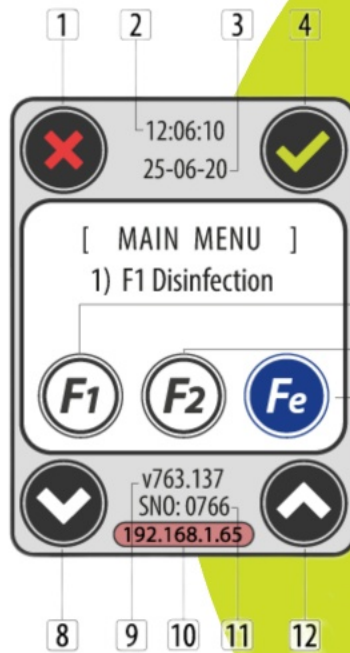

- 1 Cancel button
- 2 Time
- 3 Date
- 4 Date button
- 5 F1 function 1
- 6 F2 function 2
- 7 Fe function extra ( F3 to F10 )
- 8 Down arrow button
- 9 Version number
- 10 IP bar
- 11 Serial number
- 12 Up arrow button



## Disinfectant Program Tracing Screen Definitions



1 Disinfection process bar

2 Live progress indicator

3 Live leak test indicator



## Digital automatic leakage test during whole stages



- Cost reduction with early leak detection
- Even the smallest leak detectable early
- Prevent cross contamination
- 2 types of Pressure Indicator
  - mmHg: millimeter of mercury unit
  - mbar : milibar unit



## Disinfectant and Enzymatic

- Washing process completes with 10 liter disinfectant
- Non-foam disinfectants can be used
- Disinfection quality keep with endoFilter

Internal  
Concentrated  
Disinfectant

Original Pack  
Concentrated  
Disinfectant

Ready to Use  
Disinfectant

### Our customers preferences




*Johnson & Johnson*



**ECOLAB®**




## Internal Concentrated Disinfectant

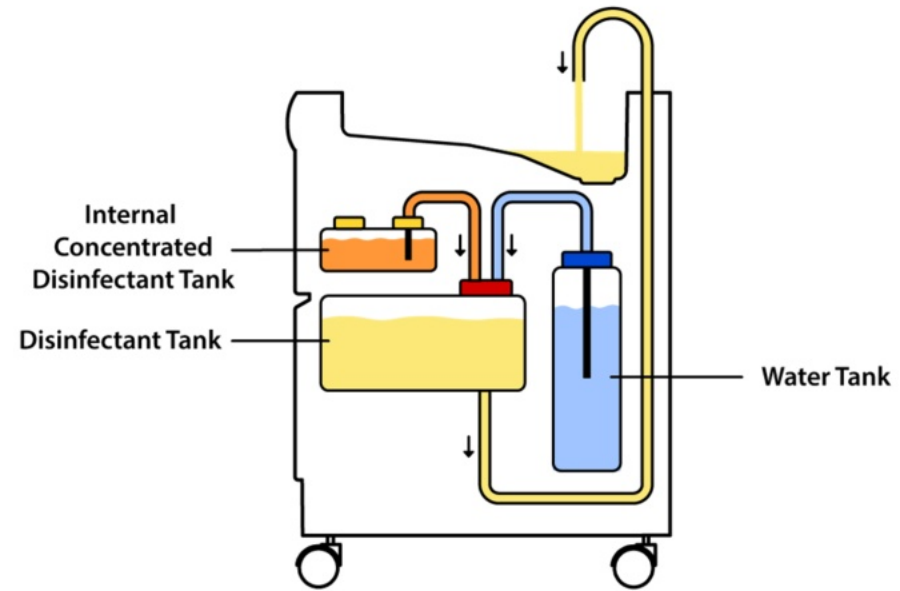




## Original Pack Concentrated Disinfectant

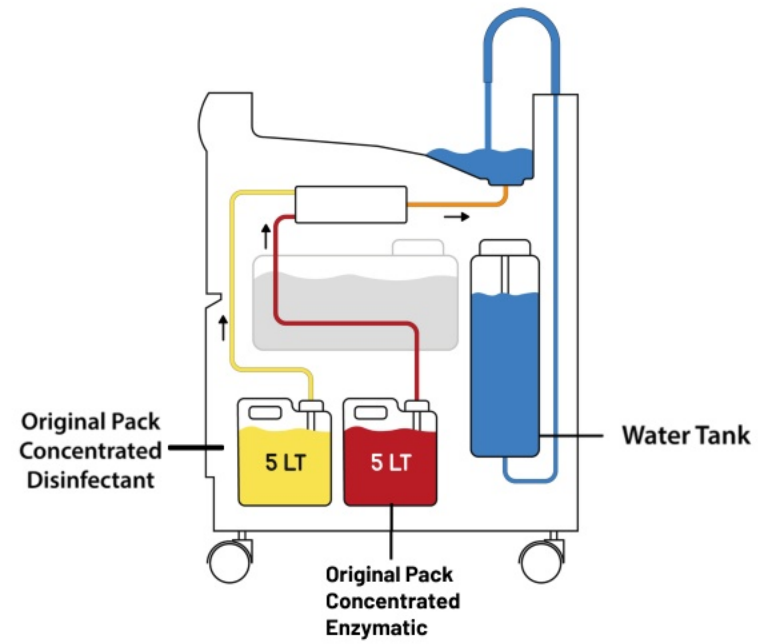




## Original Pack Concentrated Disinfectant

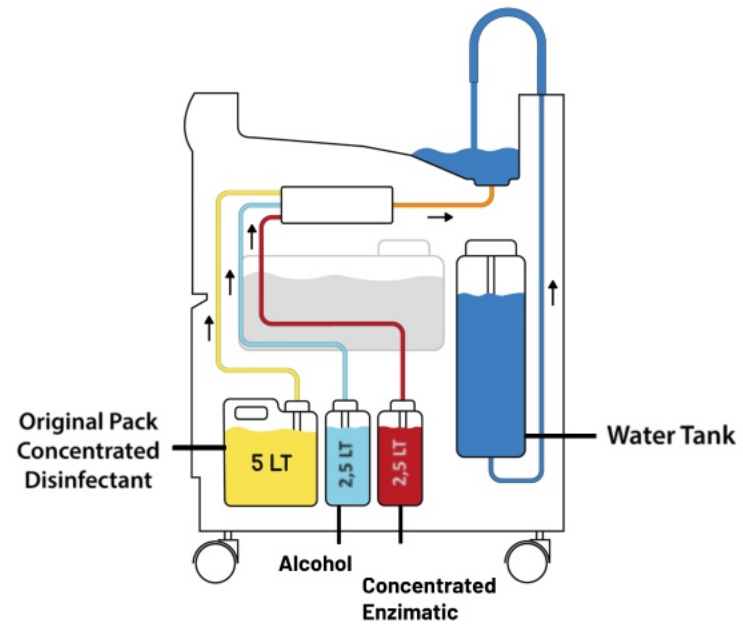



## Ready to Use Disinfectant

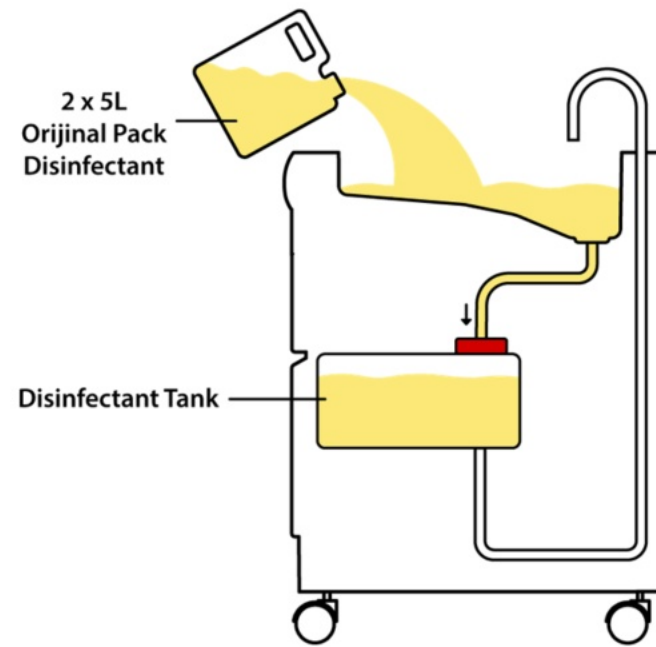






- Real time monitoring
- Quick data access
- Remote Access via tablet PC and smart phone
- Detailed process reporting
- Technical service training
- Low service cost
- Compatible with all our products
- Fault detection with remote access
- Ability to access detailed process reports and service information



## Time Table

			F1 (sn)	F2 (sn)	F3 (sn)	F4 (sn)	amount of liquid
A	On Yikama Pre-washing	Su doldurma Water Filling		65	65	65	10 lt
		Yikama Washing		60	60	60	10 lt
		Su boşaltma water drain		65	65	65	10 lt
		Kanal havası Channel air		10	10	10	2.5 lt
B	Enzimatik Enzymatic	Su doldurma Water Filling			65	65	10 lt
		Dozlama The dosing			60	60	Optional
		Yikama Washing			60	60	10 lt
		Enzimatik boşaltma Enzymatic drain			65	65	10 lt
		Kanal havası Channel air			10	10	2.5 lt
C	Enzimatik Durulama Enzymatic Rinse	Su doldurma Water Filling			65	65	10 lt
		Yikama Washing			60	60	10 lt
		Su boşaltma water drain			65	65	10 lt
		Kanal havası Channel air			10	10	2.5 lt
D	Dezenfektan Disinfectant	Dezenfektan doldurma Disinfection filling	50	50	50	50	10 lt
		Yikama Washing	300	300	300	300	9.9 lt
		Dezenfektan boşaltma Disinfection drain	65	65	65	65	9.9 lt
		Kanal havası Channel Air	10	10	10	10	2.5 lt
E	Dezenfektan Durulama Rinsing Disinfectant	Su doldurma Water filling	65	65	65	65	10 lt
		Yikama Washing	60	60	60	60	10 lt
		Su boşaltma Water drain	65	65	65	65	10 lt
		Kanal havası Channel air	10	10	10	10	2.5 lt
F	Alkol Alcohol	Dozlama The dosing				15	Optional
		Kanal havası Channel air				10	2.5 lt
TOPLAM (sn)TOTAL Sec))			625	825	1.285	1.310	
TOPLAMTOTAL (mn, sec)			10dk41sn 10mn41sec	13 dk 75sn 13mn75sn	21dk41 sn 21mn41sn	21dk 83 sn 21mn50sec	






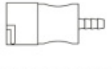


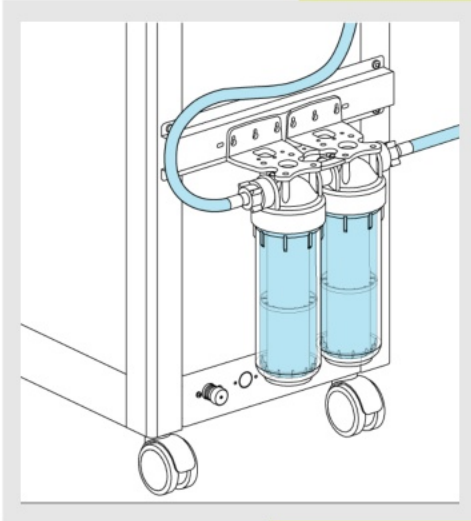







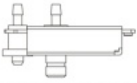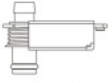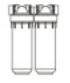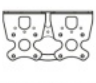










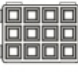






- Water filter is placed on backside of the device.
- 1 and 5 micron filter
- 0.2 micron filter optional



## Accessories

 1/2	 1/2	 1/20	 1/10	 1/10
Handle connector	Basin connector	Water filter	Water filter hanger	Water filter key
 1/2	 1/20	 1/1	 1/1	 1/10
Leakage test connector	Basin connector hose assembly	6 x Screw YSB 5x12mm	6 x Inox nut M5	1 micron filter
 1/5	 1/10	 1/10	 1/20	 1/20
Leakage test hose	Funnel	0.2 micron filter	2 x Water inlet hose	Drain hose
 1/1	 1/10	 1/5	 1/5	 1/2
8xScrew YSB 4,8X25	Repair Kit	USB technic kit	Cable repair kit	Printer paper
 1/10	 1/5	 1/10	 1/20	 1/2
5 micron filter	Allen key	User Guide	User Instruction	RFID tag (Optional)
 1/2	 1/5	 1/5	 1/5	
Clamp Y26	RFID personnel Card (Optional)	Button washing basket (Optional)	Endofilter (Optional)	



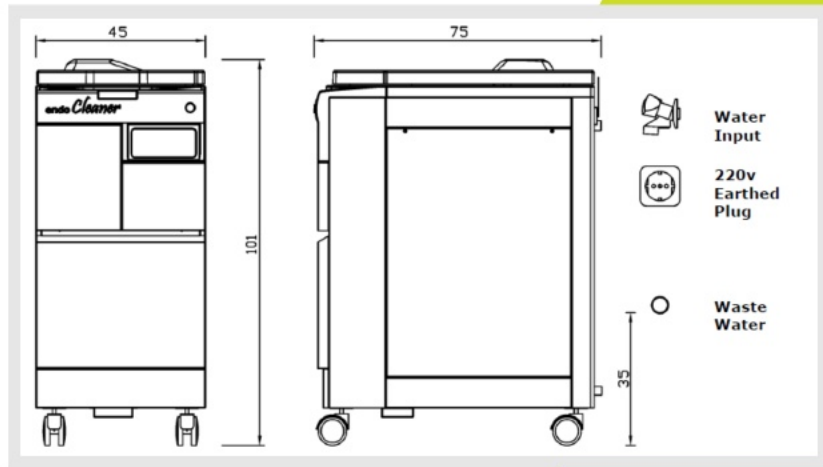

- Dimensions: 45 x 101 x 75 cm
- Brut Weight :100 kg
- Water input: At the tap for washing machines
- AC input: Mains Voltage 220-230V 50Hz
- Waste water: Maksimum 35 cm





- Delivery Term: EXW
- Brut Weight: 130 kg
- Dimension: 111(l) x 72 (w) x 130 (h)cm





**WE LOOK FORWARD TO YOUR COOPERATION TO DEVELOP OUR BUSINESS TOGETHER**



**WEB**

[www.aort.net](http://www.aort.net)

**E-MAIL**

✉ [info@aort.net](mailto:info@aort.net)

✉ [ayla@aort.net](mailto:ayla@aort.net)

✉ [dalim@aort.com.tr](mailto:dalim@aort.com.tr)

✉ [birsen@aort.net](mailto:birsen@aort.net)

✉ [sinem@aort.com.tr](mailto:sinem@aort.com.tr)