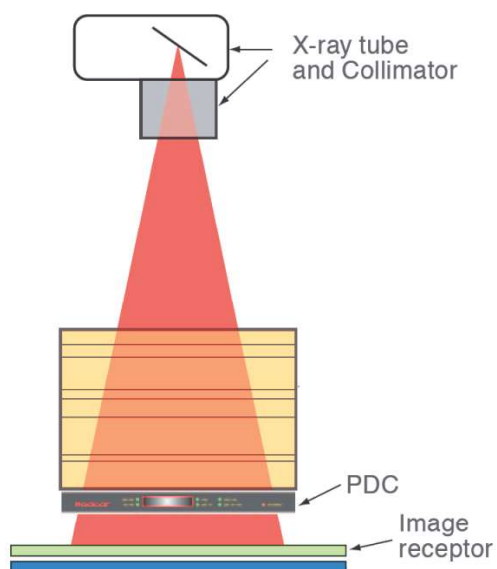

■ DAP Table

1. Measurement method



- Image receptor dose measurement.

- 1) Power-on PDC(Patient Dose Calibrator).
- 2) Prepare the DAP TEST Equipment (PDC) at the image recaptor location as shown above and proceed with the test air kerma condition.
- 3) Conduct the test according to the survey conditions for each selectable mode, and the maximum and minimum inspection conditions for the tube voltage, tube current, and tube voltage, as indicated in the result.
- 4) When the system detects radiation, the display automatically displays the dose or dose zone generation rate.