

TECHNICAL DATA SHEET

Dok.No:FR.05.01.06 /Y.T: 18.04.2023 / Rev T:16.10.2024 / Rev2
The product TDS document can be updated by the authorized unit.

DETRO® PAA 1500



PRODUCT DESCRIPTION:

High Level Medical Device and Endoscope Disinfectant

PRODUCT PROPERTIES:

Ready to use product consisting of base and activator.
High level disinfection.
Aldehyde and phenol free.

PHYSICAL AND CHEMICAL SPECIFICATIONS:

Appearance :Liquid
Color :Colorless
Odor :Characteristic

CHEMICAL COMPOSITION:

Detrox PAA 1500 Base:Hydrogen peroxide;%3, stabilizer, pH balancer, deionized water.
Detrox PAA 1500 Activator: N-acetylcaprolactam, solvents, corrosion inhibitor.
Active Ingredient: 1500 ppm peracetic acid formed by the reaction by mixing the base and activator

USAGE AREAS:

Detrox PAA 1500 is used for high level disinfection and cold sterilization of stainless steel surgical instruments, anesthesia and dental surgical instruments, rigid and flexible endoscopes, medical instruments made by plastic, silicon, glass, acrylate and rubber by immersion method.

MICROBIOLOGICAL EFFECT:

Bactericidal, fungicidal, tuberculocidal, sporicidal and virucidal effect in 5 min.

TECHNICAL DATA SHEET

Dok.No:FR.05.01.06 /Y.T: 18.04.2023 / Rev T:16.10.2024 / Rev2
The product TDS document can be updated by the authorized unit.

USAGE:

Add activator into a 5-liter bottle includes base and shake gently. Wait at least 10 minutes. It is ready to use after activation. Detro PAA 1500 is filled into a clean and closed container. Pre-cleaned and rinsed instruments are immersed thoroughly into solution. Container is closed. Wait for effect time. After effect time, rinse with sterile water or distilled water. The effectiveness of the solution is controlled by single-use test strips. Renew the solution if the test strip fails on colour change.

STORAGE:

Store containers tightly closed and in a well ventilated area between 0-25°C.

SAFETY:

In case of eye contact flush immediately with large amounts of fresh water at least 10 minutes. In case of inhalation, shut off source of exposure. Bring the victim to the fresh air. In case of skin contact, take off contaminated clothes and shoes and wash thoroughly with water and soap. In case of ingestion, rinse mouth with potable water.

PACKING:

5 L x 4 units