


2.2.2 X-ray Diagnostic Table Front Control Panel

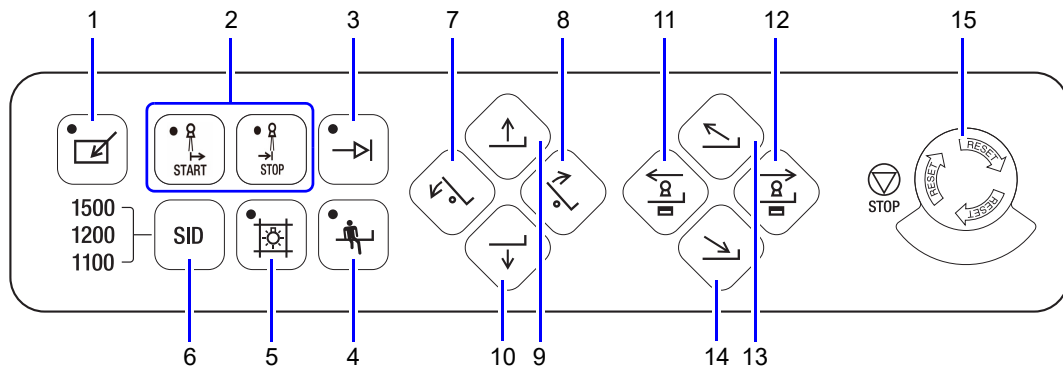

No.	Form	Name	Function
1		Operational button	When this button is lit, the X-ray diagnostic table can be operated via the front control panel. Reference P.4-53
2		Start point button (optional) ^{*1}	Sets the start and stop points in the SLOT radiography (Option). Reference P.4-181
		Stop point button (optional) ^{*1}	
3		Set button ^{*2}	Used with the position memory function. Reference P.4-66 Moves the X-ray tube and FPD to the radiography start point in the SLOT radiography (option). Reference P.4-181
4		Tabletop return button	The tabletop returns to the horizontal position and to the height set when it was installed. Reference P.4-66
5		Collimator lamp button	The collimator irradiation field lamp comes on for 30 seconds. If you press this button while the irradiation field lamp is on, the lamp will be turned off. Reference P.4-81
6		SID switchover button	Sets the distance of the FPD input surface from the focal spot of the X-ray tube unit. [*] For the system with the SID 1.8MT, the option is different. Reference P.4-65
7		Reverse tilt button ^{*2}	The tabletop tilts in the opposite direction. Reference P.4-54
8		Tilt button ^{*2}	The tabletop tilts in a vertical direction. Reference P.4-54
9		Raise tabletop button ^{*2}	Raises the tabletop. Reference P.4-60

No.	Form	Name	Function
10		Lower tabletop button *2	Lowers the tabletop. Reference P.4-60
11		Imaging unit shift button (towards the head) *2	The imaging unit moves towards where the patient's head is. Reference P.4-64
12		Imaging unit shift button (towards the feet) *2	The imaging unit moves towards where the patient's feet are. Reference P.4-64
13		Tabletop left shift button *2	The tabletop moves towards the patient's left. Reference P.4-58
14		Tabletop right shift button *2	The tabletop moves towards the patient's right. Reference P.4-58
15		Emergency stop button	Brings the movement of the tabletop to an emergency stop. Reference P.5-2

*1: In the Smart FOV mode (option), these buttons are not available.

Reference "4.12.3.1 Smart FOV (Option)" P.4-130

*2: Only moves while the button is being pressed.

2.2.3 Collimator Control Panel

Control Panel

