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
INSTALLATION INSTRUCTIONS

MAIN FEATURES

DATOSPIR *aira* is a portable device available with different kinds of transducers: Fleisch, Turbine or disposable Lilly.


It allows real-time connection via **Bluetooth** or **USB** to any PC (Personal Computer) running **W20s Spirometry Software**.

DATOSPIR *aira* has 4 models: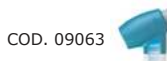

COD. 09063

DATOSPIR AIRA F

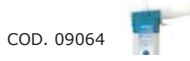COD. 09064

DATOSPIR AIRA D

COD. 09065

DATOSPIR AIRA T

COD. 09066

DATOSPIR AIRA BASIC T

INSTRUCTIONS FOR USE

Install the **W20s Spirometry Software** to use the device with a PC. Consult the **User's Manual of W20s Software**.

Then, connect the USB cable with  the connector indicated with and the other side to the computer.

If you have acquired a model with Bluetooth, connect the **Bluetooth adapter** to the PC and install the included software. Consult the user's manual of the Bluetooth adapter.

USE OF THE DEVICE


Start the W20s Software and connect the device by Bluetooth or USB. Then, select the Spirometer model and the type of connection in the Links screen (consult the W20s User's Manual).

To turn the device on or off, press and hold the key  until the green LED lights up and release the key.

To save energy, the equipment includes an auto-off system that shuts down the equipment after three minutes without communications with the W20s spirometry software.

1. TRANSDUCER PLACEMENT (See Figure 1 on page 2).
2. MANEUVER PERFORMANCE (See Figure 2 on page 2).


CONTROL AND CONNECTORS DISTRIBUTION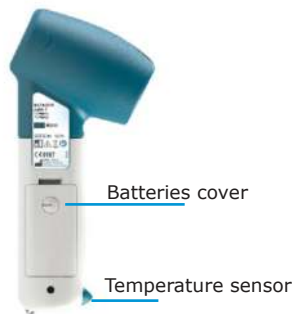

BATTERY: INSTALLATION AND CHARGING

DATOSPIR *aira* can work with two AAA alkaline batteries or two rechargeable Ni-Mh 1.2V 800mAh batteries. In both cases, the approximate autonomy with normal use is 5 days.

Use a coin to turn the screw to unblock the battery cover to remove the battery. Then, replace them according to the polarity indicated on the housing.

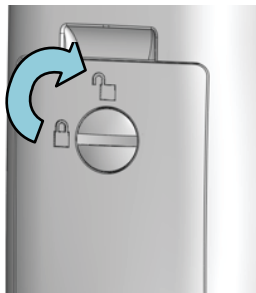VISUAL INDICATIONS AND WARNINGS

Indicator	Warning	Description
	Device on	
	Activated communications	Transferring data
	Recovery mode	It allows updating the firmware (only USB)
	BIOS Mode	It allows restoring the firmware when the communication is interrupted (USB only)
	Measurement error	Internal measurement error. Restart the device or contact Technical Support
	Low battery	Replace the batteries

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SIBEL S.A.U.