## Type (Class) 6 Kayıt Kartı Type (Class) 6 Registration Card

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*4A Type 6 Registration Cards are the products that are put into each sterilization package and that indicate success of the sterilization. 4A Type 6 Registration Cards are cycle verification indicators that confirm the presence or absence of specific time, temperature and pressure parameters during a cycle.*

*Type 6 Steam Chemical Indicator Registration Card 3,5 minutes operates up to 15 minutes at 121°C, up to 3,5 minutes at 134°C. It has additional space for adding notes. Thanks to its self adhesive property it is easy for documenting sterilization results.*

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*4A Bowie Dick Test Packs are the testing packages that should be used before starting daily machine sterilization process for steam sterilization devices. Bowie Dick Test Pack is used for the purpose of testing whether there is air leakage and efficient steam penetration in the steam sterilizers.*

*4A Bowie Dick Test Packs color change allows assurance of whether the steam sterilizers are ready for the sterilization or not.*

*Different kinds of Bowie Dick Test Packs are available in 4A production. These are 4A Bowie Dick Test Pack 200 (wrapped in medical paper, wrapped in crepe paper and BD in box) and 4A Bowie Dick Test Pack 400 (wrapped in medical paper and wrapped in crepe paper) and 4A Bowie Dick Test Pack EWS (Early Warning Test Sheet)*

Note: Products can be produced with your trademark

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**4APCD / 4AHLX**

4A PCD- Helix Load Control Test has a self adhesive indicator on the far end. The PCD test kit has a 1 m long hose, and the Helix test kit has a 1.5 m long hose. There are 100 indicators in each package.

In this product used for load control, the adhesive indicator is placed at the far end of the test apparatus, steam inlet only can be at the beginning of the hose and it can reach the chemical indicator at the end of the hose by passing at a single point.

If the chemical indicator color change is achieved despite the difficult conditions, it is an important reference for the sterilization process in that load and should be recorded as a record for the successful sterilization of the load.

Types of 4A PCD - Helix Load Control Tests are as follows:

PCD - Helix Load Control Test with Bowie Dick Indicator

PCD - Helix Load Control Test with Type 6 Indicator

PCD - Helix Load Control Test with Ethylene Oxide Indicator

PCD - Helix Load Control Test with Plasma Indicator

**4a medical**<sup>®</sup>  
producing health for the world

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Biological indicators are test systems containing viable microorganisms providing a defined resistance to a specific sterilization process.

A biological indicator provides information on whether necessary conditions were met to kill a specified number of microorganisms for a given sterilization process, providing a level of confidence in the process.

Biological indicators, depending on the specific type, can be used for various sterilization processes using steam, hydrogen peroxide gas or

ethylene oxide.

We have the following 4A Biological Indicator types:

4A Steam Biological Indicator (3h) is specifically designed for the monitoring steam sterilization processes at 121 - 134 °C. The final readout of a processed biological indicator with respect to the indicator type should be made after 3 hours (4ARSBI) or 24h (4ASBI) of incubation at 60+/- 2 °C.

4A Ethylene Oxide Biological Indicator (4AEOBI) is specifically designed for the monitoring of ethylene oxide (EO) sterilization processes. The final readout should be made after 48 hours of incubation at 37+/- 2 °C.

4A Plasma Biological Indicator (4APBI) is specifically designed for the monitoring of Hydrogen Peroxide sterilization processes. The final readout should be made after 24hours of incubation at 60+/- 2 °C.

**4a medical**®  
producing health for the world

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4A Ethylene Oxide Indicators which are Type 5 indicators are chemical indicators which gives information about success of sterilization by using inside sterilization pouches together with ethylene oxide.

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4A Formaldehyde Indicators are indicators used inside sterilization pouch in formaldehyde sterilization.

These indicators which are included in Type 4 according to ISO 11140-1 standarts give information to user whether material inside the package is sterile or not.

**4a medical**<sup>®</sup>  
producing health for the world

## Hidrojen Peroksit (Plazma) Kimyasal İndikatörü Hydrogen Peroxide (Plasma) Chemical Indicators

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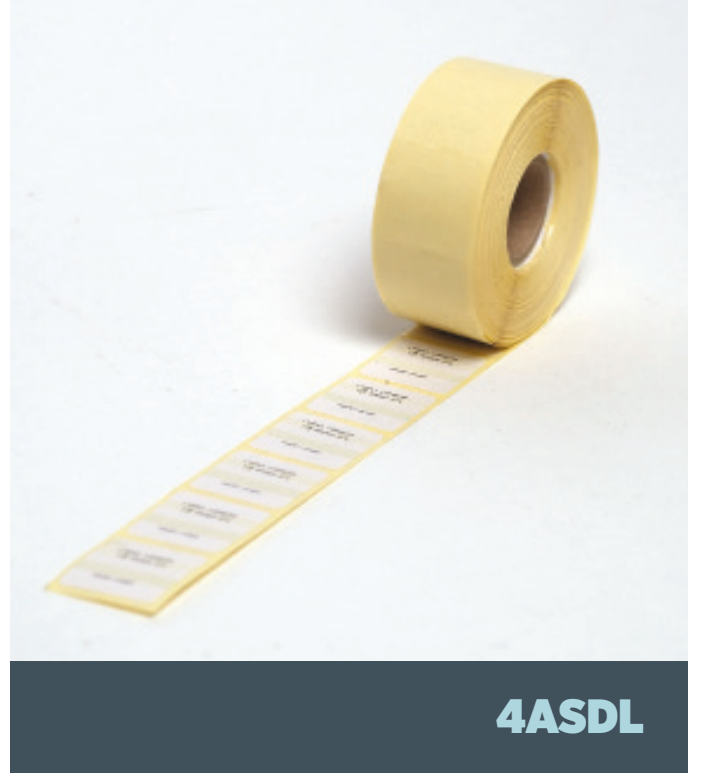
4A Plasma Indicators which are Type 4 indicators are chemical indicators which are used in sterilization inside package for sterilization with hydrogen peroxide.

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4A Steam Documentation Labels are labels with double adhesive support and process indicator for steam sterilization. Each roll contains 500 pieces of double self adhesive stickers. The indicator printed on the sticker is indicator Type 1.