


DDN 9

NEUTRAL DISINFECTANT DETERGENT FOR THE
PRE-DISINFECTION OF MEDICAL DEVICES

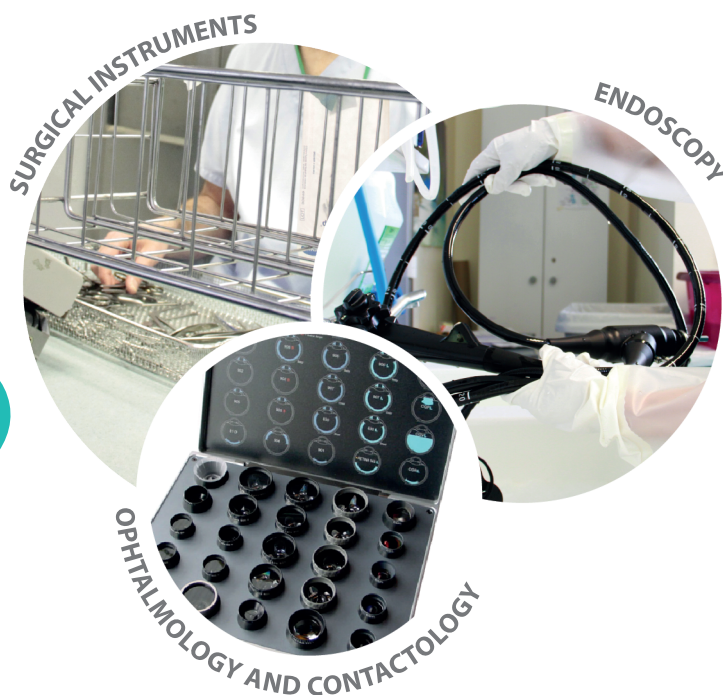

■ AREAS OF APPLICATION

- Operating room, endoscopy, ophthalmology and contactology

■ PRODUCT ADVANTAGES

- Proven efficacy on *Pseudomonas aeruginosa* biofilm
- Instrumentation integrity guaranteed for 72 hours in immersion-bath
- Compatible with a wide range of materials
- Can be used in ultrasounds vats
- Effective on enveloped virus (including corona virus)



0.5%



NEUTRAL DISINFECTANT DETERGENT FOR THE PRE-DISINFECTION OF MEDICAL DEVICES

INSTRUCTIONS FOR USE

Preparation of the solution at 0.5%:

- Wear gloves, goggles and apron
- Dilute 25 ml of DDN 9 in 5L of water and homogenize the solution

Use:

- 1- Soiled instruments have to be fully immersed
- 2- Scrub medical devices and use a swab if necessary
- 3- Contact time : 15 minutes
- 4- Rinse thoroughly

Renew the solution after use

DISINFECTING PROPERTIES

Activity	Norms	Concentration	Contact time
Bactericidal	EN 13727	0.5%	5 minutes
	EN 14561	0.5%	15 minutes
	EN 13697	0.5%	5 minutes
	EN 1276	0.25%	5 minutes
Yeasticidal	EN 13624	0.5%	5 minutes
	EN 14562	0.5%	15 minutes
	EN 1650	0.25%	5 minutes
	EN 13697	0.5%	5 minutes
Virucidal	EN 14476 <i>Virus PRV (hepatitis B model),</i>	0.5%	5 minutes
	EN 14476 + A1 <i>Virus DBV (hepatitis C model),</i>	0.5%	15 minutes
	<i>Virus VIH</i>	0.5%	15 minutes
	EN 14476 + A2 <i>Herpes Virus</i>	0.5%	5 minutes
	EN 17111		
	<i>Vaccinia Virus</i>	0.5%	15 minutes

PRECAUTIONS OF USE

See (SDS 010449)

FEATURES

- Form: **Liquid**
- Aspect : **Opalescent**
- pH at 0.5%: 7
- Expiration date: 2 years

COMPOSITION

- Non-ionic surfactants
- Sequestrants
- Quaternary ammonium

PACKAGING

- ① Box of 250 doses of 25 ml.....ref. **1044947**
- ② Box of 12 x1 L.....ref. **1044916**
- ③ Box of 4 x5 L.....ref. **1044920**



Hazardous product - Follow precautions of use. Always read label and product information before use. Medical device Class IIb.