

Ш

Ш

I

ഗ

4

D A

4

0

Z

I

0

Ш

TECHNICAL DATA SHEET

BOWIE DICK TEST PACK

PRODUCT CODE : 4ABD400, 4ABD200

RELATED STANDARDS: Bowie Dick Test Pack is classified as Type 2 according to ISO 11140-1

standard.

INTENDED USE : Bowie Dick Test Pack is used for the purpose of testing enough air

discharge and efficient steam penetration in the steam sterilizers.

OPERATION CONDITIONS: Bowie Dick Test Pack is used in steam autoclaves for Bowie Dick programs

at 121°C for 15 minutes and 134°C for 3,5 minutes.

CHEMICAL COMPOSITION: The sterilization indicator ink used for printing contains following

components in ppm levels

 Water
 35%
 (CAS # 7732-18-5)

 Diacetone Alcohol
 3%
 (CAS # 123-42-2)

 Proprietary Mixture
 62%
 (CAS # N/A)

Before Sterilization

After Sterilization



It is made of crepe paper and indicator. 12,5x12,5 filling paper, 1 indicator printed test sheet in the middle and 1 indicator printed control (medical) paper. Indicator printed test carton sheet is covered by film. 2 different

types are available: Bowie Dick Test (400) with 400 filling paper and Bowie Dick Test (200) with 200 filling paper. 4. Production date and

expiry date are placed on the package.

STORAGE AND DISPOSAL The expired products are not used. It's kept in the places with room

temperature without humidity and sun lights. It shouldn't be in contact with

liquid. Damaged and defective products are not used. Shelf life is 5 years after production date. After sterilization indicator keep sterilization colour

for 6 month.

PACKAGING : 50 pcs of Bowie Dick Test (400) in one carton box; 100 pcs of Bowie Dick

Test (200) in one carton box.

MANUFACTURER : DORT-A TIP MALZEMELERI SAN. ITH. IHR. TIC. LTD. STI.

Balıkhisar Mahallesi, Köyiçi Serpmeleri, No:795 Akyurt, Ankara, Turkey

Tel: +90 312 363 50 52 / 53 Fax: +90 312 363 52 10

Web: http://www.4amedical.com E-mail: export@4amedical.com