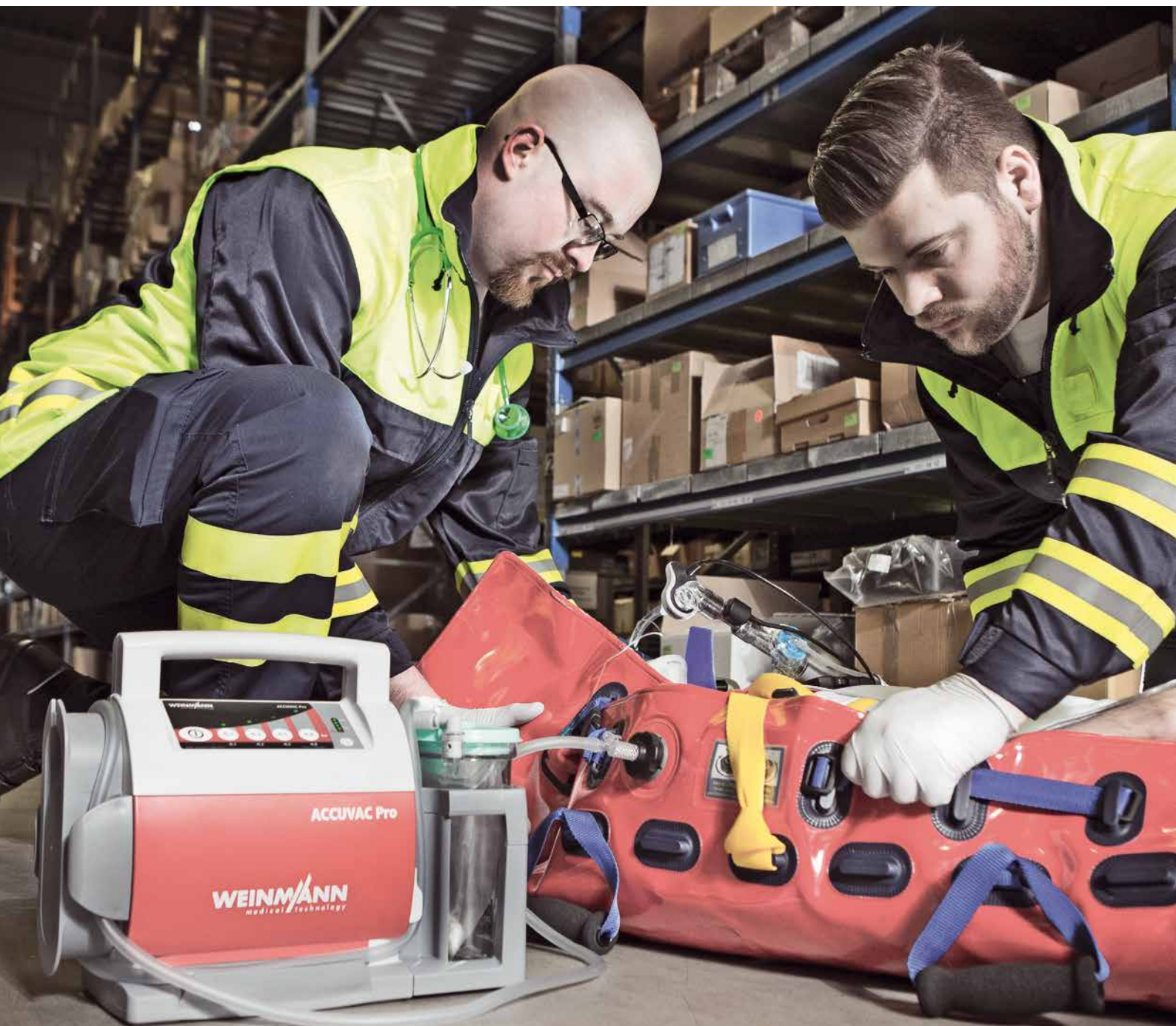
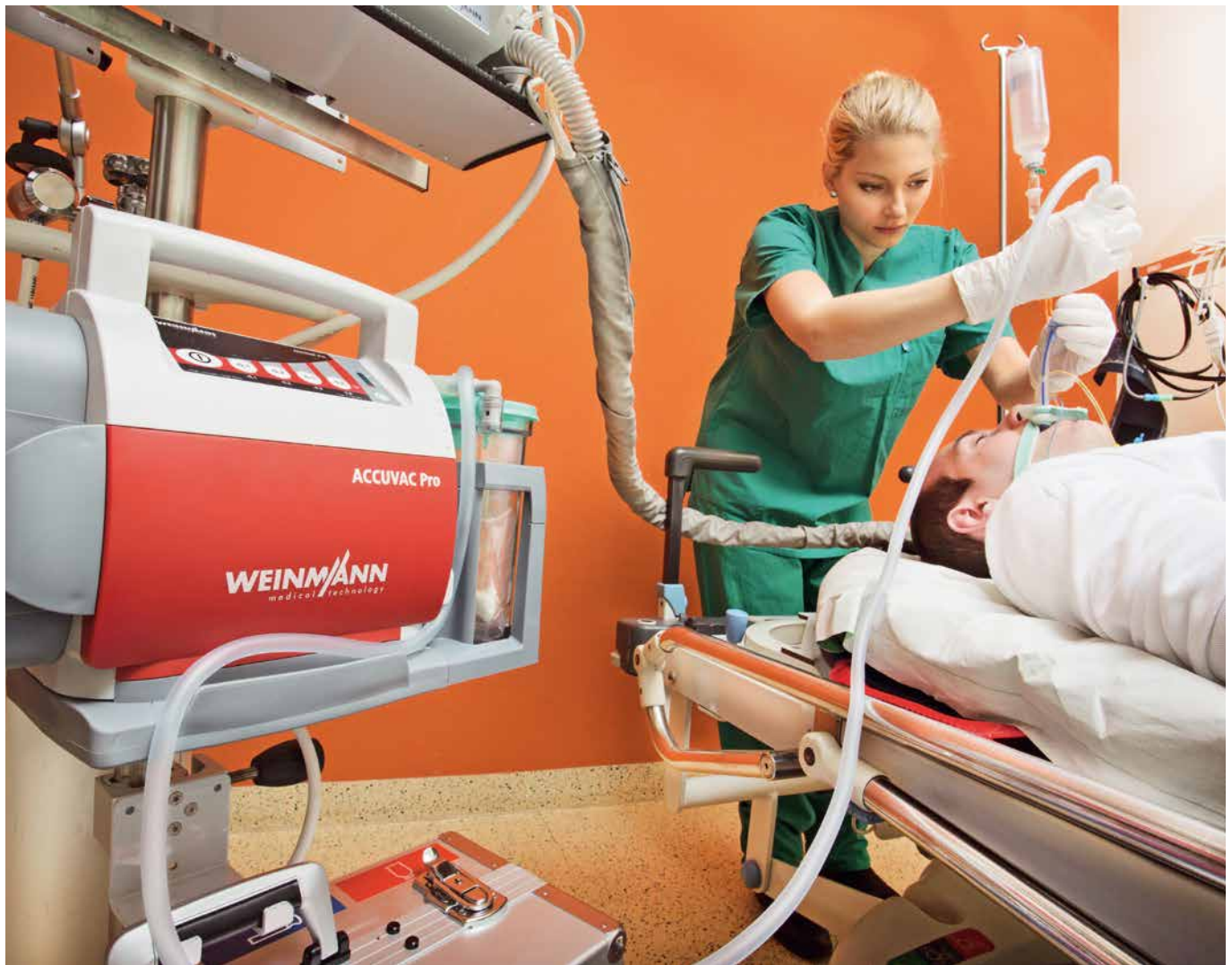
## ACCUVAC Pro suction device

Experience at its best – for Professional Demands

## ACCUVAC Pro

### Experience at its best – for professional demands

The first life-saving step involves clearing the patient's airways from obstructions so that effective ventilation can be administered. In an emergency situation the patient's survival depends on keeping the airways clear. ACCUVAC Pro ensures free and clear airways by suctioning the mouth and throat, endotracheal or bronchial areas. The pump's high suction capacity and its easy handling assist medical personnel in carrying out this effective and proven patient procedure in professional emergency medical services.

### Broad application range, flexible operation

With the choice of four pre-defined suction levels, the user can adjust the suction capacity to every situation. ACCUVAC Pro can be used on babies, children and adults. In addition to suctioning airways, ACCUVAC Pro also deflates vacuum splints and mattresses.

### Your benefits at a glance

- Variable uses
- Safe and hygienic
- Simple operation, robust
- EN 1789-compliant, air worthy as per RTCA DO-160G

## Convenient operation, broad application spectrum

- High suction capacity of about 34 liters/minute at -0.8 bar (at device inlet)
- Rechargeable battery: user can easily change battery without tools
- Modern lithium-ion rechargeable battery with greater run time of more than 60 minutes
- Four pre-defined vacuum levels can be selected by user
- Automatic function check with rapid visual and acoustic feedback
- Safe secretion collection in auto-clavable reusable secretions canister with disposable bacteria filter and overflow protection or disposable Serres® suction bag with integrated bacteria filter
- Robust housing made of impact-resistant materials
- Non-tip housing with low center of gravity

## Ideal operation in everyday medical services

- Optimum accommodation of suction tube in holder on side of device
- Single-handed release from the wall mounting with the press of a button
- Compatible with existing ACCUVAC wall mounting
- Optionally available: large accessories bag, protective bag and/or shoulder strap

## For every patient and every emergency

- Suctioning of secretions and food particles
- Endotracheal and bronchial suction
- Particularly suitable for pre-hospital care
- Deflate vacuum splints and mattresses
- Suitable for use on babies, children and adults

## Safety and reliability day after day

In the event of a malfunction, you can read out the service files from your ACCUVAC Pro and send them to WEINMANN Emergency. At best, the data are enough for our specialists to

**Service Hotline**  
**+49 40 88 18 96 122**

provide a remote diagnosis and work with you to resolve the problem. If not, we will take a closer look at your device and, as needed, we will give you a replacement device to use while yours is out of service.

## Service data: ACCUVAC Pro

|                                                       |                                                |
|-------------------------------------------------------|------------------------------------------------|
| Manufacturer's warranty                               | 2 years                                        |
| Safety check interval                                 | no check required                              |
| Maintenance schedule                                  | no maintenance required                        |
| Function check                                        | automatic                                      |
| Time required for function check                      | about 15 seconds                               |
| Battery change                                        | simple, from outside device, no tools required |
| Rechargeable battery                                  | no maintenance, no calibration required        |
| Deep discharge protection for battery                 | ✓                                              |
| Protection from electrical surge and reverse polarity | ✓                                              |
| Conversion of disposable system/reusable system       | simple, without tools                          |
| Software update can be made by operator/user          | ✓                                              |
| Remote diagnosis of device malfunction                | ✓                                              |

# Intuitive Operation Ensures Safety

## 1. Holder for suction tube

for ideal storage

## 2. Four pre-defined suction levels

quick push-button selection

## 3. Backlit

to make suction levels visible in poor lighting conditions

## 4. Release for wall mounting

single-handed release with press of button

## 5. Battery compartment cover

can be opened without tools for fast and easy access to battery

## 6. Rechargeable battery

modern lithium-ion battery with longer service life and rapid charging

## 7. Device base with tube guide

## 8. Automatic function check

with quick visual and acoustic feedback

## 9. Canister holder

impact-resistant materials increase safety for the canister systems

## 10. Secretions canister Serres®, 1000 ml

with disposable suction bag, integrated bacteria filter and solidifying agents. When the bag is full, ACCUVAC Pro stops automatically

# Technical Data

## ACCUVAC Pro

|                                                                                                                          |                                                                                                                                                                                                          |                                                                                                                                                                                                           |
|--------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Product class as per Directive 93/42/EEC                                                                                 | IIa                                                                                                                                                                                                      |                                                                                                                                                                                                           |
| RoHS-conformity as per Directive 2011/65/EU (RoHS II)                                                                    | Yes                                                                                                                                                                                                      |                                                                                                                                                                                                           |
| Classification as per EN ISO 10079-1                                                                                     | High vacuum/high flow                                                                                                                                                                                    |                                                                                                                                                                                                           |
| Standards used                                                                                                           | EN 60601-1, EN 60601-1-11, EN 60601-1-12, EN ISO 10079-1, EN 1789, RTCA DO-160 G                                                                                                                         |                                                                                                                                                                                                           |
| Dimensions (W x H x D)                                                                                                   | <ul style="list-style-type: none"> <li>• with canister system</li> <li>• with canister system + accessories bag</li> </ul>                                                                               | 370 x 277 x 146 mm<br>370 x 277 x 152 mm                                                                                                                                                                  |
| Weight                                                                                                                   | <ul style="list-style-type: none"> <li>• Device with battery, no canister system and holder</li> <li>• Reusable canister system with holder</li> <li>• Disposable canister system with holder</li> </ul> | 3.65 kg<br>1.00 kg<br>0.65 kg                                                                                                                                                                             |
| Temperature range: Operation<br>Transport/storage                                                                        | -5 °C to +50 °C<br>-40 °C to +70 °C                                                                                                                                                                      |                                                                                                                                                                                                           |
| Maximum current consumption                                                                                              | 3.8 A                                                                                                                                                                                                    |                                                                                                                                                                                                           |
| Rated voltage                                                                                                            | 12 V DC nominal (min. 10 V, max. 15 V)                                                                                                                                                                   |                                                                                                                                                                                                           |
| Suction capacity at device inlet<br>(without canister system) at -0.8 bar,<br>fully charged battery and 21 °C/1013 hPa   | 34 liters/min                                                                                                                                                                                            |                                                                                                                                                                                                           |
| Suction capacity at device inlet to reusable<br>canister system at -0.8 bar,<br>fully charged battery and 21 °C/1013 hPa | 30 liters/min                                                                                                                                                                                            |                                                                                                                                                                                                           |
| Vacuum settings                                                                                                          | Pre-defined levels: -0.1 bar, -0.2 bar, -0.5 bar and -0.8 bar, electronically regulated                                                                                                                  |                                                                                                                                                                                                           |
| Type of operation                                                                                                        | S2 60 min                                                                                                                                                                                                |                                                                                                                                                                                                           |
| Degree of protection                                                                                                     | IP34D                                                                                                                                                                                                    |                                                                                                                                                                                                           |
| Type of rechargeable battery                                                                                             | L-ion                                                                                                                                                                                                    |                                                                                                                                                                                                           |
| Rechargeable battery                                                                                                     | <ul style="list-style-type: none"> <li>• Charging time</li> <li>• Service life</li> <li>• Battery operating time</li> </ul>                                                                              | Battery status 80%: 2 hrs, 40 min (at 20 °C while not in use)<br>Battery status 100 %: about 4 hrs<br>about 500 charging cycles<br>60 min (during continuous operation with<br>fully charged/new battery) |
| Volume of secretions canister                                                                                            | 1000 ml                                                                                                                                                                                                  |                                                                                                                                                                                                           |
| Reusable suction tube                                                                                                    | 10 mm ID, 1300 mm long                                                                                                                                                                                   |                                                                                                                                                                                                           |
| Disposable suction tube                                                                                                  | 7 mm ID, 1800 mm long                                                                                                                                                                                    |                                                                                                                                                                                                           |

**CE 0124**

  
 Certified Quality  
 Management System  
 meeting EC directive 93/42/EEC,  
 Annex II (EN ISO 9001:2008/EN ISO 13485)

All rights to design and specification modifications reserved.

 Designed in Germany

 Select your preferred ACCUVAC Pro combination – designed to meet your needs:

**ACCUVAC Pro with  
 disposable canister system,  
 WM 11605**

**ACCUVAC Pro with reusable  
 canister system\*  
 WM 11600**

**ACCUVAC Pro with  
 disposable canister system  
 with accessories bag,  
 WM 11645**

**ACCUVAC Pro with  
 reusable canister system\*  
 with accessories bag,  
 WM 11640**

Combine your ACCUVAC Pro with additional accessories from page 6

\*Please note that the reusable canister system is equipped with a disposable bacteria filter.

# Accessories and Replacement Parts

## Accessories

1. Protective bag (cannot be combined with accessories bag) WM 11692
2. Shoulder strap WM 11693
3. 12-V connection cable WM 10650
4. Power supply unit und charger for 100V to 240V alternating current to ACCUVAC device plug, including power cable in compliance with EN 50075 (Europlug) WM 2620
5. Wall mounting for ACCUVAC including installation set WM 15208
6. Holding plate for standard hospital rails WM 15845
7. Attachment set with two adapters for standard hospital rails WM 15805
8. Wall mounting for power supply unit and charger WM 15844
9. Installation set pole bracket dia. 19 mm - 40 mm WM 15806
10. Conversion set for reusable canister system, consisting of: WM 17820
  - Set, reusable canister system (WM 17821)
  - Holder for reusable canister system (WM 11654)
11. Conversion set for disposable canister system, consisting of: WM 17825
  - Holder for disposable canister system (WM 11754)
  - Vacuum tube for Serres® secretions canister (WM 11761)
  - Serres® secretions canister, complete (WM 10790)
12. Conversion kit for accessories bag, consisting of: WM 17829
  - Accessories bag (WM 11691)
  - Battery compartment cover for accessories bag (WM 11614)

## Consumables

13. Serres® suction bag 1000 ml, with hydrophobic filter and solidifying agent (Packaging unit: 32 items in carton) WM 17800
14. Disposable suction tube with fingertip control for use with disposable system, 180 cm long, 7 mm ID, also available in sets of 10, 20, 32 and 50 tubes WM 10778
15. Reusable suction tube 130 cm long, 10 mm ID, also available in sets of 10, 20, and 50 tubes WM 10662
16. Fingertip control for reusable suction tube 10 mm ID, also available in sets of 10, 20 and 50 pieces WM 10666
17. Set of 10 disposable bacteria filters for reusable secretions canister WM 17830

## Replacement Parts for Reusable System

- |                                                                                      |          |
|--------------------------------------------------------------------------------------|----------|
| 1. Upper part of secretions canister cover                                           | WM 11657 |
| 2. Filter holder                                                                     | WM 11661 |
| 3. O-Ring for filter holder                                                          | WM 11663 |
| 4. Disposable bacteria filter                                                        |          |
| 5. Lower part of secretions canister cover                                           | WM 11658 |
| 6. Ball float valve                                                                  | WM 11662 |
| 7. Reusable secretions canister 1000 ml                                              | WM 11653 |
| 8. Reusable suction tube 10 mm ID                                                    | WM 10662 |
| 9. Fingertip control for reusable suction tube 10 mm ID                              | WM 10666 |
| 10. Holder for reusable canister system                                              | WM 11654 |
| • Secretions canister cover, complete, consisting of replacement parts 1 – 6         | WM 17822 |
| • Set, reusable canister system consisting of replacement parts 1 – 9                | WM 17821 |
| • Conversion set for reusable canister system consisting of replacement parts 1 – 10 | WM 17820 |

## Replacement Parts for Serres® Disposable System

- Serres® secretions canister, complete, consisting of: WM 10790
  - Secretions canister, 1000 ml (WM 10775)
  - Suction bag
  - Disposable suction tube with fingertip control (WM 10778)
- Serres® secretions canister, 1000 ml WM 10775
- Vacuum tube for Serres® secretions canister WM 11761
- Holder for disposable canister system WM 11754
- Set disposable canister system, consisting of: WM 17826
  - Vacuum tube for Serres® secretions canister (WM 11761)
  - Serres® secretions canister, complete (WM 10790)
- Angled connector for Serres® canister, reusable WM 10799

## Replacement Parts for ACCUVAC Pro

- Set, catch release WM 17827
- Lithium-ion rechargeable battery WM 11603
- Battery compartment cover (without eyelets for accessories bag) WM 11604
- Holder for suction tube WM 11664
- Device base WM 11677

## Simply Professional

WEINMANN Emergency is a family-owned, internationally active medical technology company. With our mobile system solutions for emergency, transport and disaster medicine, we set standards for saving human lives. In close collaboration with professional users in emergency medical services, hospitals and military medical corps, we develop innovative medical products for ventilation and defibrillation. For more than 100 years we have offered our customers a high degree of reliability, extensive experience and quality made in Germany.

---

### Headquarter

WEINMANN Emergency  
Medical Technology GmbH + Co. KG  
Frohbösestraße 12  
22525 Hamburg  
Germany

T: +49 40 88 18 96-0  
F: +49 40 88 18 96-480  
T: +49 40 88 18 96-120 Customer Service  
T: +49 40 88 18 96-122 After-Sales Service  
E: [info@weinmann-emt.de](mailto:info@weinmann-emt.de)

### Center for Production, Logistics, Service

WEINMANN Emergency  
Medical Technology GmbH + Co. KG  
Siebenstücken 14  
24558 Henstedt-Ulzburg  
Germany

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Conformity assessment procedure:

Directive 93/42/EEC Annex II on medical products, passed by the commission on 14<sup>th</sup> June 1993, last amended on 5<sup>th</sup> September 2007

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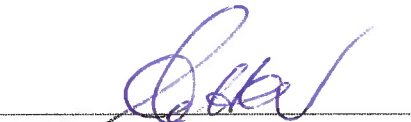
Valid till further changes on the product until May 26<sup>st</sup> 2024.

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Place and date of issue

---

i.V. Andreas Heer  
Head of Quality Management

---

i.V. Steffi Focke  
Quality Management

**EC Certificate**  
**Directive 93/42/EEC Annex II, excluding Section 4**  
**Full Quality Assurance System**  
**Medical Devices**

**Registration No.:** HD 60148646 0001

**Report No.:** 21201240 016

**Manufacturer:** WEINMANN Emergency Medical  
Technology GmbH + Co. KG  
Frohbösestr. 12  
22525 Hamburg  
Deutschland

**Products:** Medical devices for emergency and transport medicine  
(see attachment for products and sites included)  
Replaces Certificate, Registration No.: HD 60129559 0001

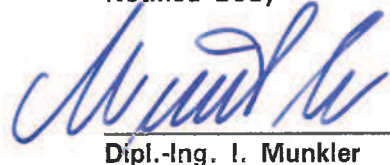
**Expiry Date:** 2024-05-26

The Notified Body hereby declares that the requirements of Annex II, excluding section 4 of the directive 93/42/EEC have been met for the listed products. The above named manufacturer has established and applies a quality assurance system, which is subject to periodic surveillance, defined by Annex II, section 5 of the aforementioned directive. For placing on the market of class III devices covered by this certificate an EC design-examination certificate according to Annex II, section 4 is required.

**Effective Date:** 2020-04-16

**Date:** 2020-04-16

Notified Body

Dipl.-Ing. I. Munkler

**TÜV Rheinland LGA Products GmbH - Tillystraße 2 - 90431 Nürnberg**  
TÜV Rheinland LGA Products GmbH is a Notified Body according to Directive 93/42/EEC  
concerning medical devices with the identification number 0197.

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
**Products included:**

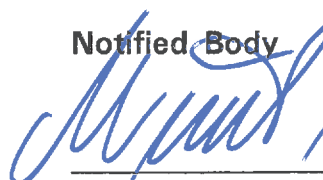**Emergency and transport medicine:**

- Suction pumps
- Ventilators
- Module systems
- Resuscitators
- Ventilation masks
- Patient hose systems
- Pressure reducers
- Click dial flowmeter
- Defibrillators-/Monitoring systems
- SpO2 sensors

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WEINMANN Emergency Medical Technology GmbH + Co. KG