

# CLEANASCOPE™ ADVANTAGE

Transport & Short Term Storage System



# CLEAR IDENTIFICATION OF CLEAN VERSUS CONTAMINATED ENDOSCOPES




## CONFIDENCE IS KNOWING IT'S CLEAN AT A GLANCE

The CLEANASCOPE™ Advantage Transport and Short-term Storage System protects your endoscopes and other delicate equipment by minimising direct handling and reducing contact with airborne contaminants.

Its patented design creates a protective environment for each disinfected endoscope. Green (clean) and red (contaminated) colour-coded liners immediately identify clean or contaminated equipment.

## OPTIMISED ENDOSCOPE WORKFLOW MAXIMISES INFECTION PREVENTION

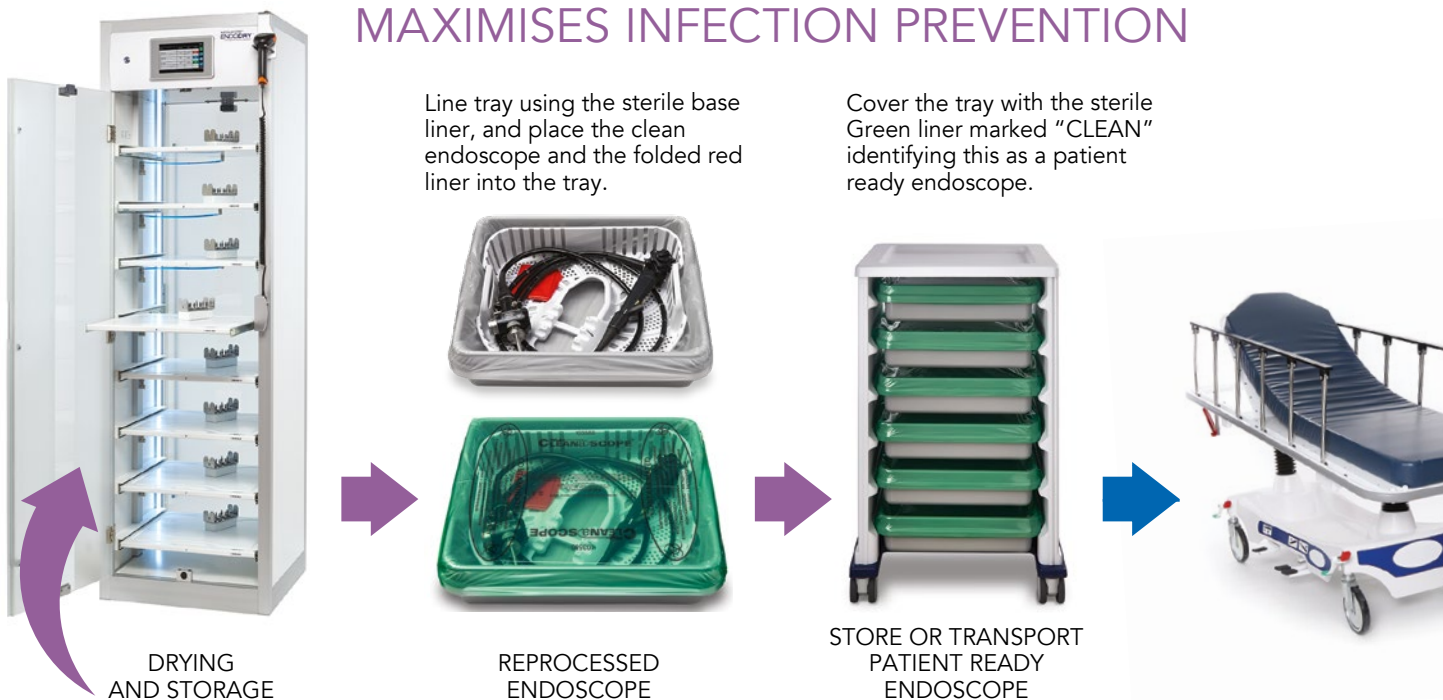



## THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, **only Cantel delivers the Complete Circle of Protection**, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimize your success.

**DRY & STORAGE** Bacteria pose significant risk to endoscopes during transport and storage. Cantel's transport, drying and storage solutions are designed to protect valuable inventory, reduce cross contamination touchpoints, eliminate moisture in the endoscope channels and control humidity. Humidity or moisture in endoscope channels is known to aid bacterial growth.

Trays fit perfectly into the contoured cart top

Provides protected short-term storage of clean scopes and equipment

Hard lids available for added protection when transporting outside department

Available in 5, 6 and 10 tray options




Robust reusable tray easily fits large endoscopes and the ADVANTAGE PLUS™ AER cassette



Three-part, single-use sterile packed liner system reduces the potential for cross-contamination

Large locking caster, wheels make transport easy and smooth

Post procedure, flush and wipe the endoscope using INTERCEPT™ Detergent and place dirty endoscope inside tray. Cover tray using the Red liner for clear identification of "CONTAMINATED" endoscope.

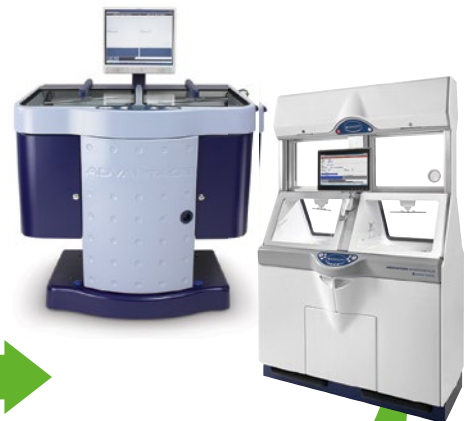

CONTAMINATED TRANSPORT

MANUAL CLEANING

FIT SECURELY IN ADVANTAGE REPROCESSOR