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*Embedding cassettes for biopsies with small holes suitable to contain very small specimens, in acetal resin (POM). Lid provided with triple closure system, removable. Writing surface. They resist to decalcifying solutions, to solvents and ultrasounds techniques. Compatible with most of automatic printers (Vogel, Olympus, Leica and similar).*

**Prodotto con marchio CE - conforme al Regolamento (UE) 2017/746 (IVDR)**  
**CE Marked product - manufactured in compliance with Regulation (EU) 2017/746 (IVDR)**

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**Classificazione EDMA > 13070104 - Embedding media (Histology/Cytology)**

**Basic UDI-DI (Global Model Number – GMN): 805577609F104CASSINCL015P**

*In Vitro Diagnostic Medical Device "specimen receptacles" suitable to contain and protect the histological sample intended for inclusion, processing and carrying out in vitro laboratory diagnostic tests. For use in professional test laboratory only.*

**National classification of medical devices (CND - For Italian law) > W05030501 - Inclusion Devices, Cassettes**

**EDMA code > 13070104 - Embedding media (Histology/Cytology)**

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#### **- For professional use only -**

*Use only for laboratory diagnostic analysis of human biological samples in healthcare settings.*

*Device without measuring function.*

*Any presence of a graduated scale is intended for indicative purposes and without a metrological function.*

*Do not vary the intended purpose of the device. Please contact the Manufacturer in the evenience of doubts concerning the intended purpose.*

*Before use check the perfect closure of the device if applicable.*

*Handle using personal protective equipment: risk of contamination.*

*After collection, store the sample as prescribed by the analytical method and deliver to the Laboratory within the maximum time specified by the same.*

*Do not use the Device in case of evident signs of breakage, injuries, cracks that could compromise its correct use.*

*Keep out of flame or heat sources which might damage the device.*

*Do not use after expiry date or if packing is opened.*

*Do not re-use: Disposable Device. Any reuse could cause sample contamination.*

*If sterile (ionizing radiations) or aseptic Device: Microbiological status in undamaged pack.*

*Do not re-sterilize.*

*Use the Device only with compatible and / or supplied accessories (caps, supports, etc.)*

*Do not put under direct sun rays; store in a dry, cool place (U.R. max 75% at 26°C)*

*Store in dry place, temperature range: min. -10°C (14°F) max +50°C (122°F)*

*Disposal according to applicable regulations, risk of infection.*

*Before use with particular substances check the resistance/compatibility chart in our catalogue.*

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**Aptaca S.p.A.** Regione Monforte, 30 - 14053 Canelli (Asti) Italy