ENDOSCOPY SYSTEMS & THERAPEUTIC DEVICES ENDOSOLUTIONS FOR G

FROM SCREENING TO TREATMENT



#### COMPANY

Unchanging Values	4
Developing Technologies	6
Experiencing Endoscopy	18
Daily Endoscopy Practice	20

#### **PRODUCTS & THERAPEUTIC DEVICES**

23
40
42
44
47
51
61
79
81
89
93
95
99
103
113

#### SERVICE

Service & Partnership	117
Training & Education	120
Product Recommendations	122



# DEDICATED TO **PATIENTS' QUALITY OF LIFE**



## THE UNCHANGING VALUES OF THE FUJIFILM GROUP

We will use leading-edge, proprietary technologies to provide top-quality products and services that contribute to the advancement of culture, science, technology and industry, as well as improved health and environmental protection in society.

Our overarching aim is to improve the quality of life of patients worldwide through early detection and successful treatment of disease.

#### HEALTHCARE

As a total healthcare company, we are involved in the development of a wide range of businesses in the three areas of prevention, diagnosis, and treatment. We are contributing to the health of people around the world by responding to unmet medical needs, early detection of diseases, and support for the development and manufacture of innovative vaccines and pharmaceuticals.

Our clinically proven products and technologies are continuously being refined to make the work of health professionals more effective and efficient.

At Fujifilm we are constantly innovating and creating new solutions that address the practical needs of our global customers in various business fields including healthcare, graphics systems, optical devices, recording media and photographic technologies.

Every year we invest around seven percent of our consolidated turnover in research and development which includes dedicated research and the nurturing of close working relationships with international specialists. This ensures that we not only meet high-quality requirements but also contribute to the advancement of culture, science, industry, and technology as well as improved health and environmental protection in society. The focus at Fujifilm is firmly on holistic patient care which means that our service portfolio includes expert technical assistance, a comprehensive range of hygiene products and individual consulting.


We have expanded our business globally, in order to support patients and medical service providers with rapid, precise diagnosis and efficient treatment.

Today, Fujifilm operates in around 50 group companies and branches in Europe, employing over 4,500 people engaged in R&D, manufacturing, sales, and service support.





# **DEVELOPING TECHNOLOGIES** BEYOND THE EXPECTED

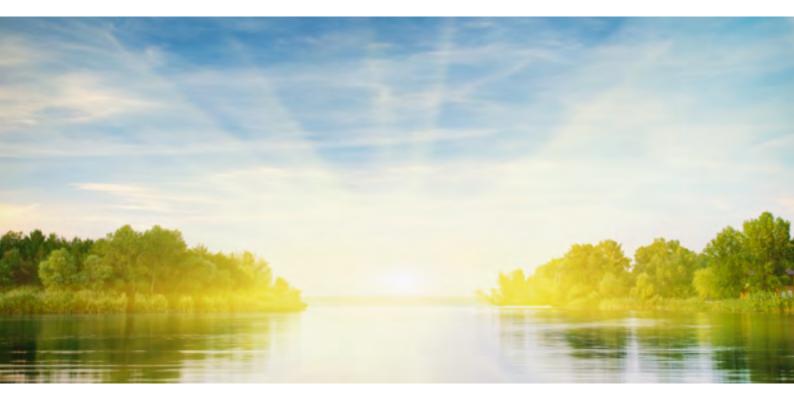


# **ADVANCED TECHNOLOGIES, EXCELLENT POSSIBILITIES**

Fujifilm has been developing image processing technologies for many years. Excellent image capture and advanced data processing further improve technology in our endoscopy portfolio.



BLI & Zoom

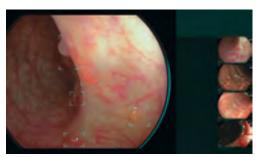

LCI & CAD EYE

The Multi Light<sup>™</sup> technology, for example, uses distinct light wavelengths to support lesion detection and characterisation by high-contrasting LCI and BLI images.

Fujifilm continues to develop imaging solutions: One particular focus has been the development of Al-driven smart technologies for the radiology field as well as medical ultrasound and, more recently, endoscopy: CAD EYE.

CAD EYE is intended to make procedures more efficient by improving polyp detection and characterisation rates.

#### **SELECTION OF OUR TECHNOLOGIES**



#### MULTI LIGHT™ TECHNOLOGY

Illumination suitable for observation using variable LED light intensity.



#### LCI TECHNOLOGY

Increased contrast in red colour leads to improved visibility of abnormalities, inflammation and delineation.



#### BLI TECHNOLOGY

The combination of special light wavelengths results in improved contrast imaging for characterisation.



#### CAD EYE

This function, developed utilising AI technology, supports colonic polyp detection and characterisation during colonoscopy.



#### CMOS TECHNOLOGY

Brilliant image transmission with reduced noise thanks to a CMOS-chip positioned directly in the tip.



#### FICE TECHNOLOGY

Provides the possibility to enhance slight colour differences such as vascular and mucosal patterns without tissue staining. The procedure digitally selects three wavelengths of light and displays reconstructed images.



#### SUPER CCD TECHNOLOGY

The Super CCD and high-performance optical system provides high-quality images.



#### WATER JET TECHNOLOGY

The Water Jet function supports constant visualisation for both diagnostic and therapeutic procedures.



**COLOASSIST TECHNOLOGY** Flexible adjustment to be expected for easier insertion in addition to advanced force transmission and adaptive bending.



DICOM TECHNOLOGY The goal of the DICOM Standard is to achieve

I he goal of the DICOM Standard is to achieve compatibility and improve workflow efficiency between imaging systems and other information systems.



MULTI ZOOM TECHNOLOGY Easy-to-control optical magnification in stepwise or continuous magnification mode.



ANTI-BLUR FUNCTION The clearest image among multiple images is automatically selected.



CLOSE FOCUS Observation up to 2 mm supports diagnosis of the disease.



HD TECHNOLOGY Combine equipment displaying this logo to ensure that you view HDTV images on your monitor.




#### SEE MORE. DETECT MORE. MULTI LIGHT™ TECHNOLOGY.



Achieving improved diagnostic and therapeutic results in endoscopic procedures is highly dependent on image quality. As one of the world's largest imaging companies, our long-standing experience in medical imaging has allowed Fujifilm's engineers to develop Multi Light<sup>™</sup> technology, fulfilling the need for improved visualisation in endoscopy – today and in the future. This illumination system meets the high brightness and contrast standards enabling the observation modes LCI and BLI. Specifically designed for this illumination system, the ELUXEO™ 700 series of endoscopes provides detailed high-resolution imaging for both diagnosis and pre-therapeutic assessment.


#### IMPROVED ILLUMINATION USING VARIABLE LED LIGHT INTENSITY

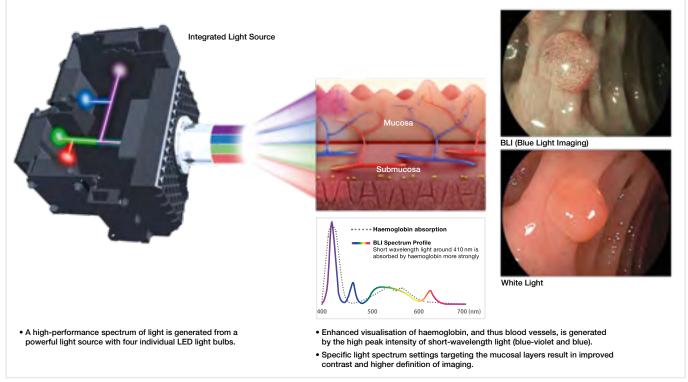


This drawing is for illustration purposes only and not a complete representation.

High-intensity illumination based on Multi Light<sup>™</sup> technology creates high-quality images with White Light Imaging and the observation modes LCI and BLI. With the involvement of numerous clinical experts, the ideal composition of four LEDs for each observation mode has been developed to achieve excellent results in illumination. With a simple push of a button, you can easily switch between the following observation modes:


#### WHITE LIGHT IMAGING

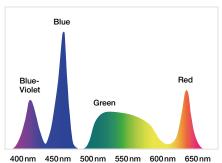

#### LCI MODE

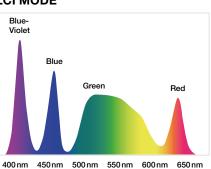

#### **BLI MODE**

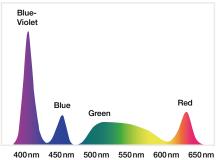



# LCI (LINKED COLOR IMAGING) MODE

LCI differentiates the red colour spectrum more effectively than White Light Imaging thanks to its preprocess composition of light spectrum and advanced signal processing. The increased colour contrast in red colour leads to improved visibility of abnormalities, inflammation and delineation.





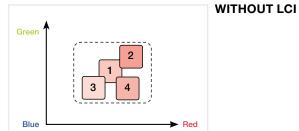

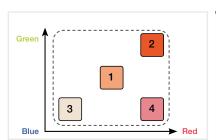


Oesophagus – LCI Mode


#### WITH LCI

# BLI (BLUE LIGHT IMAGING) MODE 🔛

High-intensity contrast imaging with BLI is expected to be helpful for improved visualisation of superficial vascular and mucosal patterns. Focussing on the characteristics of short wavelength absorption of haemoglobin (at 410 nm) combined with specific white light spectral colours results in improved contrast imaging.

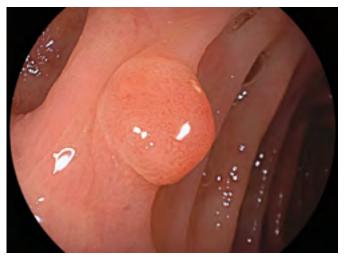

Colon - White Light Imaging

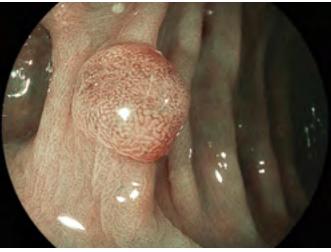


Colon – BLI Mode

#### CAD EYE - ARTIFICIAL INTELLIGENCE


#### DETECTION AND CHARACTERISATION SUPPORT IN THE COLON

CAD EYE has been developed utilising AI deep learning technology. It has been trained with a powerful supercomputer located in Fujifilm's global AI technology centre in Tokyo, utilising an immense amount of clinical images using Fujifilm endoscopy systems. As a result, CAD EYE is a customised detection and characterisation support compatible with the ELUXEO<sup>™</sup> system.

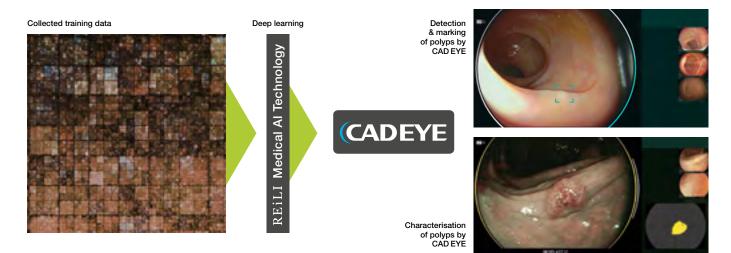

#### **USER-FRIENDLY INTERFACE**

The development of the interface has been designed to enable comfortable procedures. It does not interfere with clinical images and minimises required eye movement. Its display is designed to besimple and intuitive for excellent support during long hours in the examination room.

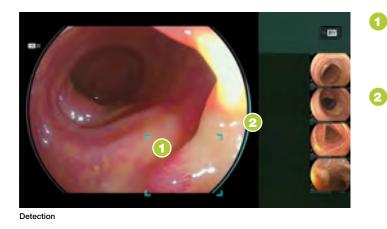



Characterisation

#### Detection box

Displays the area where the suspicious polyp is detected. Different sizes of the Detection Box are available.

#### Visual assist circle

Lights up in the direction where the suspicious polyp is detected.

## Detection sound

Sound signal when a suspicious polyp is detected. Volume can be defined for each user.



#### Status bar

Indicates the status of characterisation analysis regarding the suspicious area.

#### Visual Assist Circle

GREEN: Characterisation HYPERPLASTIC YELLOW: Characterisation NEOPLASTIC

#### Position map

Indicates the position of the suspicious area, this software is characterising.

#### Characterisation result

HYPERPLASTIC: hyperplastic polyps & SSL NEOPLASTIC: adenoma and cancer

## COLOASSIST PRO - REAL-TIME VISUALISATION

Taking into account that the structure, mobility and anatomical complexity of the colon varies with each individual, the insertion of the endoscope into the colon requires a high level of technical expertise. Fujifilm's Endoscope Visualisation System displays the configuration of the endoscope in real-time by reproducing a coloured graphical representation of the endoscope next to endoscopic view<sup>\*</sup>.

# HOW THE VISUALISATION SYSTEM WORKS

Coils within the Transceiver Dish generate an electromagnetic field that is received by coils integrated within the length of the dedicated ELUXEO<sup>™</sup> EC-760S-G/L, M colonoscope and within the Hand Marker. This electromagnetic field is used to determine the position of the coils in the endoscope.The current position of the Hand Marker is displayed on the endoscope shape visualisation screen which can be used to refer to the endoscope's location within the patient.







# CMOS TECHNOLOGY WITH MEGAPIXEL




With the CMOS chip built directly into the tip of the endoscope, the signal is digitally transmitted through the device, thus providing high-resolution imaging. All 760, 720 and 600 endoscopes are equipped with CMOS.

# Analogue transmission ! Digital transmission Chip in the tip – CMOS Image sensor Progressive scanning Interlaced scanning (complete frames) (half-frames) АВС ABC

The CMOS chip is positioned directly in the tip of the endoscope and transforms the analogue signal into a digital signal at the site of examination. This ensures brilliant image transmission with reduced noise.

CMOS technology supports 60 frame progressive scanning technology where complete images are processed, rather than half-frames processed when using the interlaced scanning method. The result is a high-resolution image and smooth moving images with reduced blurring.

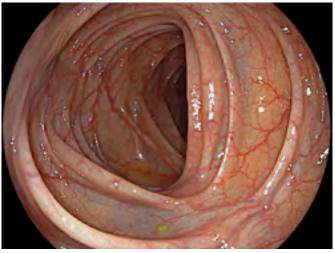

Colon in super high resolution




#### Better visibility for detection and diagnosis

FICE – Flexible Spectral Imaging Colour Enhancement – enhances colour differences such as vascular and mucosal patterns without the need for tissue staining. The procedure digitally selects three wavelengths of the light and displays the reconstructed images. The endoscope switch allows physicians to change between the conventional image and the FICE image in a split second, ensuring an uninterrupted examination with the eyes always concentrated on the monitor.



#### XENON endoscopy White Light image

Nearly all of the red wavelength is reflected.

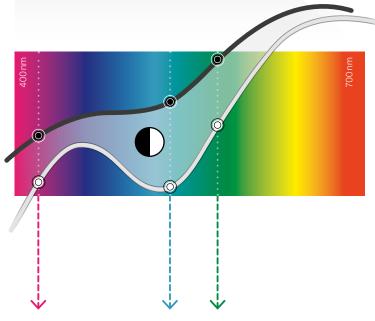

#### DUAL MODE


# Simultaneously displays a FICE image and a White Light image on the same monitor

A dual view of a FICE image and a White Light image on the same monitor allows you to collect more information for examination and diagnosis.



#### FICE (Flexible Spectral Imaging Colour Enhancement)

The contrast is enhanced and the vascular pattern is highlighted by focusing on the difference in wavelength reflection of mucosa and blood vessels.

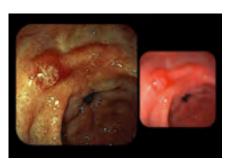




#### Optical zoom for precise focusing

Fujifilm's latest Multi Zoom technology enables programming of up to three magnification modes to realise an easy-to-control endoscopic zoom (stepwise or continuous magnification mode).

- 2-step zoom
- 3-step zoom
- 5-step zoom

The optical zoom allows a close examination of the mucosa tissue and capillary structures in combination with excellent focusing and orientation during magnification throughout the wide focal plane.

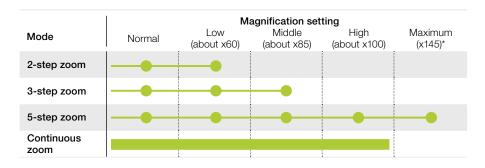

#### E-ZOOM

#### Electronic zoom provides better visibility

E-Zoom images can be provided by pressing the endoscope button once. Normally, E-Zoom increases the noise in an image. The E-Zoom function can be used with the 600 series to produce a FICE image with less noise so that it is possible to observe the surface pattern detail as well as the vascular pattern.

<sup>\*</sup> In combination with 26" screen.

## COLOASSIST ADJUST



#### Specifically developed for the 760 series colonoscopes

ColoAssist Adjust features the Flexibility Adjuster with different levels of stiffness as well as Advanced Force Transmission and Adaptive Bending, expected to be helpful for manoeuvrability.


#### FLEXIBILITY ADJUSTER

The stiffness of the flexible portion of the colonoscope can be easily adjusted according to your preference by turning the flexibility adjustment ring (level 0 to 3). This is expected to be helpful during intubation when passing segments such as the sigmoid colon and the transverse colon, leading the endoscope to follow the intestinal tract more smoothly.



#### ADVANCED FORCE TRANSMISSION 💭


The flexible portion is designed to transmit the pushing, pulling and rotating movements from the hand to the distal end of the endoscope. It is intended to be helpful for manoeuvrability inside the digestive tract.

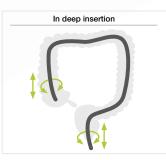

#### ADAPTIVE BENDING




The end of the bending section is soft, allowing the endoscope to bend easily. The flexible bending section has been designed to return more easily to its straight form after passing through the tight curves of the colon.

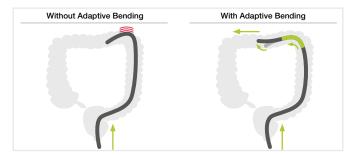

# 



With Over Megapixel CMOS image sensor, 760, 720 and 600 series endoscopes produce high-resolution images, while the CMOS technology realises less noise and brilliant images. The CMOS image sensor can change the analogue signal to digital in the tip of the endoscope. This short transmission path reduces quality losses.

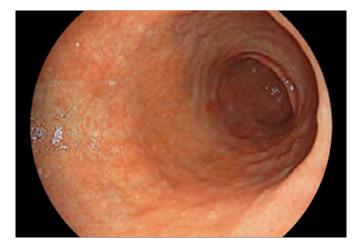



# CLOSE FOCUS ENHANCES IMAGING FOR DIAGNOSIS


The high-performance optical system enhances Close Focus observation capability up to 2 mm. The focus at the edges of an image has been improved. The combination of the Megapixel CMOS image sensor and the high-performance optical system assists various observations ranging from close-up to distant views.



#### ANTI-BLUR FUNCTION



This function extracts the best still image from multiple images to offer a sharp and clear image every time.

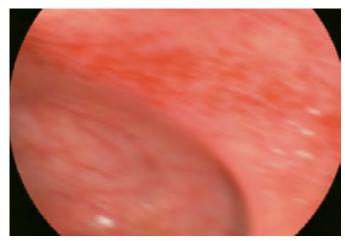

Freezing the image during the examination: High quality image sequences are stored in temporary memory and can be accessed later



Automatic selection and display of a sharp image

#### **AUTO PHOTOMETRIC CONTROL**

The automatic light control setting of the IRIS mode adjusts the lighting in accordance with the positioning of the endoscope, providing you with a well-balanced picture. You always get excellent illuminated images\* whether focusing close-up or from a distance.



Distant focus



Close Focus

# EXPERIENCING ENDOSCOPY FROM SCREENING TO TREATMENT



## RELIABLE GUIDANCE, IMPROVED OUTCOMES

As one of the leading companies in the development of endoscopy technology, Fujifilm is always striving to provide high quality products, excellent services and highly customised business solutions in the world of endoscopy.


We regularly set new benchmarks, for example, with the introduction of the LED Multi Light<sup>™</sup> technology providing the observation modes LCI and BLI, with devices for double balloon endoscopy as well as the effective Flamingo device to meet the therapeutic challenges of the buried bumpersyndrome.

#### **PRODUCT PORTFOLIO**

#### For diagnostic and interventional endoscopy Fujifilm is pushing the boundaries of endoscopic imaging technologies, AI empowered.

We have built a portfolio of solutions which are able to support the healthcare system to focus on the patient's needs.

Fujifilm offers a wide range of endoscopic and therapeutic technologies from endoscopy systems to devices and services, providing endoscopy solutions from screening to treatment.

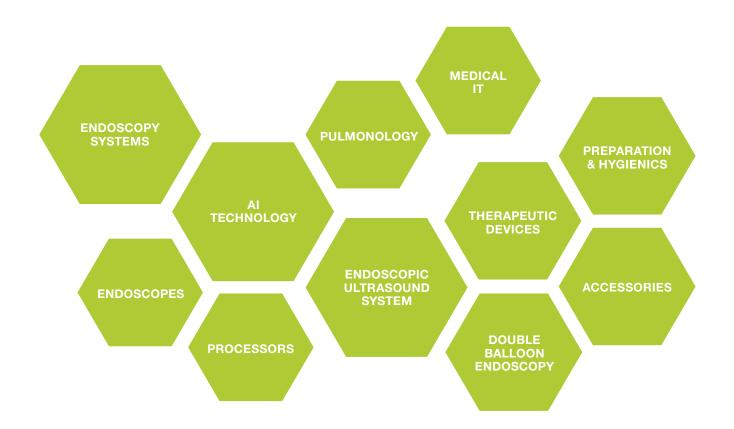

# DAILY ENDOSCOPY PRACTICE



## STEP BY STEP THROUGH THE PROCEDURE

Endoscopic examinations and treatments are complex procedures that require fingertip sensitivity as well as extensive, reliable equipment. All the better if everything comes from a single source and the processes can thus be designed to be as safe and uncomplicated as possible. Fujifilm can guide you expertly through the procedure from start to finish.




#### A G-EYE endoscope is used throughout the examination:

- It flattens topography to detect hidden polyps.
- It stabilises the endoscope during characterisation and treatment.
- It reduces bowel slippage.

#### **STEP 1:** DETECTION



**STEP 2: CHARACTERISATION** 

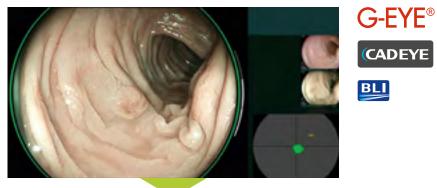


#### **STEP 3:** TREATMENT



# G-EYE® cross snare zero



# **ENDOSCOPES**

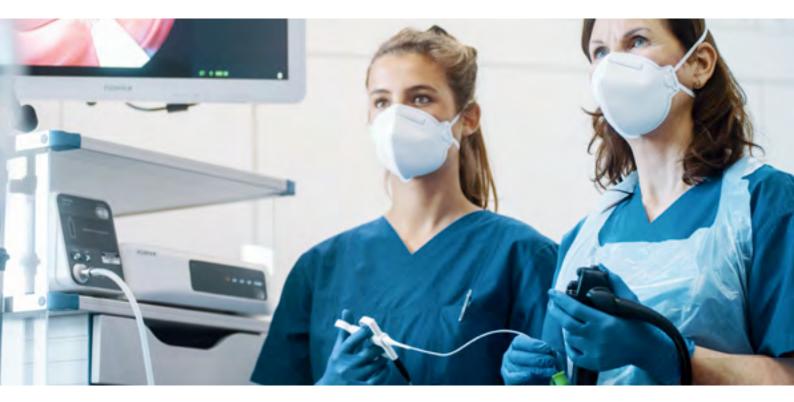

ELUXEO™ | CAD EYE | G-EYE | COLOASSIST PRO

Upper GI Endoscopy	24
Double Balloon Endoscopy	30
Lower GI Endoscopy	34
Visualisation System	40
G-Eye	42
Artificial Intelligence	44

# UPPER GI ENDOSCOPY



# **SEE MORE, DETECT MORE**

The ELUXEO<sup>™</sup> 700 series is compatible with special light observation modes such as Linked Color Imaging (LCI) and Blue Light Imaging (BLI), expanding Fujifilm's offerings in image enhanced endoscopy to support diagnosis, characterisation and treatment.

> JAZZ valves:\* protection against infection and cross contamination.



\* Manufactured by FUJIFILM medwork GmbH.

# ONE-STEP CONNECTOR FOR EASY PLUG-IN





The 700 series endoscopes can easily be plugged in by its One-Step Connector. This is the first connector to incorporate an integrated wireless power supply that provides high-speed transmission of data. The design helps to simplify the cleaning process and reduces the potential for accidental damage.



## G7 GRIP FOR COMFORT IN DAILY PRACTICE

In close cooperation with leading endoscopists, Fujifilm has renewed the layout and size of the components of the control portion and repositioned the angulation knobs to increase accessibility from the grip. The G7 grip is designed to have an easy and comfortable feel that improves performance and reduces stress during clinical procedures.

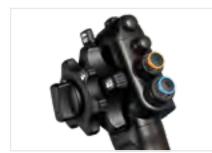

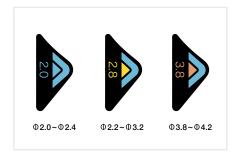

Each 700 series endoscope displays the information required to choose compatible accessories, which helps to facilitate on-the-spot decision making.

- EC-760ZP-V/M (4)(5)
- 1 Colour code of G7 control portion
- 2 Identification colour of working channel size
- 3 Working channel diameter
- 4 Corporate brand logo
- 5 Model number


#### EG-760R ELUXEO<sup>™</sup> VIDEO GASTROSCOPE Routine Type

# 

This routine gastroscope from the ELUXEO<sup>™</sup> 760 series is equipped with CMOS technology and provides HD images and videos for daily practice. Close Focus allows observation from as little as 2 mm in depth.



Field of view	140°
Observation range	2-100mm
Bending capability	Up 210°/Down 90° Right 100°/Left 100°
Ø Distal end	9.2mm
Ø Flexible portion	9.3mm
Ø Working channel	2.8mm
Working length	1,100mm
Total length	1,400mm

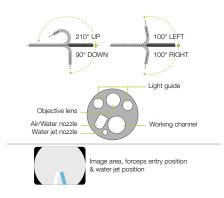

100° LEFT

100° RIGHT

210° UF

90° DÓV

# EG-760Z ELUXEO" VIDEO GASTROSCOPE Optical Magnification

Field of view

Observation range

This zoom gastroscope features the well-known 145x Multi Zoom\* which leads to clear and more detailed visualisation, supporting deeper analysis of mucosal structures. It has a small bending radius and similar functionality to the routine gastroscope including all features.



#### Up 210°/Down 90° Light guide Bending capability Right 100°/Left 100° Ø Distal end 9.9mm Objective lens Water jet nozzle Air/Water nozzle Working channel Ø Flexible portion 9.8mm Ø Working channel 2.8mm mage area, forceps entry position Working length 1,100mm er jet positior 1,400mm **Total length**

Normal 140°/Close 56°


Normal 3-100mm

Close 1.5-2.5mm

1.5-100mm

## Small Bending Radius

Features a tight bending section radius with improved angulation. It is designed to approach the targeted observation point and lesion more easily and with less effort.

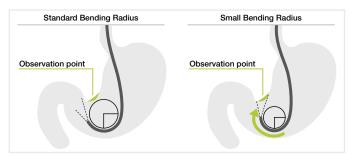

\* In combination with 26" screen.


#### EG-760CT ELUXEO<sup>™</sup> VIDEO GASTROSCOPE Therapeutic Type


This gastroscope from the ELUXEO<sup>™</sup> 760 series is equipped with a large 3.8 mm working channel that is especially suitable for therapeutic procedures compared to the standard gastroscope EG-760R with a working channel of 2.8 mm. In addition to therapeutic use, the gastroscope features LCI, intended to improve detection, and BLI, intended to characterise lesions, making it an excellent gastroscope for observation.




# Enlarged working channel for improved suction performance

The 3.8 mm working channel has a higher suction capacity compared to other gastroscopes, especially when the therapeutic accessory is inserted into the working channel.

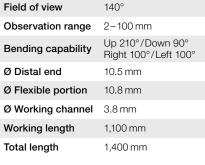

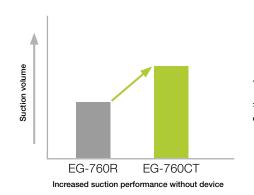

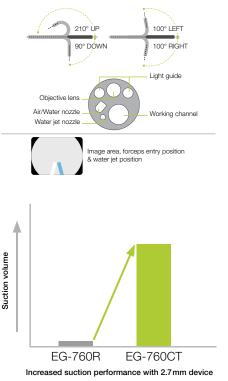


#### EG-740N ELUXEO<sup>™</sup> VIDEO GASTROSCOPE Ultra Slim Type

# 

This ultra slim gastroscope with a distal end diameter of 5.8 mm is expected to be useful for narrow GI anatomy and for cases featuring stenosis. The slim distal end also supports a soft transnasal insertion and offers the potential to alleviate patients' discomfort.



Field of view	140°
Observation range	3–100 mm
Bending capability	Up 210°/Down 90° Right 100°/Left 100°
Ø Distal end	5.8 mm
Ø Flexible portion	5.9 mm
Ø Working channel	2.4 mm
Working length	1,100 mm
Total length	1,400 mm

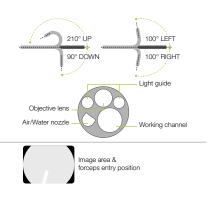


# EI-740D/S ELUXEO" VIDEO GASTROSCOPE & SIGMOIDOSCOPE Dual Channel

The ELUXEO<sup>™</sup> EI-740D/S is intended for use in both the upper gastrointestinal tract and sigmoid colon and combines the former EG-530D gastroscope, and ES-530WE sigmoidoscope in one exceptional endoscope. With its two instrument channels (3.7 and 3.2 mm) and improved suction performance, it is especially suitable for various therapeutic procedures.



Field of view	140°
Observation range	3–100 mm
Bending capability	Up 210°/Down 90° Right 100°/Left 100°
Ø Distal end	12.8 mm
Ø Insertion tube	12.8 mm
Ø Working channel	3.7 mm/ 3.2 mm
Working length	1,030 mm
Total length	1,330 mm

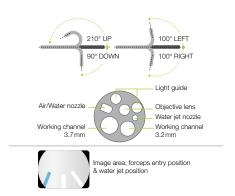

#### Large dual instrument channels


Compared to the 530 series dual instrument channel endoscopes, the EI-740D/S endoscope has an improved instrument channel capacity. The smaller instrument channel is increased from 2.8 to 3.2 mm, therefore providing an 3.2 and 3.7 mm dual instrument channel configuration. By accommodating two endotherapeutic devices simultaneously, the endoscope allows for a broader range of therapeutic possibilities.



# EG-720R ELUXEO Lite VIDEO GASTROSCOPE Routine Type



Field of view	140°
Observation range	2-100 mm
Bending capability	Up 210°/Down 90° Right 100°/Left 100°
Ø Distal end	9.2 mm
Ø Flexible portion	9.3 mm
Ø Working channel	2.8 mm
Working length	1,100 mm
Total length	1,400 mm

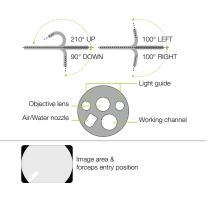

# DOUBLE BALLOON ENDOSCOPY SYSTEM



## EXAMINING THE SMALL INTESTINE IN DOUBLES



By developing the double balloon endoscope, Fujifilm pioneered examining and treating the entire small intestine. The two-balloon system provides an unparalleled level of detail and is, to this day, the gold standard in examination of the small intestine. It is also commonly used in ERCPs with altered conditions post-surgery.

**ONE-STEP CONNECTOR** 

**PUSH & PULL SYSTEM** 

ENLARGED WORKING CHANNEL OF 3.2 mm FOR EFFICIENT TREATMENT

#### **DOUBLE BALLOON ENDOSCOPY**

Double balloon endoscopy is a technique that allows the whole length of the small intestine to be visualised, opening doors to many therapeutic interventions.

Fujifilm developed the DBE system to meet the clinical needs for more precise and efficient diagnoses and treatment.



Oral insertion (small intestine)




Anal insertion (small intestine)

#### Working channel with 3.2 mm diameter

The enlarged 3.2 mm working channel suits procedures such as haemostasis and balloon dilation. As it enables blood or mucus to be aspirated while a therapeutic device is inserted, quicker haemostasis is possible. The large working channel is also designed for easier insertion and removal of a balloon catheter before and after dilation of stricture.



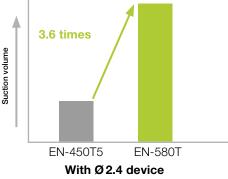

The 3.2 mm working channel provides greater suction performance than conventional models.\*




# Specially designed one-step connector and relocated balloon air feed inlet for better operability


The balloon air feed inlet has been relocated from the control portion to the connector portion, creating a better examination environment. Also, a one-step connector (TY-06 or TY-500) specially designed for the balloon air feed inlet on the endoscope is provided, simplifying the preparation.

\* According to Fujifilm data

# **EN-580T** ENTEROSCOPE Therapeutic Type

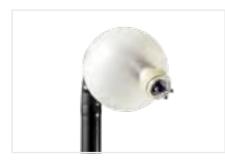

Viewing direction	0° (Forward)	
Field of view	140°	180° UP 180° DOWN
Observation range	2-100mm	
Bending capability	Up 180°/Down 180° Right 160°/Left 160°	Objective lens
Ø Distal end	9.4mm	Air/Water nozzle
Ø Flexible portion	9.3mm	
Ø Working channel	3.2mm	Image a
Working length	2,000mm	forceps
Total length	2,300mm	

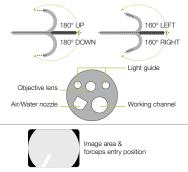

160° LEFT 160° RIGHT

Light guide

Working Channel


# EN-580XP ENTEROSCOPE Slim Type




Viewing direction	0° (Forward)	
Field of view	140°	180° UP 16 180° DOWN 16
Observation range	2-100mm	
Bending capability	Up 180°/Down 180° Right 160°/Left 160°	Objective lens
Ø Distal end	7.5mm	Air/Water pozzle
Ø Flexible portion	7.7 mm	All/Water hozzie
Ø Working channel	2.2mm	Image area &
Working length	2,000mm	forceps entry position
Total length	2,300mm	

#### EI-580BT SHORT DOUBLE BALLOON ENDOSCOPE

# 

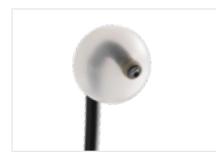

Viewing direction	0° (Forward)
Field of view	140°
Observation range	2–100mm
Bending capability	Up 180°/Down 180° Right 160°/Left 160°
Ø Distal end	9.4mm
Ø Flexible portion	9.3mm
Ø Working channel	3.2mm
Working length	1,550mm
Total length	1,850mm

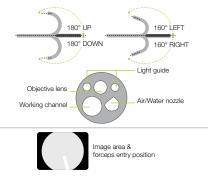

Some tubes and balloons used for EN-580T and EN-580XP are made from non-latex materials.


## PB-30 BALLOON CONTROL UNIT

To be used to control pressure inside the balloons which are inflated and deflated during DBE examinations.



Maximum flow rate of pump	170ml±50ml/10 sec.
Set pressure accuracy	±2 kpa
Set pressure of balloon	5.6 kpa
Weight	7.0 kg (Main unit) 0.4 kg (Remote switch)
Power	AC100–240V 50/60 Hz/0.8A
Dimensions (W x H x D)	145 x 170 x 410 mm

#### ST-05B/ST-10 BALLOON SETTING TOOLS

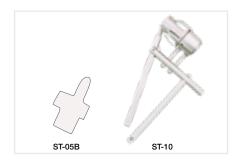

To fix the balloon and the rubber bands.





# LOWER GI ENDOSCOPY



# **AI FOR COLONOSCOPY**

All ELUXEO<sup>™</sup> 700 series colonoscopes are compatible with Fujifilm's CAD EYE software to support physicians to detect and characterise colonic polyps, developed utilising Al technology.

#### **ELUXEO<sup>™</sup>ULTRA**

The Eluxeo<sup>™</sup> Ultra platform features a range of tools and technologies. It will evolve continuously, so our best endoscopy system for your patients today will remain so tomorrow.

## EC-760R-V/M, I, L ELUXEO" VIDEO COLONOSCOPE Routine Type

Field of view

Ø Distal end

Observation range

Bending capability

This routine colonoscope has a wide field of view of 170° as well as a large working channel diameter of 3.8 mm. It features the G7 grip and the Flexibility Adjuster. In addition, it has a slim diameter of 12.0 mm and includes a water jet function and CMOS technology.

170°

2-100 mm


12.0 mm

Up 180°/Down 180°

Right 160°/Left 160°




# Ø Flexible portion 12.0 mm Ø Working channel 3.8 mm Working length 1,330/1,520/1,690 mm Total length 1,650/1,840/2,010 mm

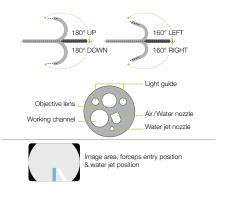


#### Wide 170° field of view

With EC-760R and EC-760P, a wide 170° field of view is available. It is designed to observe and approach smoothly, even areas that are hard to observe, such as the reverse side of folds.

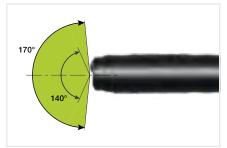


# EC-760Z-V/M, L ELUXEO<sup>®</sup> VIDEO COLONOSCOPE Optical Magnification

The zoom colonoscope is an all-rounder. It features the brilliant and easy-to-operate 145x Multi Zoom\* magnification which leads to more detailed visualisation, supporting a deeper analysis of mucosal and vascular patterns. Compared to the EC-760ZP ultra slim zoom type, it comes with a stiffer insertion tube and a larger working channel (3.8 mm vs. 3.2 mm), meaning it is also well-suited for basic therapeutic procedures.

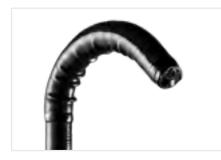


Field of view	Normal 140°/Close 56°
Observation range	1.5–100 mm Normal 3–100 mm Close 1.5–2.5 mm
Bending capability	Up 180°/Down 180° Right 160°/Left 160°
Ø Distal end	12.8 mm
Ø Insertion tube	12.8 mm
Ø Working channel	3.8 mm
Working length	1,330 mm/1,690 mm
Total length	1,650 mm/2,010 mm

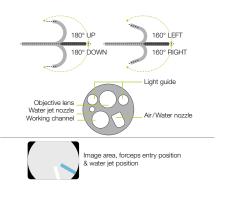

# EC-760ZP-V/M, L ELUXEO VIDEO COLONOSCOPE Optical Magnification


The slim zoom colonoscope features the brilliant and easy-to-operate Multi Zoom with 145 x maximum magnification\*. Together with BLI, details of the mucosal and vascular patterns become visible. Like the routine endoscope, it features the full range of functionalities including flexible adjustment even with the slim diameter of 11.8 mm.

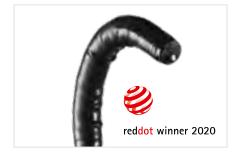

Field of view	Normal 140°/Close 56°	180° UP
Observation range	1.5–100mm Normal 3–100mm Close 1.5–2.5mm	180° DOWN 160° RIGHT
Bending capability	Up 180°/Down 180° Right 160°/Left 160°	Light guide
Ø Distal end	11.7 mm	Objective lens
Ø Flexible portion	11.8mm	Working channel
Ø Working channel	3.2mm	
Working length	1,330/1,690mm	Image area, forceps entry position & water jet position
Total length	1,650/2,010mm	

# EC-760P-V/M, L ELUXEO<sup>\*\*</sup> VIDEO COLONOSCOPE Ultra Slim Type

This ultra slim colonoscope from the ELUXEO<sup>™</sup> 760 series has a distal end diameter of only 11.1 mm and is therefore expected to be useful for narrow GI anatomy, cases featuring stenosis and therapeutic use. A wide 170° field of view enables a visualisation even in hard-to-observe areas. It features the G7 grip and the Flexibility Adjuster for easier insertion.

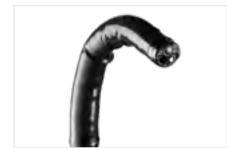


Field of view	170°	180° UP	
Observation range	2-100mm		
Bending capability	Up 180°/Down 180° Right 160°/Left 160°	180° DOWN 160° RIGHT	
Ø Distal end	11.1 mm	Objective lens	
Ø Flexible portion	11.5mm		
Ø Working channel	3.2mm	Working channel Air/Water nozzle	
Working length	1,330/1,690mm		
Total length	1,650/2,010mm	Image area, forceps entry position & water jet position	

\* In combination with 26" screen The CAD EYE AI technology is available for all colonoscopes.


## EC-740T/M, L ELUXEO" VIDEO COLONOSCOPE Slim & Treatment Type

# 

This slim colonoscope is equipped with Advanced Force Transmission, 210° up-angulation and a G7 grip that supports excellent manoeuvrability. It is expected to be useful for more challenging anatomies and narrow GI anatomy, such as stenosis, severe inflammation, or anatomical adhesion. With the additional observation modes LCI, intended to improve detection, and BLI, intended to characterise lesions, this is an excellent colonoscope for both observation and therapeutic procedures.

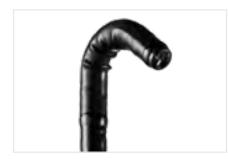

Field of view	140°
Observation range	3–100mm
Bending capability	Up 210°/Down 160° Right 160°/Left 160°
Ø Distal end	9.8mm
Ø Flexible portion	10.7 mm
Ø Working channel	3.2mm
Working length	1,330/1,690mm
Total length	1,630/1,990mm

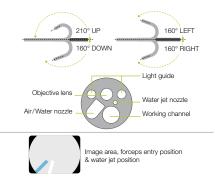

#### SMART BEND

1) t Bend

Smart Bend provides excellent manoeuvrability, observation and therapeutic treatments from 210° up-angulation and a small bending radius. It is expected to be useful for treatment of difficult-to-reach lesions.



Smart Bend colonoscope: Up to 210°




Colonoscope without Smart Bend

# EI-740D/S ELUXEO VIDEO GASTROSCOPE & SIGMOIDOSCOPE Dual Channel

The ELUXEO<sup>™</sup> EI-740D/S is intended for use in both the upper gastrointestinal tract and sigmoid colon and combines the former EG-530D gastroscope, and ES-530WE sigmoidoscope in one exceptional endoscope. With its two instrument channels (3.7 and 3.2 mm) and improved suction performance, it is especially suitable for various therapeutic procedures.



Field of view	140°
Observation range	3–100 mm
Bending capability	Up 210°/Down 90° Right 100°/Left 100°
Ø Distal end	12.8 mm
Ø Insertion tube	12.8 mm
Ø Working channel	3.7 mm/ 3.2 mm
Working length	1,030 mm
Total length	1,330 mm

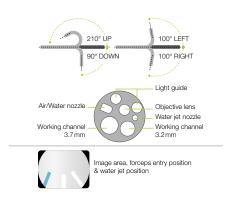


#### Large dual instrument channels


Compared to the 530 series dual instrument channel endoscopes, the EI-740D/S endoscope has an improved instrument channel capacity. The smaller instrument channel is increased from 2.8 to 3.2 mm, therefore providing an 3.2 and 3.7 mm dual instrument channel configuration. By accommodating two endotherapeutic devices simultaneously, the endoscope allows for a broader range of therapeutic possibilities.



# EC-720R/M, I, L ELUXEO Lite VIDEO COLONOSCOPE Routine Type

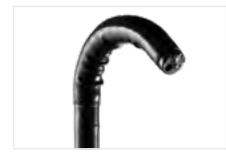

Field of view	170°				
Observation range	2-100 mm				
Bending capability	Up 180°/Down 180° Right 160°/Left 160°				
Ø Distal end	12.8 mm				
Ø Flexible portion	12.8 mm				
Ø Working channel	3.8 mm				
Working length	1,330/1,520/1,690 mm				
Total length	1,630/1,820/1,990 mm				

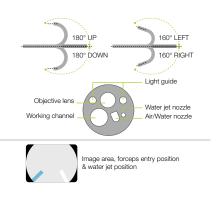

# **COLOASSIST PRO:** ENDOSCOPE VISUALISATION SYSTEM








The endoscope shape is to be displayed on the monitor for VP-7000 by using the PoP function


# **EXCELLENT VISUALISATION**

Fujifilm's Endoscope Visualisation System displays the configuration of the endoscope in real-time by reproducing a coloured graphical representation of the endoscope next to endoscopic view.

The three display modes support physicians in understanding the behaviour of colonoscopes during the intubation, to enable the identification of loop formation and to reduce patient burden during an endoscopy.

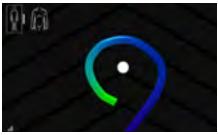

Single Screen Mode: Displays the endoscope model from one direction (out of 4)

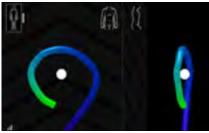

Dual Screen Mode: Displays the endoscope model from two directions




Sky View Mode: Displays the endoscope model obliquely from above the foot section of the patient

# EC-760S-G/L, M ELUXEO" COLONOSCOPE Routine Type

This routine colonoscope with visualisation function from the ELUXEO<sup>™</sup> 760 series is equipped with CMOS technology and provides HD images and videos for daily practice. Close Focus allows observation from a distance as close as 2 mm.



Field of view	170°			
Observation range	2–100 mm			
Bending capability	Up 180°/Down 180° Right 160°/Left 160°			
Ø Distal end	12.8 mm			
Ø Flexible portion	12.8 mm			
Ø Working channel	3.8 mm			
Working length	1,330/1,690mm			
Total length	1,650/2,010 mm			
Endoscope Visualizer	Compatible			

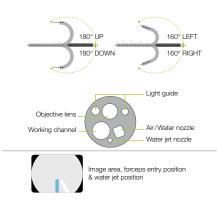

### **EV-3D** ENDOSCOPE VISUALIZER



Power	100–240V ~ 50/60 Hz
Current consumption	1.0-0.7A
Video Output	DVI-I (1280x1024px/ 1920x1080px) HD-SDI (1920x1080px)
Supported endoscopes	EC-760S-G/L, M
Dimensions (W x H x D)	380 x 80 x 445 mm (including projection)
Weight	6.5kg
Frequency band	200–2000 Hz

The system requires the Endoscope Visualizer EV-3D with peripherals (handmarker, transceiver dish) as well as the dedicated colonoscope (EC-760S-G/L,M). It is compatible with CAD EYE AI technology.



Design Award Winner Endoscope Visualisation System

veddot winner 2022



# **ELUXEO**<sup>™</sup> FEATURING **G-EYE**



## **FLATTENED TOPOGRAPHY**

The G-EYE® 760R endoscope is equipped with a permanently integrated balloon at the bending section of the routine colonoscope. On demand, the reusable balloon can be inflated, thereby flattening the colonic walls and improving the detection of hidden polyps\*. It is also compatible with CAD EYE AI technology.

#### FOR INCREASED ADR\*

CONTROLLED WITHDRAWAL™ CENTRALISES ENDOSCOPE OPTICS, REDUCES BOWEL SLIPPAGE

#### **ENDOSCOPE STABILISATION DURING INTERVENTIONS**

\* "G-EYE colonoscopy is superior to standard colonoscopy for increasing adenoma detection rate: an international randomized controlled trial", Shirin H. et al., Gastrointestinal Endoscopy 2019; 89(3): 545-553 Halpern, Z. et al. Comparison of adenoma detection and miss rates between a pouch believe adenoscopy or prodemized toadem attributed to adenoscopy or prodemized toadem attributed to adenoscopy or prodemized toadem attributed to adenoscopy of the prodemized toadem attributed to additional trademized to additional to additional trademized to additional to additional trademized to additi

novel balloon colonoscope and standard colonoscopy: a randomized tandem study (Endoscopy 2015; 47(03): 238-244 DOI: 10.1055/s-0034-1391437) Assisting visualisation, stabilisation and control during colonoscopy: Besides detection enhancement, physicians could benefit from the G-EYE<sup>®</sup> system throughout the whole procedure, from assistance in delooping during intubation, via Controlled Withdrawal<sup>™</sup> that reduces bowel slippage, through to supporting therapeutic interventions e.g. EMR/ESDs by stabilising and anchoring the endoscope tip.

#### INTUBATION



G-EYE<sup>®</sup> could be used for delooping with anchoring function

#### DETECTION



G-EYE<sup>®</sup> flattens topography to detect hidden polyps

Controlled Withdrawal<sup>™</sup> with partially inflated balloon



#### **CHARACTERISATION**



G-EYE<sup>®</sup> stabilises the endoscope during characterisation

Controlled Withdrawal<sup>™</sup> with partially inflated balloon



#### TREATMENT




G-EYE<sup>®</sup> stabilises the endoscope during treatment

### **G-EYE® 760R**



The G-EYE<sup>®</sup> endoscope is based on the ELUXEO<sup>™</sup> EC-760R routine colonoscope, comes in three different lengths and features the same technical specifications including LCI and BLI imaging modes. To operate, the Spark<sup>2</sup>C Inflation System is required.

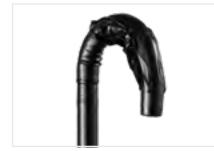


Field of view	170°	
Observation range	2-100 mm	180° UP 160° LEFT
Bending capability (when deflated)	Up 180°/Down 180° Right 160°/Left 160°	
Ø Distal end	12.0 mm	Light guide
Ø Insertion tube	12.0 mm	Objective lens
Ø Working channel	3.8 mm	Working channel Water jet nozzle
Working length	1,330/1,520/1,690 mm	
Total length	1,650/1,840/2,010 mm	Image area, forceps entry position & water jet position
Balloon diameter	Up to 55 mm	



SPARK<sup>2</sup>C Inflation System

Manufactured by Smart Medical Systems Ltd.

# CAD EYE: *Eluxeo*™ meets Artificial intelligence

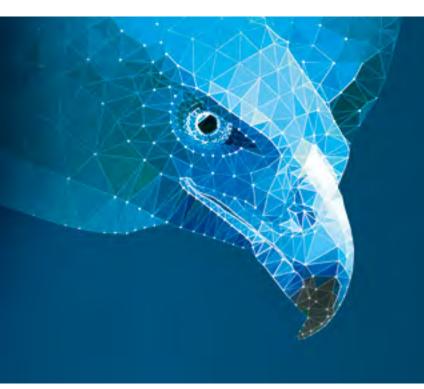

### **INTELLIGENT SUPPORT**

CAD EYE works with the expansion unit EX-1 and the CAD EYE software EW10-EC02. The documentation software EW10-SC01 enables still image and movie recording (up to 30 hours) in the EX-1 memory and features data transfer via the network. It can easily be controlled with the endoscope switch or directly at the processor.



# FOR COLONIC POLYP DETECTION & CHARACTERISATION

MOVIE/STILL IMAGE RECORDING & NETWORK FUNCTION

> FUTURE CAD APPLICATIONS CAN BE INSTALLED

COMPATIBLE WITH ALL ELUXEO<sup>™</sup> 700 SERIES COLONOSCOPES, G-EYE 760R & COLOASSIST PRO




The novel function CAD EYE has been developed utilising AI deep learning technology and is compatible with Fujifilm's ELUXEO<sup>™</sup> endoscopy series to support endoscopic lesion detection and characterisation in the colon.


### **REAL-TIME DETECTION**



CAD EYE is aimed at improving the real-time polyp detection rate to expert level, helping recognise flat lesions, multiple polyps simultaneously as well as any lesions at the corner of the image. CAD EYE Detection is possible with White Light and LCI (Linked Color Imaging) mode.

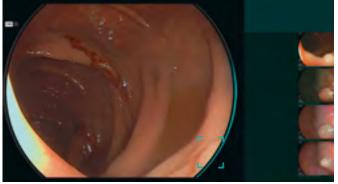


White Light Mode

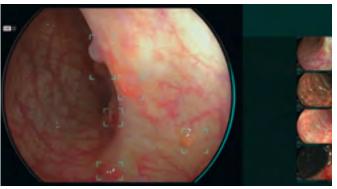


LCI Mode

# CHARACTERISATION SUPPORT

Once a suspected polyp is detected by CAD EYE Detection (WLI or LCI), CAD EYE Characterisation - in combination with BLI can support endoscopists in the predictive histopathological diagnosis of the polyp. This function analyses in real-time and without freezing or zooming if a polyp is hyperplastic or neoplastic, which is visually indicated by the use of different colour codes in the Position Map. CAD EYE Characterisation is intended to make procedures more efficient by increasing the accuracy of diagnosis to an experts' level.\*

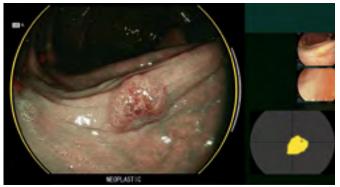

BLI Mode - Neoplastic

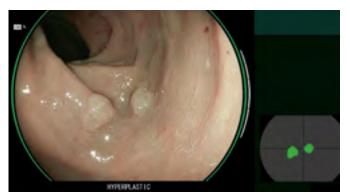

BLI Mode - Hyperplastic





# LIGHT SOURCES & PROCESSORS



BL-7000 | VP-7000 | EP-6000

Light Source & Processor 7000	48
Light Source & Processor 6000	50

# BL-7000 & VP-7000



## FOR DEMANDING PROCEDURES

Our long-standing experience in developing imaging technologies also provides you with an excellent processor for your application at all times. All models offer digital image processing and video interfaces. With ergonomic user controls, these video processors save valuable time and facilitate more comfortable examinations.

Video Processor VP-7000



# FUJIFILM group

### **BL-7000** ELUXEO<sup>®</sup> 4-LED LIGHT SOURCE with High Durability

A reliable light source is a prerequisite for use in large clinics as well as smaller outpatient centres to ensure procedures can take place as scheduled. To achieve high standards, the eco-friendly ELUXEO<sup>™</sup> 7000 system features the 4-LED Multi Light<sup>™</sup> Source, which outperforms conventional Xenon or Halogen light sources: With 10,000 hours' average life expectancy for the LED lights, the ELUXEO<sup>™</sup> system has far longer durability while having much lower energy consumption, resulting in better cost-efficiency.

Light source	4-LED
Air supply pump	High, Mid, Low, Off
Power rating	100-240V, 50/60Hz, 1.2-0.7A
Dimensions (W x H x D)	390 x 155 x 485 mm (including projection)
Weight	12.0 kg
Optical radiation safety	Class 1 LED product

			1	1	ł
 -					
		Life expectancy in hours'	Life expectancy in hours <sup>1</sup>	Life expectancy in hours <sup>1</sup>	

Our confidence in the ELUXEO system BL-7000 is reflected by Fujifilm's **Durability Warranty**, which covers any defect of the LED light source unit that is attributable to a manufacturing or assembly fault under normal use for a period of five years or 10,000 operating hours, whichever comes first.<sup>1</sup>

### VP-7000 ELUXEO™ HIGH-PERFORMANCE VIDEO PROCESSOR

#### Hulti Light technology

The ELUXEO<sup>™</sup> video processor VP-7000 enables you to make use of the many features provided by Fujifilm's wide range of endoscopes along with the 4-LED illumination system and its LCI and BLI visualisation modes. In addition to the ELUXEO 700 series endoscopes, it is also compatible with the 600 and 500 series of endoscopes. The processor creates high-quality images displayed in full HD on the monitor. Automatic back-up mode for data storage is integrated and the processor is DICOM compatible.

Compatible endoscopes	700/600/500 series
Output	DVI-D x2, DVI-I x1, HD-SDI x2, RGB-TV x1, S VIDEO x1, VIDEO x1
Input	1 channel PoP
External memory	USB Flash Drive
Power rating	100-240V, 50/60Hz, 0.8-0.5A
Dimensions (W x H x D)	390 x 110 x 485 mm (including projection)
Weight	9.0 kg

# EP-6000 ELUXEO"Lite VIDEO PROCESSOR with Built-in LED Light Source

The ELUXEO<sup>™</sup> Lite EP-6000 combines a reliable 3-LED light source with a processor that enables you to make use of the many features provided by Fujifilm's wide range of endoscopes. Available combined with the 700 series LCI (Linked Color Imaging) and BLI (Blue Light Imaging) visualisation modes.

Due to the use of economical LED lamps with a long durability this system is very eco-friendly. It is also compatible with the 600 and 500 series of endoscopes. The ELUXEO<sup>™</sup> Lite EP-6000 creates quality images and videos displayed in full HD on the monitor.

Automatic back-up mode for data storage is integrated and the processor is also DICOM compatible.

Light source	3-LED
Air supply pump	High, Mid, Low, Off
Compatible endoscopes	700, 600 and 500 series endoscopes*
Output	DVI-D x2, RGB-TV x1, S VIDEO x1, VIDEO x1
External memory	USB Flash Drive
Power rating	100–240V, 50/60Hz, 2.0–1.1A
Dimensions (W x H x D)	395 x 210 x 485 mm (including projection)
Weight	15.0 kg
Optical radiation safety	Class 1 LED product

Available observation modes	White Light	BLI	LCI	FICE
700 series	•	•	•	•
600/500 series	٠			٠



\* Excluding 590 series endoscopes, EG-530UT2, EG-530UT, EG-530UR2 and EG-530UR.



# ESD



CLUTCHCUTTER | FLUSHKNIFE | JW-3 | PURASTAT

Hoods	52
Knives	54
Water Pump	58
Injection Needles	59
Haemostat	60
Insufflator	60



#### ST HOODS

ST Hoods have a transparent, small caliber tip, maintaining a clear field of view.

Model	Material Code	Characteristics	Ø Outer (mm)	Ø Inner tip (mm)	Tip length (mm)	Accessory guide	Applicable endoscopes	Unit
DH-28GR	16426426	Transparent; silicone rubber, polycarbonate resin	11.5	8.0	7.0	Without	EG-760Z EG-600ZW EC-740T/M, L	5
DH-29CR	16426438	Transparent; silicone rubber, polycarbonate resin	12.4	8.0	7.0	Without	EG-760CT EC-760P-V/M, L	5
DH-30CR	16426440	Transparent; silicone rubber, polycarbonate resin	13.8	8.0	7.0	Without	EC-760R-V/M, I, L EC-760ZP-V/M, L EC-760Z-V/M, L EC-720R/M, I, L EC-600WM/WI/WL EC-600ZW/M, L	5
DH-33GR	16635493	Transparent; guide type; silicone rubber, polycarbonate resin	11.8	7.0	7.0	Yes	EG-760Z EG-760CT EC-740T/M, L	5
DH-34CR	16635508	Transparent; guide type; silicone rubber, polycarbonate resin	13.6	7.0	7.0	Yes	EC-760R-V/M, I, L EC-760ZP-V/M, L EC-600WM/WI/WL	5

2

2



#### ST Hood features

- Providing a clear and stable view by maintaining the distance between the distal end and mucosal layer
- Side holes for better drainage of fluid to maintain a clear endoscopic view
- Sterile and single-use for better infection control
- Guide groove to stabilise endotherapeutic devices

OBSERVATION HOOD	

<b>DH-17EN</b> Transparent 11.5 1.5 EI-580BT, EN-580T	Model	Characteristics	Ø Outer (mm)	Tip length (mm)	Applicable endoscopes
	DH-17EN	Transparent	11.5	1.5	EI-580BT, EN-580T



DH-17EN

### **TOP** ENDOSCOPIC HOODS\*

The slit and drainage hole allows fluid to drain away quickly.

Model	Material Code	Characteristics	Ø Outer (mm)	Tip length (mm)	Applicable endoscopes	Unit
SHS	70100142568	Soft and transparent	8.6-9.8		EG-760R	5
SHM	70100139249	Soft and transparent	9.8-10.6		EG-760Z EG-760CT EC-740T/M, L	5
SHL	70100139248	Soft and transparent	10.6 - 12.6		EC-760R-V/M, I, L EC-760ZP-V/M, L EC-760P-V/M, L	5
SHLL	70100143067	Soft and transparent	12.6 -14.0		EC-760Z-V/M, L	5
SHM-B	70100139250	Soft and black	9.8–10.6		EG-760Z EG-760CT EC-740T/M, L	5
SHL-B	70100139251	Soft and black	10.6 - 12.6		EC-760R-V/M, I, L EC-760ZP-V/M, L EC-760P-V/M, L	5
SHF-020	70100143067	Hard and transparent	9.8	7.0	EG-760Z EC-740T/M, L	5
SHF-035	70100139658	Hard and transparent	10.8	7.0	EG-760CT EC-760P-V/M, L	5
SHF-045	70100141234	Hard and transparent	11.5	7.0	EC-760R-V/M, I, L	5
SHF-050	70100139656	Hard and transparent	12.2	7.0	EC-760ZP-V/M, L	5
SHF-055	70100139657	Hard and transparent	13.2	7.0	EC-760Z-V/M, L	5



SHM-B / L-B





THIRD PARTY PRODUCT

\* Manufactured by Top Corporation, Japan. Not for sale in Canada.



# CLUTCHCUTTER



# THE 3-IN-1 ESD TOOL FOR VARIOUS TECHNIQUES

The ClutchCutter is a well proven endoscopic mucosal dissection tool that can enable the user to grasp, and electrically dissect and coagulate the mucosa of the upper and lower GI tract. In procedures that need to be performed perpendicular to the muscular layer, the ClutchCutter makes it possible to grasp, lift up and incrementally dissect the tissue.



TOOTHED JAWS TO GRIP THE MUCOSA EFFICIENTLY

**ROTATABLE DISTAL JAWS FOR A PRECISE LESION APPROACH** 

INSULATED OUTER EDGE TO REDUCE RISK OF DAMAGING LOCAL TISSUE

### **CLUTCH CUTTER** SINGLE USE ESD KNIFE

#### 3-in-1 ability

DP2618DT -35-: 3.5 mm jaw length

All stages of ESD can be performed with this 3-in-1 tool, from marking to making incisions and coagulating.

DP2618DT -50-: 5.0 mm jaw length



1. Incision



2. Dissection



3. Coagulation

# **FLUSHKNIFE**



FlushKnife Ns

FlushKnife Bis

## ENHANCED USABILITY IDEAL FOR ALL PHYSICIANS

One knife covers everything from marking (coagulation), dissection to arrest of bleeding. The high versatility of this knife improves the efficiency of the procedure by reducing the exchange of instruments during the procedure.

THINNER SHEATH FOR ENHANCED SUCTION PERFORMANCE<sup>1</sup> EXCELLENT MANOEUVRABILITY AND STABILITY<sup>2</sup> FLUID SUPPLY CAPABILITY WITH WATER PUMP JW-3

> Due to tapered catheter design between distal section and handle to increase working channel lumen
>  Due to increased catheter diameter on the distal section to secure stable position in the working channel

#### **FLUSHKNIFE BTS**

FlushKnife BTS has a ball tip which touches a wider range of tissue and supports effective coagulation.

Model	Material Code	Characteristics	Identifier	Colour code	Knife length (mm)	Working length (mm)	Ø Working channel (mm)
DK2620JI	16659607 16659619 16659621 16659633	Ball tip, Slim Type	- B15 - - B20 - - B25 - - B30 -	Green Blue Yellow Black	1.5 2.0 2.5 3.0	2,000	2.8 or more
DK2623JI	16659669 16659671	Ball tip, Long slim type	- B15 - - B20 -	Green Blue	1.5 2.0	2,300	2.8 or more


#### **FLUSHKNIFE NS**

FlushKnife NS has a slim needle shaped tip.

Model	Material Code	Characteristics	Identifier	Colour code	Knife length (mm)	Working length (mm)	Ø Working channel (mm)
DK2620JI	16659554 16659566 16659578 16659580 16659592	Needle tip, Slim Type	- N10 - - N15 - - N20 - - N25 - - N30 -	White Green Blue Yellow Black	1.0 1.5 2.0 2.5 3.0	2,000	2.8 or more
DK2623JI	16659645 16659657	Needle tip, Long slim type	- N15 - - N20 -	Green Blue	1.5 2.0	2,300	2.8 or more




1. Marking (coagulation)

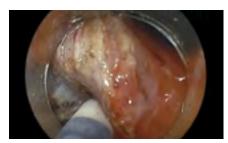


4. Submucosal dissection



2. Water jet

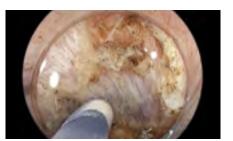

5. Submucosal supply

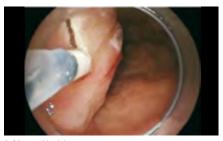

3. Mucosal incision



6. Coagulation

#### **Recommended applications**

Oesophagus     O     A     A       Stomach     O     O     O     O       Colon     O     O     A     A	Diameter	1 mm	1.5mm	2mm	2.5mm	3mm
	Oesophagus				$\triangle$	$\triangle$
	Stomach	$\bigcirc$	$\bigcirc$	$\bigcirc$	$\bigcirc$	$\bigcirc$
	Colon	$\bigcirc$			$\bigtriangleup$	$\triangle$

Please note: Examples of the suitable protruding lengths are suggested by Takashi Toyonaga M.D. of Kobe University Hospital. A physician must take into consideration each particular condition of the area or lesion to be dissected when selecting a protruding knife length.

 $\bigcirc$  Best indication  $\bigcirc$  Possible Use  $\triangle$  Indicated in certain cases

57





#### JW-3 WATER PUMP

Advanced piping technology enables water flow to be quickly stopped. The one litre water bottle enables prolonged water use and minimises the need for constant refilling.

#### Water jet system and rotable water supply port

The water jet system keeps the tip of the knife clean by washing off debris and lesion tissue adhering to the tip, thereby maintaining the sharpness of the knife throughout the treatment. The rotable water supply port allows comfortable usage of both tube and syringe connection.







(2)

Design Award Winner JW-3 Water Pump



#### FOR INFECTION PREVENTION AND SAFETY



Tip of JT-3DT to connection to saline bag

#### Autoclavable tubes and water tank, disposable tubes

Both tubes (JT-3RC, JT-3RC2, and JT-3RW) and water tank are autoclavable for effective sterilisation and are re-usable. In addition, sterilised disposable tube (JT-3DC, JT-3DW, and JT-3DT) can be selected in view of infection management.

# A line-up for therapeutic device (FlushKnife Slim: DK2620JI and DK2623JI) connection tube capable of submucosal supply

JT-3DT, designed for therapeutic device (FlushKnife Slim: DK2620JI and DK2623JI) is a disposable tube that connects to a saline bag via a port connector. By combining this tube with FlushKnife Slim: DK2620JI and DK2623JI, it's possible to supply saline solution into the submucosal layer during ESD treatment.

#### For safety use, to avoid human error


If the cover of the pump head is opened during operation, operation is automatically stopped to protect users from injuries and accidents. The tube detector automatically recognises which tube is in use. In consideration of safety use, the maximum flow rate is set for each tube. During continuous flow, the water supply is automatically stopped after a set duration for each type of detected tube, thus avoiding an incidence of human error.

Max. water flow rate and
automatic cut-off timer

Tube-Typ	Max. water flow rate	Automatic cut-off timer
A: Instrument channel tube	840 ml/min.	approx. 20 seconds
B: Water jet tube	220 ml/min.	approx. 20 seconds
C: Disposable therapeutic tube	209 ml/min.	approx. 8 seconds

### WATER PUMP ACCESSORIES

Product name	Material Code	Where to use	Characteristics	Total tube length (mm)	Unit
JT-3RC Instrument channel tube	16642836	Instrument channel (mode A)	Suction/irrigation tubing, reusable	2,230	1
JT-3RC2 Instrument channel-joint tube	16642850	Instrument channel (mode A)	Suction/irrigation tubing, reusable	195	5
JT-3DC Disposable instrument channel tube	16642862	Instrument channel (mode A)	Endoscopic insufflation tubing set, single-use	2,065	10
JT-3RW Water jet tube	16642848	Water jet (mode B)	Suction/irrigation tubing, reusable	1,340 Luer connector	2
JT-3DW Disposable water jet tube	16642874	Water jet (mode B)	Endoscopic insufflation tubing set, single-use	1,330 Luer connector	10
JT-3DT Disposable therapeutic tube	16642886	Therapeutic device (mode C)	Endoscopic insufflation tubing set, single-use	3,350 Luer connector	10
WT-3JW Water tank	16642824	Water tank	Surgical irrigation pump		2

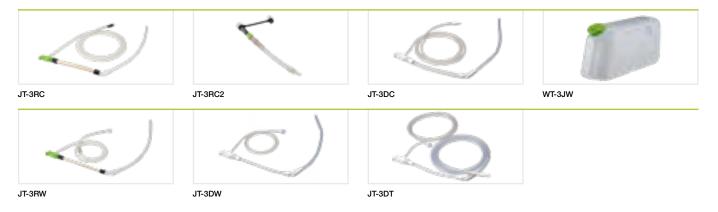

#### MANTA INJECTION NEEDLES

Manta ensures reliable retraction and extension of the needle, even in acutely angulated device positions, and prevents damage to the working channel. Enhanced kink-resistance and single handed operation provide for ease of use.

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A3-05-5-18-240	70100146666	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	2,400	2.0	5
INJ1-A3-07-5-18-240	70100146657	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,400	2.0	5
<b>_</b>	<u> </u>					

INJ-A3



#### **PURASTAT HAEMOSTAT\***

#### THIRD PARTY PRODUCT

PuraStat is a slightly viscous solution of synthetic peptides. When in contact with blood, the synthetic peptide selfassembles into a 3-dimensional matrix which rapidly coats the bleeding point leading to haemostasis. It is transparent and can be reapplied. PuraStat can be combined with coagulation, clips etc and is applicable to narrow spaces. It is easy to handle and ready to use a pre-filled syringe.

Product code	Material Code	Characteristics	Unit
PURASTAT 1ml 621-034	70100152373	Haemostatic gel 1 ml syringe	1
PURASTAT 3ml 621-035	70100152372	Haemostatic gel 3 ml syringe	1
PURASTAT 5ml 621-036	70100152371	Haemostatic gel 5 ml syringe	1
PURASTAT Nozzle System Type-L	70100153471	Haemostatic gel catheter	5






#### GW-100 CO<sub>2</sub> INSUFFLATOR

#### Fast resorption of insufflated CO<sub>2</sub> for patient friendly examinations

- Direct connection to the hospitals' medical CO<sub>2</sub> pipeline as well as to medical CO<sub>2</sub> cylinder
- Easy-to-use CO<sub>2</sub> flow rate switching function and compact design •
- 2 controlled flow rate settings

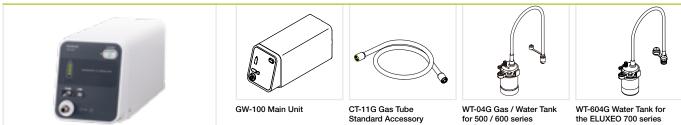

Tube sets for the connection of GW-100 to the medical gas pipeline and medical gas cylinders are available.

endoscopes



# POLYPECTOMY



CROSS SNARE ZERO | RESECTION MASTER | MANTA | OCTOPUS | FLAMINGO

Polypectomy Snares	62
Injection Needles	68
Foreign Body Retrieval Nets	70
Foreign Body Forceps	72
Foreign Body Protection Cap	72
Polyp Traps	73
Spray Catheters	74
Haemostat	74
Cytology Brushes	75
Examination Pants	75
Buried Bumper Therapy	76

# **CROSS SNARE ZERO**

# COLD POLYP RESECTION IN A NEW DIMENSION

Less effort, perfect cutting performance – that's what CrossSnare ZERO offers you thanks to its unique reinforced spiral tube design.

Equipped with extra thin cutting wire and available in two different sizes, CrossSnare ZERO is the perfect choice for polyps up to 9 mm and supports you optimally in implementing the ESGE guideline\*.

PERFECT RESECTION THANKS TO UNIQUE TUBE DESIGN MULTIPLE SNARE DIAMETERS FOR DIMINUTIVE POLYPS OPTIMAL FOR THE IMPLEMENTATION OF THE ESGE GUIDELINE\*


### CROSS SNARE ZERO POLYPECTOMY SNARES for Cold Snaring

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-Z2-07-23-220	70100147742	Kite, CWT	7	2,200	2.8	10
POL1-Z2-12-23-220	70100147743	Kite, CWT	12	2,200	2.8	10





# Large opening angle, even if the snare is partially retracted

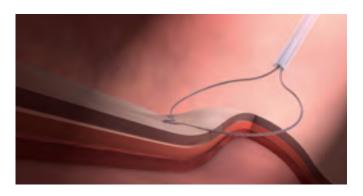

#### **Compacted Wire Technology (CWT)**

Compacted Wire Technology combines the benefits of a mono-filament cutting wire with those of a braided cutting wire:


- High energy density for a clean cut
- Low thermal input due to the CWT structure
- Precision-control exploration

### **CROSS SNARE** HYBRID POLYPECTOMY SNARES for Cold & Hot Snaring

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-X1-10-23-220-OL	70100146804	Kite, CWT	10	2,200	2.8	10
POL1-X1-15-23-220-OL	70100146758	Kite, CWT	15	2,200	2.8	10


8


2

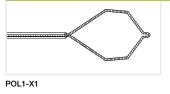

### POLYPECTOMY SNARES

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-B1-10-23-220-OL	70100146684	Convex, CWT	10	2,200	2.8	10
POL1-B1-10-23-260-OL	70100147822	Convex, balloon enteroscopy	10	2,600	2.8	10
POL1-B1-15-23-220-OL	70100146542	Convex, CWT	15	2,200	2.8	10
POL1-B1-15-23-260-OL	70100146685	Convex, balloon enteroscopy	15	2,600	2.8	10
POL1-B1-20-23-220-OL	70100146695	Convex, CWT	20	2,200	2.8	10
POL1-B1-20-23-260-OL	70100147810	Convex, balloon enteroscopy	20	2,600	2.8	10
POL1-B1-30-23-220-OL	70100146586	Convex, CWT	30	2,200	2.8	10
POL1-B1-30-23-260-OL	70100146712	Convex, balloon enteroscopy	30	2,600	2.8	10
POL1-D1-30-23-220-OL	70100146813	Convex, combi, CWT	15/30	2,200	2.8	10
POL1-K1-25-23-220-OL	70100146726	Asymmetrical, CWT	25	2,200	2.8	10
POL1-B7-10-23-220-OL	70100146697	Oval, braided wire	10	2,200	2.8	10
POL1-B7-15-23-220-OL	70100146626	Oval, braided wire	15	2,200	2.8	10
POL1-B7-20-23-220-OL	70100146698	Oval, braided wire	20	2,200	2.8	10
POL1-B7-30-23-220-OL	70100146659	Oval, braided wire	30	2,200	2.8	10
POL1-B8-10-23-220-OL	70100146627	Oval, rotable, braided wire	10	2,200	2.8	10
POL1-B8-15-23-220-OL	70100146628	Oval, rotable, braided wire	15	2,200	2.8	10
POL1-B8-22-23-220-OL	70100146675	Oval, rotable, braided wire	22	2,200	2.8	10
POL1-B8-30-23-220-OL	70100146570	Oval, rotable, braided wire	30	2,200	2.8	10
POL1-F1-30-23-220-OL	70100146740	Hexagonal, braided wire	30	2,200	2.8	10

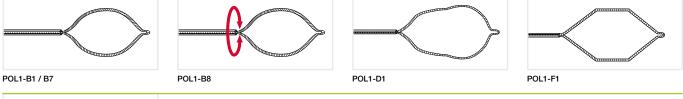

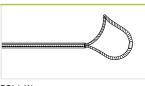

POL1-K1

# More than just a cold snare – a hybrid comprising HOT and COLD

Over 90% of all polyps are smaller than 10 mm. Especially a number of small, flat lesions in the caecum or right colon constitute special challenges for the instrumentation. CrossSnare is the first hybrid snare, and the only one, to have been specially developed for the majority of polyps of clinical relevance in terms of ablation.

The large opening angle, even if the snare is partially retracted, ensures optimal snare adjustment to match the size of the polyps between 3 mm and 15 mm. Compacted Wire Technology provides tissue-sparing cutting properties – both with and without HF current.






#### POLYPECTOMY SNARES

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-B9-10-23-220-OL	70100146687	Oval, monofil	10	2,200	2.8	10
POL1-B9-15-23-220-OL	70100146603	Oval, monofil	15	2,200	2.8	10
POL1-B9-22-23-220-OL	70100146571	Oval, monofil	22	2,200	2.8	10
POL1-B9-30-23-220-OL	70100146694	Oval, monofil	30	2,200	2.8	10
POL1-B3-10-23-220-OL	70100146624	Oval, rotable, monofil	10	2,200	2.8	10
POL1-B3-15-23-220-OL	70100146604	Oval, rotable, monofil	15	2,200	2.8	10
POL1-B3-22-23-220-OL	70100146625	Oval, rotable, monofil	22	2,200	2.8	10
POL1-B3-30-23-220-OL	70100146686	Oval, rotable, monofil	30	2,200	2.8	10





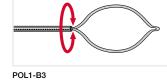

#### **RESECTION MASTER** POLYPECTOMY SNARES for EMR

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-C5-10-23-220-OL	70100146793	Oval, flat metal band	10	2,200	2.8	10
POL1-C5-15-23-220-OL	70100146802	Oval, flat metal band	15	2,200	2.8	10
POL1-C5-22-23-220-OL	70100146803	Oval, flat metal band	22	2,200	2.8	10
POL1-C5-30-23-220-OL	70100146801	Oval, flat metal band	30	2,200	2.8	10



#### Figure: POL1-C5-22-23-220-OL

Ideal resection thanks to an optimised hold on the tissue



#### Figure: POL1-B9-15-23-220-OL

Gentle tissue coagulation and controlled cut



2

# **SNARE INFLATOR** POLYPECTOMY SNARES with Integrated Injection Needle


Product code	Material Code	Characteristics	Ø Snare (mm)	Needle Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-H4-15-27-230-OL	70100146793	Oval, CWT	15	5	2,300	3.2	5
POL1-H3-22-27-230-OL	70100146802	Oval, monofil	22	5	2,300	3.2	5
POL1-H3-30-27-230-OL	70100146803	Oval, monofil	30	5	2,300	3.2	5



#### Figure: POL1-H4-15-27-230-OL

Integrated injection needle, optimal for "Lift & Cut"



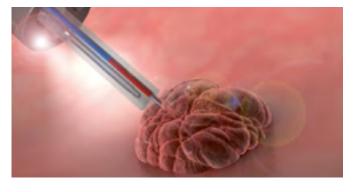



Manufactured by Fujifilm medwork GmbH.



#### MANTA INJECTION NEEDLES

Manta ensures reliable retraction and extension of the needle, even in sharply angulated device positions, and prevents feared damage to the working channel. Maximum kink-resistance and single handed operation ease the application.

2

2

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A3-05-5-18-240	70100146666	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	2,400	2.0	5
INJ1-A3-07-5-18-240	70100146657	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,400	2.0	5



INJ-A3

### ICUS INJECTION NEEDLES

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A1-05-5-18-180	70100146672	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	1,800	2.0	10
INJ1-A1-07-5-18-180	70100146611	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	1,800	2.0	10




Manufactured by Fujifilm medwork GmbH.

#### **ICUS<sup>2</sup>** INJECTION NEEDLES

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A2-05-3-23-230	70100147786	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   3	2,300	2.8	10
INJ1-A2-05-4-23-180	70100146693	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   4	1,800	2.8	10
INJ1-A2-05-4-23-230	70100146549	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   4	2,300	2.8	10
INJ1-A2-07-3-23-230	70100147814	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   3	2,300	2.8	10
INJ1-A2-07-4-23-180	70100146614	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   4	1,800	2.8	10
INJ1-A2-07-4-23-230	70100146671	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   4	2,300	2.8	10
INJ1-A2-07-5-23-180	70100146665	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	1,800	2.8	10
INJ1-A2-07-5-23-230	70100146615	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,300	2.8	10
INJ1-A2-09-5-23-180	70100146554	Latching mechanism, distal needle guide, suitable with Histoacryl	0.9   5	1,800	2.8	10
INJ1-A2-09-5-23-230	70100146598	Latching mechanism, distal needle guide, suitable with Histoacryl	0.9   5	2,300	2.8	10



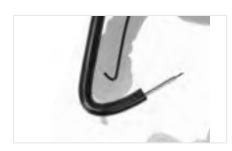

**Figure: INJ1-A2-05-3-23-230** 20% reduction\* in friction for precision puncture



80% less effort\* when administering injections



20% easier\* piercing

\* Compared to previous injection needles

# OCTOPUS



# **HIGH VOLUME**

There is no escape: The OCTOPUS retrieval net takes on polyps and foreign bodies. The sturdy structure of the frame ensures reliable removal.

Versatility in use, it can be used for several foreign bodies – due to its reservoir function. Direct rotatability and a high degree of flexibility facilitate the process.

#### FOR POLYPS & FOREIGN BODIES

**STABLE FRAMED NET** 

**DIRECT ROTATABILITY\*** 

#### **OCTOPUS** FOREIGN BODY RETRIEVAL NET

Product code	Material Code	Characteristics	Volume (cm³)	Length (mm)	Ø Working channel (mm)	Unit
FOR1-N1-20-180	70100146681	Hexagonal, framed, paediatric	3	1,800	2.0	5
FOR1-N1-30-220	70100146563	Oval, framed	6	2,200	2.8	5
FOR1-N1-40-220	70100146682	Reservoir function, framed	8	2,200	2.8	5
FOR1-N2-30-220	70100146654	Kite, framed	6	2,200	2.8	5
FOR1-N3-30-220	70100146607	Kite, framed, rotatable	6	2,200	2.8	5






FOR1-N1-40


FOR1-N2-30

2



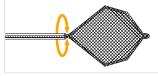

FOR1-N3-30

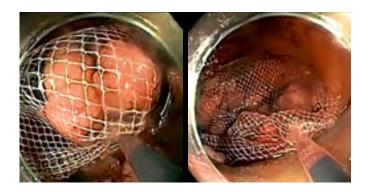

Owing to a new reservoir design OCTOPUS enables easy and reliable recovery of multiple objects simultaneously. Whereby 1:1 rotatability facilitates manoeuvring during foreign body recovery. The extremely sturdy net reliably maintains its position and does not slip.




### FOREIGN BODY FORCEPS

Product code	Material Code	Characteristics	Opening width (mm)	Ø Tube (mm)	Length (mm)	Ø Working channel (mm)	Unit
FOR1-A1-23-230	70100146637	Alligator, metal spring, coated	9	2.3	2,300	2.8	5
FOR1-A2-23-230	70100146653	Hybrid jaw configuration alligator/ rat tooth, metal spring, coated	9	2.3	2,300	2.8	10
FOR1-A2-23-260	70100146565	Hybrid jaw configuration alligator/ rat tooth,metal spring, coated balloon enteroscopy	8	2.3	2,300	2.8	5
FOR1-P1-18-230	70100146585	Pelican, coated, paediatric	9	1.8	2,300	2.8	5
FOR1-P1-23-230	70100146638	Pelican, coated	15.5	2.3	2,300	2.8	5
FOR1-H3-23-230	70100146566	Three-arm grasping forceps, PTFE catheter		2.3	2,300	2.8	5







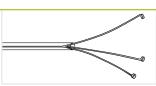

FOR1-A1






FOR1-H3

### FOREIGN BODY PROTECTION CAP

Product code	Material Code	Characteristics	Unit
WS0420000	70100146961	Foreign body protection cap	10

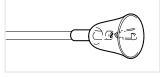


WS0420000

#### KOALA POLYP TRAPS

The two removable strainers of the koala polyp trap facilitate easy removal of tissue specimens and therefore protect against contamination with biomaterial. The suction function is limitless throughout the procedure.

Product code	Material Code	Characteristics	Unit
TRP1-S2	70100147772	2 integrated strainers, white	25
TRP1-S4	70100149546	2 integrated strainers, white 2 additional strainers, grey	25

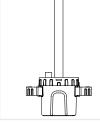



#### Minimised contamination risk

On account of the two chambers of the Koala it is no longer necessary to deposit a contaminated strainer outside the system. That reduces contamination with biomaterial, and hence the infection risk.

TRP1-S2 / S4

Strainer

#### POLYP TRAPS

Product code	Material Code	Characteristics	Unit
MIS1-Polyp trap	70100146707	4 chambers, connecting tubes to suction pump/endoscope	10
MIS1-Polyp trap50	70100146556	4 chambers, connecting tubes to suction pump/endoscope	50



 
 00146707
 to suction pump/endoscope
 10

 00146556
 4 chambers, connecting tubes to suction pump/endoscope
 50

TRP1-S2 / S4

Manufactured by Fujifilm medwork GmbH.





#### SPRAY CATHETER

Product code	Material Code	Characteristics	Length (mm)	Ø Working channel (mm)	Unit
SPE1-B6-18-120	70100146734	Spray catheter	1,200	2.0	10
SPE1-C6-18-220	70100146735	Spray catheter	2,200	2.0	10



SPE1-B6 / C6

#### **PURASTAT** HAEMOSTAT\*

#### THIRD PARTY PRODUCT

2

PuraStat is a slightly viscous solution of synthetic peptides. When in contact with blood, the synthetic peptide selfassembles into a 3-dimensional matrix which rapidly coats the bleeding point leading to haemostasis. It is transparent and can be reapplied. PuraStat can be combined with coagulation, clips etc and is applicable to narrow spaces. It is easy to handle and ready to use in pre-filled syringe.

Product code	Material Code	Characteristics	Unit
PURASTAT 1ml 621-034	70100152373	Haemostatic gel 1 ml syringe	1
PURASTAT 3ml 621-035	70100152372	Haemostatic gel 3 ml syringe	1
PURASTAT 5ml 621-036	70100152371	Haemostatic gel 5 ml syringe	1
PURASTAT Nozzle System Type-L	70100153471	Haemostatic gel catheter	5





### CYTOLOGY BRUSHES


Product code	Material Code	Characteristics	Ø Brush (mm)	Length (mm)	Ø Working channel (mm)	Unit
WS0372512	70100146960	Bronchoscopy, handle, 1 lumen, distal protection ball	2.5	1,200	2.0	5
WS0373016	70100146972	Gastroscopy, handle, 1 lumen, distal protection ball	3.0	1,600	2.8	5
WS0373022	70100146980	Colonoscopy, handle, 1 lumen, distal protection ball	3.0	2,200	2.8	5




WS03

### COLONOSCOPY EXAMINATION PANTS

Product code	Material Code	Characteristics	Unit
MIS1-Colo-Pants1	70100146618	Colonoscopy examination pants, dark blue, fleece, including elastic, slit, Bermuda (width 61cm/length 65cm)	100
MIS1-Colo-Pants	70100146617	Colonoscopy examination pants, dark blue, fleece, including elastic, slit with hook-and-loop cover, prolonged waistline, Bermuda (width 70 cm/length 60 cm)	100
MIS1-Colo-Pants-XXL	70100146555	Colonoscopy examination pants, dark blue, fleece, including elastic, slit with hook-and-loop cover, prolonged waistline, Bermuda (width 85cm/length 70cm)	10

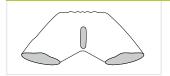

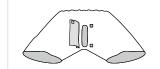

MIS1-Pants1

MIS1-Pants / Pants-XXL

2

# **FLAMINGO**

#### **CO-INVENTED BY PROF. DORMANN**



Inspired by nature, extremly versatile and effective: The FLAMINGO kit – a system to meet the therapeutic challenges of the buried bumper syndrome. A simple yet precise excision of the PEG's retaining plate, thanks to the unique cutting wire design, which allows multidimensional cutting.



"The FLAMINGO kit has been uniquely designed to meet the particularly difficult circumstances of buried bumper therapy. I use the Flamingo Kit because I gain confidence over other techniques used and at the same time can significantly reduce the examination time."


Co-Inventor Prof. Dr. med. Arno Dormann, 08.02.2022

#### MULTIDIMENSIONAL CUTTING FOR PRECISE EXCISION OF THE RETAINING PLATE

ACCURATE POSITIONING FACILITATES THE STAR-SHAPED INCISION OF THE MUCOSA


## FLAMINGO KIT

FLAMINGO KI	IT					2
Product code	Material Code	Characteristics	Cutting Length (mm)	Length (mm)	Accepted guide wire	Unit
		Flamingo kit consisting of:				
POL3-P1-40-30-30	70100140707	1 Flamingo for 15Fr PEG probe, 2 lumen, CWT, rotatable	6 / 10	300	.035"	1
	70100146767	1 Foreign body forceps, alligator/rat tooth		2,300		
		1 guide wire, hydrophilic tip		1,200		

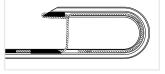


POL3-P1


#### **PUSH TECHNIQUE**

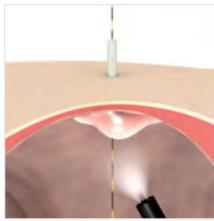

Exploration with the PEG probe using a guide wire

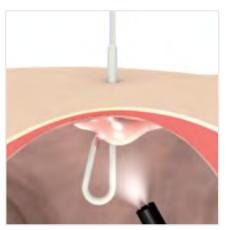

Cutting the PEG probe free with the FLAMINGO by making a star-shaped incision

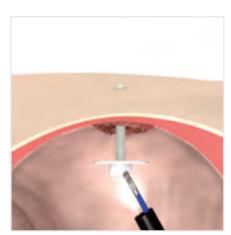

Removal of the PEG probe from the stomach with the foreign body forceps







# BIOPSY




ENDO BITE<sup>2</sup>


**Biopsy Forceps** 


#### **ENDOBITE**<sup>2</sup> BIOPSY FORCEPS

EndoBite2 with mono pull wire ensures large sizes of sample material ideal for histological assessment because the cut surfaces are completely closed, even if the position of the endoscope is challenging. The mono pull wire prevents the branches from tilting and thus protects the working channel against wear and damage.

Product code	Material Code	Characteristics	Ø Jaws (mm)	Length (mm)	Ø Working channel (mm)	Unit
BIO1-C4-18-120	70100146705	Oval jaws, coated, Bronchoscopy	1.8	1,200	2.0	10
BIO1-C4-18-180	70100146591	Oval jaws, coated, Paediatrics	1.8	1,800	2.0	10
BIO1-C4-18-260	70100146676	Oval jaws, coated, Balloon Enteroscopy	1.8	2,600	2.0	10
BIO1-C4-23-180	70100146647	Oval jaws, mono pull wire, coated	2.3	1,800	2.8	10
BIO1-C4-23-230	70100146662	Oval jaws, mono pull wire, coated	2.3	2,300	2.8	10
BIO1-D4-23-180	70100146574	Oval jaws, mono pull wire, coated, spike	2.3	1,800	2.8	10
BIO1-D4-23-230	70100146678	Oval jaws, mono pull wire, coated, spike	2.3	2,300	2.8	10
BIO1-C3-23-180	70100146645	Oval jaws, mono pull wire	2.3	1,800	2.8	10
BIO1-C3-23-230	70100146592	Oval jaws, mono pull wire	2.3	2,300	2.8	10
BIO1-D3-23-180	70100146552	Oval jaws, mono pull wire, spike	2.3	1,800	2.8	10
BIO1-D3-23-230	70100146593	Oval jaws, mono pull wire, spike	2.3	2,300	2.8	10
BIO1-G4-23-180	70100146575	Alligator, mono pull wire, coated	2.3	1,800	2.8	10
BIO1-G4-23-230	70100146557	Alligator, mono pull wire, coated	2.3	2,300	2.8	10
BIO1-H4-23-180	70100146679	Alligator, mono pull wire, coated, spike	2.3	1,800	2.8	10
BIO1-H4-23-230	70100146680	Alligator, mono pull wire, coated, spike	2.3	2,300	2.8	10

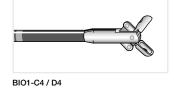

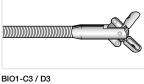

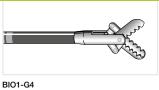



2

2

MACRO BIOPSY FORCEPS

Product code	Material Code	Characteristics	Ø Jaws (mm)	Length (mm)	Ø Working channel (mm)	Unit
AMHBFJ2.8*2300	70100146977	Alligator, mono pull wire, coated, macro forceps	2.8	2,300	3.2	10
AMHBFK2.8*2300	70100146955	Alligator, mono pull wire, coated, macro forceps, spike	2.8	2,300	3.2	10





AMHBFJ2

AMHBFK2





# FOREIGN BODY



OCTOPUS | FLAMINGO

Foreign Body Retrieval Nets	82
Foreign Body Forceps	84
Foreign Body Protection Cap	85
Buried Bumper Therapy	86



# OCTOPUS



# **HIGH VOLUME**

There is no escape: The OCTOPUS retrieval net takes on polyps and foreign bodies. The sturdy structure of the frame ensures reliable removal.

Versatility in use, it can be used for several foreign bodies – due to its reservoir function. Direct rotatability and a high degree of flexibility facilitate the process.

## FOR POLYPS & FOREIGN BODIES

**STABLE FRAMED NET** 

**DIRECT ROTATABILITY\*** 

#### **OCTOPUS** FOREIGN BODY RETRIEVAL NET

Product code	Material Code	Characteristics	Volume (cm³)	Length (mm)	Ø Working channel (mm)	Unit
FOR1-N1-20-180	70100146681	Hexagonal, framed, paediatric	3	1,800	2.0	5
FOR1-N1-30-220	70100146563	Oval, framed	6	2,200	2.8	5
FOR1-N1-40-220	70100146682	Reservoir function, framed	8	2,200	2.8	5
FOR1-N2-30-220	70100146654	Kite, framed	6	2,200	2.8	5
FOR1-N3-30-220	70100146607	Kite, framed, rotatable	6	2,200	2.8	5






FOR1-N1-40


FOR1-N2-30

2



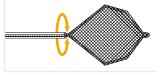

FOR1-N3-30

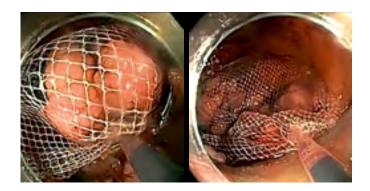

Owing to a new reservoir design OCTOPUS enables easy and reliable recovery of multiple objects simultaneously. Whereby 1:1 rotatability facilitates manoeuvring during foreign body recovery. The extremely sturdy net reliably maintains its position and does not slip.




## FOREIGN BODY FORCEPS

Product code	Material Code	Characteristics	Opening width (mm)	Ø Tube (mm)	Length (mm)	Ø Working channel (mm)	Unit
FOR1-A1-23-230	70100146637	Alligator, metal spring, coated	9	2.3	2,300	2.8	5
FOR1-A2-23-230	70100146653	Hybrid jaw configuration alligator/ rat tooth, metal spring, coated	9	2.3	2,300	2.8	10
FOR1-A2-23-260	70100146565	Hybrid jaw configuration alligator/ rat tooth, metal spring, coated balloon rnteroscopy	8	2.3	2,300	2.8	5
FOR1-P1-18-230	70100146585	Pelican, coated, paediatrics	9	1.8	2,300	2.8	5
FOR1-P1-23-230	70100146638	Pelican, coated	15.5	2.3	2,300	2.8	5
FOR1-H3-23-230	70100146566	Three-arm grasping forceps, PTFE catheter		2.3	2,300	2.8	5







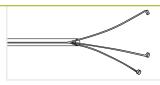

2

FOR1-A1

FOR1-A2

2

FOR1-P1

FOR1-H3

## FOREIGN BODY PROTECTION CAP

Product code	Material Code	Characteristics	Unit
WS0420000	70100146961	Foreign body protection cap	10



WS0420000

# **FLAMINGO**

#### **CO-INVENTED BY PROF. DORMANN**



Inspired by nature, extremly versatile and effective: The FLAMINGO kit – a system to meet the therapeutic challenges of the buried bumper syndrome. A simple yet precise excision of the PEG's retaining plate, thanks to the unique cutting wire design, which allows multidimensional cutting.



"The FLAMINGO kit has been uniquely designed to meet the particularly difficult circumstances of buried bumper therapy. I use the Flamingo Kit because I gain confidence over other techniques used and at the same time can significantly reduce the examination time."


Co-Inventor Prof. Dr. med. Arno Dormann, 08.02.2022

#### MULTIDIMENSIONAL CUTTING FOR PRECISE EXCISION OF THE RETAINING PLATE

ACCURATE POSITIONING FACILITATES THE STAR-SHAPED INCISION OF THE MUCOSA


## FLAMINGO KIT

POL3-P1-40-30-30       70100146767         Flamingo kit consisting of:       1 Flamingo for 15Fr PEG probe, 6 / 10       300       .035"       1         2 lumen, CWT, rotatable       2 lumen, CWT, rotatable       2,300       1         1 Foreign body forceps, alligator/rat tooth       1 guide wire, hydrophilic tip       1,200	Product code	Material Code	Characteristics	Cutting Length (mm)	Length (mm)	Accepted guide wire	Unit
POL3-P1-40-30-30     70100146767     2 lumen, CWT, rotatable       1 Foreign body forceps, alligator/rat tooth     2,300	POL3-P1-40-30-30		Flamingo kit consisting of:				
1 Foreign body forceps, 2,300 alligator/rat tooth		70100146767		6 / 10	300	.035"	1
1 guide wire, hydrophilic tip 1,200		10100140707			2,300		
			1 guide wire, hydrophilic tip		1,200		

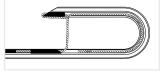


POL3-P1


#### **PUSH TECHNIQUE**

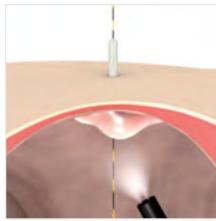

Exploration with the PEG probe using a guide wire

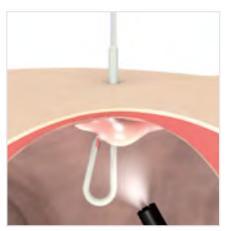

Cutting the PEG probe free with the FLAMINGO by making a star-shaped incision

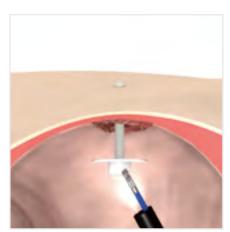


Removal of the PEG probe from the stomach with the foreign body forceps







# HAEMOSTASIS & LIGATION

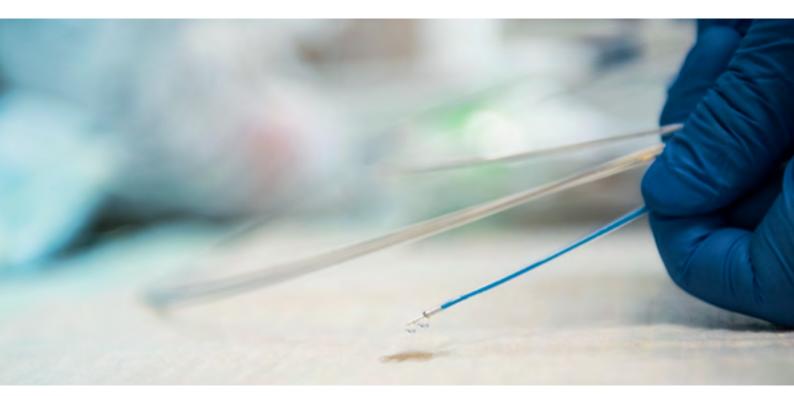

MANTA | ICUS<sup>2</sup> | PURASTAT

Injection Needles	90
Haemostasis Clip	92
Haemostat	92



#### MANTA INJECTION NEEDLES

Manta ensures reliable retraction and extension of the needle, even in sharply angulated device positions, and prevents feared damage to the working channel. Maximum kink-resistance and single handed operation ease the application.

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A3-05-5-18-240	70100146666	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	2,400	2.0	5
INJ1-A3-07-5-18-240	70100146657	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,400	2.0	5



INJ-A3

## **ICUS** INJECTION NEEDLES

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A1-05-5-18-180	70100146672	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	1,800	2.0	10
INJ1-A1-07-5-18-180	70100146611	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	1,800	2.0	10



INJ1-A1



2

#### **ICUS<sup>2</sup>** INJECTION NEEDLES

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A2-05-3-23-230	70100147786	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   3	2,300	2.8	10
INJ1-A2-05-4-23-180	70100146693	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   4	1,800	2.8	10
INJ1-A2-05-4-23-230	70100146549	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   4	2,300	2.8	10
INJ1-A2-07-3-23-230	70100147814	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   3	2,300	2.8	10
INJ1-A2-07-4-23-180	70100146614	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   4	1,800	2.8	10
INJ1-A2-07-4-23-230	70100146671	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   4	2,300	2.8	10
INJ1-A2-07-5-23-180	70100146665	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	1,800	2.8	10
INJ1-A2-07-5-23-230	70100146615	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,300	2.8	10
INJ1-A2-09-5-23-180	70100146554	Latching mechanism, distal needle guide, suitable with Histoacryl	0.9   5	1,800	2.8	10
INJ1-A2-09-5-23-230	70100146598	Latching mechanism, distal needle guide, suitable with Histoacryl	0.9   5	2,300	2.8	10



#### Figure: INJ1-A2-05-3-23-230

- 20% reduction\* in friction for precision puncture
- 80% less effort\* when administering injections
- 20% easier\* piercing



#### Proven reliable needle guidance

No matter whether it is a case of injection-supported polypectomy or haemostasis, permits precision injection thanks to easier piercing behaviour and smoother functioning. In future, have confidence in proven reliable needle guidance and thereby protect your working channel.



#### HAEMOSTASIS CLIP

The haemostasis clip offers simple and safe haemostasis. With dedicated, 90°-angled hook design as well as pointed and overlapping jaws, effective tissue traction is ensured. Compact clip design helps to maintain tissue defect access during multiple clip placements.

Product code	Material Code	Characteristics	Opening width (mm)	Length (mm)	Ø Working channel (mm)	Unit
SPE1-X9-26-230	70100151102	Coated, 360° positioning	12	2,100	2.8	10

SPE1-X9-26-230

#### **PURASTAT** HAEMOSTAT\*

#### THIRD PARTY PRODUCT

2

PuraStat is a slightly viscous solution of synthetic peptides. When in contact with blood, the synthetic peptide selfassembles into a 3-dimensional matrix which rapidly coats the bleeding point leading to haemostasis. It is transparent and can be reapplied. PuraStat can be combined with coagulation, clips etc and is applicable to narrow spaces. It is easy to handle and ready to use in pre-filled syringe.

Product code	Material Code	Characteristics	Unit	
PURASTAT 1ml 621-034	70100152373	Haemostatic gel 1 ml syringe	1	
PURASTAT 3ml 621-035	70100152372	Haemostatic gel 3 ml syringe	1	
PURASTAT 5ml 621-036	70100152371	Haemostatic gel 5 ml syringe	1	
PURASTAT Nozzle System Type-L	70100153471	Haemostatic gel catheter	5	* PuraStat is a trademark for 3-D Ma





# DILATION



Dilation Balloons	94
Insufflation Device	94



Product code	Material Code	Characteristics	Ø Balloon (mm)	Length Balloon (mm)	Working length (mm)	Accepted guide wire	Ø Working channel (mm)	Unit
DIL1-A1-06-40-23-35	70100146636	Tapered catheter (2.3/1.8 mm)	6	40	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-08-40-23-35	70100146669	Tapered catheter (2.3/1.8 mm)	8	40	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-10-40-23-35	70100146663	Tapered catheter (2.3/1.8 mm)	10	40	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-12-60-23-35	70100146605		12	60	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-14-60-23-35	70100146664		14	60	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-16-60-23-35	70100146670		16	60	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-18-60-23-35	70100146667		18	60	2,,300	.035" ≥ 2,000 mm	2.8	2
DIL1-A1-20-60-23-35	70100146606		20	60	2,,300	.035" ≥ 2,000 mm	3.2	2

2

8


# **DILATION BALLOONS for Short & Long Wire Technique**



DIL1-A1

#### INSUFFLATION DEVICE

Product code	Material Code	Characteristics	Unit
DIL1-B1-60	70100146584	60 ml volume, 12 atm maximum pressure	1

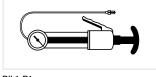

DIL1-B1



# **SEMS**



GPS WIRE

**Guide Wires** 

# **GPS WIRE**



# **ARRIVE SAFELY**

The atraumatic tip of the GI Probing Scout ensures conservative exploration. The sturdy sheath reliably guides therapy instruments to their destination. A smooth transition between the hydrophilic tip and the coating enhances the precision control of your instruments.

In addition, handling is facilitated by the newly designed insertion aid.

## **PRECISE EXPLORATION**

STABLE INSTRUMENT GUIDANCE

SMOOTH TRANSITION BETWEEN TIP AND COATING

# GPS WIRE GUIDE WIRES with Hydrophilic Tip

Product code	Material Code	Characteristics	Diameter	Length (mm)	Unit
WIR1-S6-35-260	70100146742	Straight, Nitinol, insulated, green/white	.035"	2,600	5
WIR1-S6-35-450	70100146743	Straight, Nitinol, insulated, green/white	.035"	4,500	5
WIR1-J6-35-260	70100147764	J-tip, Nitinol, insulated, green/white	.035"	2,600	5
WIR1-J6-35-450	70100146828	J-tip, Nitinol, insulated, green/white	.035"	4,500	5





## **GUIDE WIRES**

Product code	Material Code	Characteristics	Diameter	Length (mm)	Unit
WIR1-F7-35-260	70100146812	Straight, Nitinol, coated, extra stiff	.035"	2,600	2
WIR1-F7-35-400	70100146827	Straight, Nitinol, coated, extra stiff	.035"	4,000	2



WIR1-F7

8



# **BALLOON ENTEROSCOPY**



AXS\_TOME | MANTA | ICUS | ENDOBITE<sup>2</sup>


Sphincterotomes	100
ERCP-Cannula	100
Stone Extraction Baskets	101
Injection Needles	101
Biopsy Forceps	102
Polypectomy Snares	102
Guide Wire	102




# AXS\_TOME SPHINCTEROTOMES 2 Lumen

Product code	Material Code	Characteristics	Cutting length (mm)	Accepted guide wire	Length (mm)	Ø Working channel (mm)	Unit
PAP1-D2-30-21-OL	70100146601	Micro-Tip, CWT	30	.021"	2,600	2.0	2
PAP1-N2-20-21-OL	70100146696	B II, CWT, balloon enteroscopy	20	.021"	2,600	2.0	2



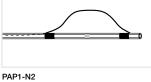

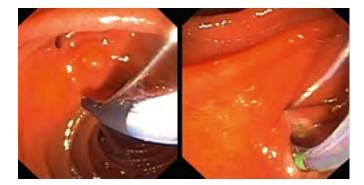

#### Anatomical 3-D orientation

Based on systematic 11 o'clock orientation the AXS\_tome sphincterotomes also follow the left frontal anatomy of the distal choledochus.

2

8

#### **ERCP-CANNULA**

Product code	Material Code	Characteristics	Length (mm)	Accepted guide wire	Ø Working channel (mm)	Unit
CAN1-B3-18-260-35	70100146580	Filiform, 1 lumen, radiopaque marking	2,600	.035"	2.0	2



CAN-B3

#### STONE EXTRACTION BASKETS

Product code	Material Code	Characteristics	Ø Basket (mm)	Length (mm)	Ø Working channel (mm)	Unit
BAS1-C1-20-18-260	70100146573	Half twisted 😽	20	2,600	2.0	2
BAS1-C2-25-23-260	70100146642	Half twisted 😽	25	2,600	2.8	2



#### MANTA INJECTION NEEDLES

Manta ensures reliable retraction and extension of the needle, even in sharply angulated device positions, and prevents feared damage to the working channel. Maximum kink-resistance and single handed operation ease the application.

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A3-05-5-18-240	70100146666	Latching mechanism, distal needle guide, suitable with Histoacryl	0.5   5	2,400	2.0	5
INJ1-A3-07-5-18-240	70100146657	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,400	2.0	5

INJ-A3

#### **ICUS** INJECTION NEEDLES

Product code	Material Code	Characteristics	Ø Needle   Length (mm)	Length (mm)	Ø Working channel (mm)	Unit
INJ1-A1-07-5-18-260	70100146611	Latching mechanism, distal needle guide, suitable with Histoacryl	0.7   5	2,600	2.0	10




2



#### **ENDOBITE**<sup>2</sup> BIOPSY FORCEPS

Product code	Material Code	Characteristics	Ø Jaws (mm)	Length (mm)	Ø Working channel (mm)	Unit
BIO1-C4-18-260	70100146676	Oval jaws, coated	1.8	2,600	2.0	10

2

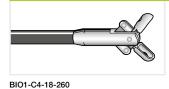

#### POLYPECTOMY SNARES

Product code	Material Code	Characteristics	Ø Snare (mm)	Length (mm)	Ø Working channel (mm)	Unit
POL1-B1-10-23-260-OL	70100147822	Convex	10	2,600	2.8	10
POL1-B1-15-23-260-OL	70100146685	Convex	15	2,600	2.8	10
POL1-B1-20-23-260-OL	70100147810	Convex	20	2,600	2.8	10
POL1-B1-30-23-260-OL	70100146712	Convex	30	2,600	2.8	10



POL1-B1

# GUIDE WIRE Product code Material Code Characteristics Diameter Length (mm) Unit WIR1-F1-21-500 70100147850 Straight, Nitinol, coated .021" 5,000 2

WIR1-F1



# ACCESSORIES



KOALA | RACOON BRUSH | SHEPHERD | FROG | JAZZ

Polyp Traps	104
Cleaning Brushes	105
Endoscope Protector	107
Lavage Catheters	107
Biopsy Valves	108
Spray Catheters	109
Endoscope Valve Kit	110
Miscellaneous	112



#### **KOALA** POLYP TRAPS

The two removable strainers of the koala polyp trap facilitate easy removal of tissue specimens and therefore protect against contamination with biomaterial. The suction function is limitless throughout the procedure.

Product code	Material Code	Characteristics	Unit
TRP1-S2	70100147772	2 integrated strainers, white	25
TRP1-S4	70100149546	2 integrated strainers, white 2 additional strainers, grey	25





#### **Minimised contamination risk**

On account of the two chambers of the Koala it is no longer necessary to deposit a contaminated strainer outside the system. That reduces contamination with biomaterial, and hence the infection risk.

TRP1-S2 / S4

Strainer

#### POLYP TRAPS


Product code	Material Code	Characteristics	Unit
MIS1-Polyp trap	70100146707	4 chambers, connecting tubes to suction pump/endoscope	10
MIS1-Polyp trap50	70100146556	4 chambers, connecting tubes to suction pump/endoscope	50




TRP1-S2 / S4

## RACOON BRUSH CLEANING BRUSHES

Product code	Material Code	Characteristics	Ø Brush (mm)	Length (mm)	Ø Working channel (mm)	Unit
CLE1-G1-50-18-180	70100146651	For gastroscope, 1 brush	5	1,800	min. 2.0	50
CLE1-G2-50-18-180	70100146644	For gastroscope, 2 brushes	5	1,800	min. 2.0	50
CLE1-C1-50-18-230	70100146548	For colonoscope, 1 brush	5	2,300	min. 2.0	50
CLE1-C1-65-18-230	70100146691	For colonoscope, 1 brush	6	2,300	min. 2.0	50
CLE1-C2-50-18-230	70100146582	For colonoscope, 2 brushes	5	2,300	min. 2.0	50
CLE1-C3-50-18-230	70100146595	For colonoscope, 3 brushes	5	2,300	min. 2.0	50
CLE1-V2-012	70100147811	For endoscope valve intake	11/5	120		10
CLE1-V2-012-50	70100147826	For endoscope valve intake	11/5	120		50
CLE1-C4-18-230	70100146583	For colonoscope, 1 brush, distal protection ball	5	2,300	2.2-4.2	50
CLE1-C6-18-230	70100146706	For colonoscope, 2 brushes, distal protection ball	5	2,300	2.2-4.2	50
CLE1-D2-18-230	70100146692	For colonoscope and endoscope valve intake, 2 brushes, distal protection ball	10/5	2,300	2.2-4.2	50

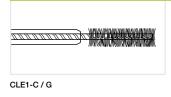

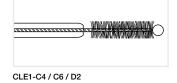


## CLEANING BRUSHES for Gastroscopes & Colonoscopes of 700 Series

Product code	Material Code	Where to use	Characteristics
WB7024FW	16518827	Working channel	Reusable
WB7025DC	16560832	Working channel	2
WB11003FW	16518841	Cylinder port	Reusable
WB11003DV	16560820	Cylinder port	2



WB7024FW

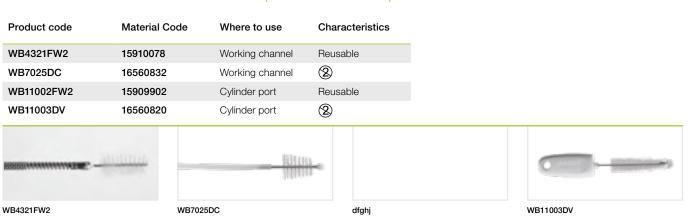

#### CLEANING BRUSHES for Gastroscopes & Colonoscopes of 600 / 580 Series

#### CLEANING BRUSHES for Double-Balloon Endoscopes

Product code	Material Code	Characteristics	Where to use	Suitable for
WB4324FW2	16406921	Reusable	Working channel	EN-580T/XP
WB7025DC	16560832	2	Working channel	EN-580T/XP
WB11002FW2	15909902	Reusable	Cylinder port	EN-580T/XP
WB11003DV	16560820	2	Cylinder port	EN-580T/XP
WB7024FW	16518827	Reusable	Working channel	EI-580BT
WB11003FW	16518841	Reusable	Cylinder port	EI-580BT
WB11003DV	16560820	2	Cylinder port	EI-580BT

2



#### CLEANING BRUSHES Colonocope Kit

Product code	Material Code	Characteristics	Ø Working channel (mm)	Unit
CLE1-KIT2-COLO	70100147806	Colonoscope kit consisting of: 1 CLE1-C6-18-230 For colonoscope, 2 brushes, distal protection ball 1 CLE1-V2-016 For endoscope valve intake	2.2-4.2	50



## **SHEPHERD** ENDOSCOPE PROTECTOR

Product code	Material Code	Characteristics	Ø Inner (mm)	Ø Outer (mm)	Length (mm)	Unit
SHP1-U1	70100147808	Endoscope protector in red, white, blue, green	4–10	37	140	50
medw@rk						
SPE1-B						

# LAVAGE CATHETER

Product code	Material Code	Characteristics	Length (mm)	Ø Working channel (mm)	Unit
SPE1-B5-18-120	70100146790	LL-connector	1,200	2.0	5
SPE1-B5-21-120	70100146822	LL-connector	1,200	2.8	5

SHP1-U1

2

8



# FROG



# **PROTECTS AND SEALS TIGHT**

Grips as tight as a frog's suction pad: the novel FROG Biopsy Valve seals especially tight, yet remains easy to open and close. Less friction allows quicker and less complicated insertion of instruments. Supplied in a practical, space saving dispenser with mounting option.



OPTIMISED TIGHTNESS LESS FRICTION EASY TO OPEN AND CLOSE

## FROG BIOPSY VALVES

Product code	Material Code	Characteristics	Unit
VAL1-F1-100	70100146824	For Fujifilm and Olympus endoscopes	100
VAL1-P1-100	70100146810	For Pentax endoscopes	100



VAL1-F1

# SPRAY CATHETER

Product code	Material Code	Characteristics	Length (mm)	Ø Working channel (mm)	Unit
SPE1-B6-18-120	70100146734	Spray catheter	1,200	2.0	10
SPE1-C6-18-220	70100146735	Spray catheter	2,200	2.0	10



SPE1-B6 / C6

8

# JAZZ VALVE KIT



# **PLAYFULLY EASY**

When speed is essential you want the complete endoscope valve kit at hand in an instant. One packaging unit contains air/water valve, suction valve and biopsy valve.

#### PROTECTION AGAINST INFECTION AND CROSS CONTAMINATION

NO RISKS INDUCED BY CLEANING, DRYING AND STORAGE

FAST AND RELIABLE PATIENT DOCUMENTATION DUE TO ADDITIONAL PRODUCT LABEL

PERFECT VALVE OPERATION EVERY TIME

**REDUCING TIME AND EFFORT** 



## JAZZ ENDOSCOPE VALVE KIT for 700 Series

Product code	Material Code	Characteristics	Compatible endoscope	Unit
VAL1-G7-50	70100146737	Disposable endoscope valve kit, consisting of air/water valve, suction valve and biopsy valve FROG	Fujifilm 700 series with G7 grip	50

VAL1-G7



#### Traceability

Fast and reliable patient documentation due to additional self-adhesive label with product data.

## FORCEPS VALVES

Product code	Material Code	Characteristics	Compatible endoscope	Unit
FV-001	16632659	Forceps valve	Fujifilm endoscopes with G5 / G7 grip	10
FV-002	16663165	Forceps valve	Fujifilm EUS endoscopes	20



FV-002

2



## **BITE BLOCKS**

Ergonomic controls, perfect fit, maximum protection.

Product code Mat	terial Code	Characteristics	Ø Inner (mm)	Unit
BYT1-A3-20 701	100146578	Standard, white, with OEKO TEX® strap 🛞	20	100
AN10018004 701	100147834	With strap, latex-free	13	25
30148 701	100147740	With oxygen connection, reusable		5



## PRODUCT CLIP

Product code	Material Code	Characteristics	Unit
MIS2-PRO-CLIP	70100147813	Reusable product clip for fixation and reprocessing of flexible accessories, autoclavable up to 134° C, opening width 0-10 mm	20



MIS2-Pro-Clip

## **B1** DROPLET REDUCTION MOUTHPIECE

Product code	Material Code	Characteristics	Drape size (mm)	Compatible endoscopes	Unit
B1 Droplet reduction mouthpiece with drape	16673263	Mouthpiece with drape that functions as a face shield to reduce droplets from the gap between the mouthpiece and the corners of mouth With sponge rubber to reduce droplets from the oral cavity Latex-free silicone rubber band to fix the drape	300 x 300	Endoscopes or intubation tubes of the diameter from 3 to 16 mm	10



B1 Droplet reduction mouthpiece



# **MONITORS & CARTS**



EIZO™

EIZO Monitors	114
Endoscopy Carts	115



Monitors for clear and natural images: Fujifilm's medical equipment incorporates advanced video technology. High-performance monitors used in conjunction with Fujifilm endoscopes reproduce sharp, genuine images.

## **EIZO** MONITORS\*

#### THIRD PARTY PRODUCT

	Model	Description	Input signal	Output signal	Dimensions WxHxD (mm)	Weight (kg)
	EIZO CURATOR (UHD) EX3241 32" LCD with 4K resolution	LED backlight with high brightness of max. 700 cd/m <sup>2</sup> , high-resolution of 3840 x 2160 px (4K), optical bonding for reduced reflections, Smart Resolution Sparse Coding (S.R.S.C.) for high-definition images and enhanced sharpness	Display Port (HDCP 1.3), HDMI (HDCP 2.2/1.4), BNC (12G-SDI), BNC (3G-SDI), DVI-D x 2 (HDCP 1.4)	BNC (12G-SDI), BNC (3G-SDI), DVI-D	760 × 463 × 87	12.8
NEW	EIZO CURATOR EX3220 32" LCD	LED backlight with high brightness of max. 650 cd/m <sup>2</sup> , high-definition, full multi-modality, degree of protection IP45 (front), IP32 (rear)	BNC (3G-SDI) x 2, DVI-D x 2 (HDCP 1.4), D-Sub 15 pin (mini), BNC (RGB C-Sync or Component), S-Video, BNC (Composite)	BNC (3G-SDI) x 2, DVI-D, BNC (RGB C-Sync or Component), S-Video, BNC (Composite)	760 × 463 × 99	9.9
NEW	EIZO CURATOR EX2721 27" Color LCD	LED backlight with high brightness of max. 900 cd/m², high-definition, full multi-modality	BNC (3G-SDI), DVI-D x 2 (HDCP 1.4), D-Sub 15 pin (mini), S-Video, BNC (Composite)	BNC (3G-SDI), DVI-D, S-Video, BNC (Composite)	660 x 400 x 87	8.5
	EIZO Colour LCD 26" HD-Type LCD	LED backlight with high brightness of max. 700 cd/m <sup>2</sup> , high-definition, full multi-modality, displays up to three video sources simultaneously	BNC (3G-SDI) x 2, DVI-D x 2 (HDCP 1.4), D-Sub 15 pin (mini), BNC (RGB C-Sync or Component), S-Video, BNC (Composite)	BNC (3G-SDI) x 2 (Composite), DVI-D, BNC (RGB C-Sync or Component), S-Video, BNC (Composite)	643 x 396 x 83	8.4



#### **EIZO CURATOR EX3241: 4K RESOLUTION IN COMBINATION WITH** THE ELUXEO<sup>™</sup> SYSTEM

Monitors might not be available in all countries. Please check with your local distributor. \* Manufactured by EIZO Corporation, Japan.

**Strong and reliable carts:** The stable and flexible frames in different widths and heights empower enhanced procedural workflow. Used as single equipment tower, they ensure optimized space amount in surgical suites and offer maximum storage capacity.

# ENDOSCOPY CARTS\*\*

THIRD PARTY PRODUCT

Model	Material Code	Dimensions (WxHxD mm)	Weight (kg)	Characteristics
ITD-03 EUS Iso Plus	70100128566	618 x 1500 x 624	approx. 85	EUS cart 2× endoscope hanger, 4× shelves 2,000 VA, isolating transformer with insulation monitoring function
ITD-03 Iso Plus	70100113037	540 x 1510 x 580	approx. 40	<b>Uni-cart sonoprobe</b> 3× shelves, 1× monitor stand 660 VA, isolating transformer
ITD-00 Plain Cart	70100113035	614 x 1277 x 625	approx. 70	Classic cart 1× endoscope hanger, 3× shelves





ITD-03 EUS Iso Plus

ITD-03 Iso Plus





# SERVICE



Service & Partnership	118
Training & Education	120
Product Recommendations	122

# OUR COMMITMENT TO SERVICE



# THINK GLOBALLY – ACT LOCALLY

Our service strategy aims for the highest customer satisfaction by offering a comprehensive service and being close to the customers. Eight service centers with the headquarters in Willich (Germany) are spread throughout Europe and employ highly qualified in-house technicians and experts in the field, allowing faster and better coverage for all customer needs.

#### Our service network includes:

Willich (GER) Montigny, Toulouse (FRA) Barcelona, Porto (IBERIA) Milano (ITA) Prague (CZE) Istanbul (TUR)

#### Our full comprehensive service contract covers:

In-house repair service All repair costs Highly qualified field service engineers Large variety of loan devices Maintenance service and damage prevention Support for reprocessing and on-site consulting

#### **POWERED BY PARTNERSHIP**

Fujifilm, a pioneer in the field of diagnostic imaging and information systems for medical institutions, operates in about 50 group companies in Europe and employs over 4,500 people engaged in R&D, manufacturing, sales and service. Dialogue and continuous partnership have a special significance for us and at our locations. Our products and technologies are constantly being developed in agreement with you to meet your specific needs. Your contact partners are available for you – no matter where you are. Living this kind of partnership inspires us to do all we can to make the world a little better.



# OUR TRAINING OFFERS



# **ANYTIME – ANYWHERE**

One of the biggest challenges in endoscopy is the provision of training and education for healthcare professionals. To benefit endoscopy services, enhanced onsite training opportunities and greater hands-onsupport from manufacturers is key. Above all, there is a need for flexibility from the side of the manufacturers: Delivering training and education on site is of great advantage for the clinicians.

Having heard the challenges faced by endoscopists, Fujifilm has developed a series of training and education courses. The latest development is the mobile service unit first presented in 2022 – the EndoRunner.

EndoRunner

The 'EndoRunner' is a mobile hub containing all the latest innovations in endoscopy from Fujifilm. The EndoRunner travels to hospitals and conferences across Europe to offer training and education opportunities to clinicians. It includes around twenty different flexible endoscopes and more than fifty different therapeutic devices for all kinds of applications.

To request a visit from the EndoRunner, get in touch at **endorunner\_feg@fujifilm.com** 

Further information and details about the EndoRunner tour can be found at **www.fujifilm-endorunner.com** 





**MOBILE HUB** 



#### **LEARNING PLATFORM**

SeeLearn is Fujifilm's new online learning platform providing information on Fujifilm endoscopy technologies, sharing patient cases and studies, presenting webinars and learning tools to support education for professionals anytime and anywhere. Please visit

#### seelearn-emea.fujifilm.com/





#### **ON-SITE TRAINING**

Fujifilm offers hands-on trainings on a regular basis. If you are interested to join please contact for further information **endoscopy\_eu@fujifilm.com** 

endoscopy\_eu@rujimm.cor





# PRODUCT RECOMMENDATIONS

Recommended endoscopes for different gastrointestinal segments	Diseases	Special endoscopes to cope with these diseases	Special features of the special endoscope	Endoscopes for further diagnosis
Oesophagus				
EG-760R	Zenker diverticle	EG-760CT; EI-740D/S	WCH* 3.8; dual channel	
EG-760Z	Other oesophagus diverticle	EG-760CT; EI-740D/S	WCH 3.8; dual channel	
EG-760CT EG-740N EI-740D/S	Barrett oesophagus	EG-760CT; EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
EG-720R EG-600WR EG-580NW2 EG-580UR	Oesophagitis	EG-760CT; EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	
	Mallory Weiss syndrome	EG-760CT	WCH 3.8	
EG-5800R EG-580UT	Oesophagus varices	2 endoscopes prepared		
EG-530NP	Tumors	EG-760CT; EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
	Squamous cell carcinoma	EG-760CT; EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
	Achalasia/POEM	EG-760R; EG-760CT	WCH 3.8	
	Stenosis	EG-740N; EG-580NW2; EG-530NP	Small outer diameter	EG-580UT/UR
Gastro intestinal				
EG-760R EG-760Z	Gastritis	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	
EG-760CT EG-740N	Dyspepsia	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	
EI-740D/S EG-720R	Ulcus ventriculi	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
EG-600WR EG-580NW2	Ulcus perforation	EG-760CT; EI-740D/S	WCH 3.8; dual channel	
EG-530NP	Ulcus carcinomas	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
	Ulcus bleeding	EG-760CT; EI-740D/S	WCH 3.8; dual channel	
	Gastro carcinomas	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
	Precancerosis	EG-760Z; EG-760R; EG-720R; EG-600WR	Magnification, high-quality images	EG-580UT/UR
	Stomach exit stenosis	EG-740N; EG-580NW2; EG-530NP	Small outer diameter	
	Vessel abberation	EI-740D/S	WCH 3.8; dual channel	EG-580UT/UR
	Fundus varices	EG-760R; EG-760CT	WCH 3.8	
Duodenum				
EG-760R EG-760Z	Duodenitis	EG-760Z; EG-760R; EG-720R; EG-600WR; (EI-580BT)	Magnification, high-quality images, stabilises position	
EG-760CT EG-740N	Duodenal ulcer	EG-760Z; EG-760R; EG-720R; EG-600WR; (EI-580BT)	Magnification, high-quality images, stabilises position	
EI-740D/S EG-720R EG-600WB	Coeliac disease	EG-760Z; EG-760R; EG-720R; EG-600WR; (EI-580BT)	Magnification, high-quality images, stabilises position	
EG-600WR EI-580BT EG-580UT/UR	Bleeding	EG-760CT; (EI-580BT); EI-740D/S	WCH 3.2; WCH 3.8; dual channel, stabilises position	
EG-530NP	Tumors	EG-760Z; EG-760R; EG-720R; EG-760CT; EG-600WR; (EI-580BT)	Magnification, high-quality images, stabilises position	EG-580UT/UR

Recommended endoscopes for different gastrointestinal segments	Diseases	Special endoscopes to cope with these diseases	Special features of the special endoscope	Endoscopes for further diagnosis
Small Intestine				
EN-580T EN-580XP	Tumors of the small intestine	EN-580T	Bigger working channel	
	Erosive and ulcerated defects	EN-580XP	Small outer diameter	
	Bleeding	EN-580T	Bigger working channel	
	Vessel anomaly	EN-580T	Bigger working channel	
Biliary Tract and Pancre	as			
EN-580T EN-580XP EI-580BT EG-580UT/UR ED-580XT	Bile duct stones	EI-580BT; ED-580XT	Stabilises position	EG-580UT/UR
	Cholelithiasis	EI-580BT; ED-580XT	Stabilises position	
	Postoperative alterations	EI-580BT; ED-580XT	Stabilises position	
	Malignant stenosis	EI-580BT; ED-580XT	Stabilises position	EG-580UT/UR
	Tumors of the papilla	EG-760Z; EG-760R; EG-720R; EG-600WR; EI-580BT; ED-580XT	Magnification, high-quality images, stabilises position	
	Environmental tumors	EG-760Z; EG-760R; EG-720R; EG-600WR; EI-580BT	Magnification, high-quality images, stabilises position	EG-580UT/UR
	Infections	EG-760Z; EG-760R; EG-720R; EG-600WR; EI-580BT	Magnification, high-quality images, stabilises position	
Colon				
EC-760ZP-VM/VL EC-760R-VM/VI/VL EC-760P-VM/VL EC-740TM/TL EC-720RM/RI/RL EC-600WM/WI/WL EN-580T EN-580XP EG-580UT/UR EC-530DL	Colourectal polyps	EC-760ZP-VM/VL; EC-760R-VM/VI/VL; EC-760P-VM/VL; EC-740TM/TL; EC-720RM/RI/RL; G-EYE 760R; EC-600WM/WI/WL	Magnification, high-quality images, Smart Bend	
	Flat adenomas	EC-760ZP-VM/VL; EC-760P-VM/VL; G-EYE 760R	Magnification, high-quality images	
	Malignant tumors	EC-760ZP-VM/VL; EC-760P-VM/VL; G-EYE 760R	Magnification, high-quality images	EG-580UT/UR
	Intestinal inflammation	EC-760ZP-VM/VL; EC-760R-VM/VI/VL; EC-760P-VM/VL; EC-720RM/RI/RL; G-EYE 760R; EC-600WM/WI/WL	Magnification, high-quality images	
	Irritable bowel syndrome	EC-760ZP-VM/VL; EC-760R-VM/VL/L; EC-760P-VM/VL; EC-720RM/RI/RL; G-EYE 760R; EC-600WM/WI/WL	Magnification, high-quality images	
	Ulcerative colitis	EC-760ZP-VM/VL; EC-760R-VM/VI/VL; EC-760P-VM/VL; EC-720RM/RI/RL; G-EYE 760R; EC-600WM/WI/WL	Magnification, high-quality images	
	Crohn's disease	EC-760ZP-VM/VL; EC-760R-VM/VL/L; EC-760P-VM/VL; EC-720RM/RI/RL; G-EYE 760R; EC-600WM/WI/WL	Magnification, high-quality images	
	Hemorrhoids	2 endoscopes prepared		
	Anal diseases	EC-760P-VM/VL; EC-740TM/TL; G-EYE 760R	Smart Bend	

All endoscopes are compatible with the video processors ELUXEO<sup>™</sup> 7000 system and ELUXEO<sup>™</sup> Lite EP-6000. All endoscopic ultrasonography systems are compatible with processor SU1.





# FUJIFILM Europe GmbH

Balcke-Dürr-Allee 6 | 40882 Ratingen, Germany +49 2102 5364-0 | endoscopy\_eu@fujifilm.com www.fujifilm-endoscopy.com

