CERTIFIED COMPANY UNI EN ISO 9001 & UNI CEI EN ISO 13485

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#### SAFETY CONTAINERS ROUND SHAPE

*This line includes round shape disposable and puncture resistant containers for the disposal of sharp and cutting waste and it comes in sever sizes.*

*The conical shape and the indent on the lid ease the disposal and the remove of needles, whilst the anti-defluxion flow wings on the lid prevent the outflow of the waste. All the models have temporary and definite closure and are fitted with solid handle; the three most volumious models are fitted with an ergonomic handle that eases use.*

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*Ecological plastic material (colored copolymer polypropylene compliant to REACH regulations and with masterbatch heavy metal free and cadmium free). The container body is yellow and the lid is red to allow the immediate detection and warning of the dangers of the content.*

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*Compliant to AFNOR NF X-30-511  
Compliant to NF Mark  
Compliant to ISO 23907-1 - Kite Mark  
Compliant to ADR - UN Certificate  
Compliant to FDA K071517  
Compliant to TRBA 250*

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- *Ergonomic handle for the more spacious models*
- *Machanism for unhooking insulin needles*
- *Anti-defluxion wings that prevent the outflow of the waste in containers cod. 7015 – 7020 – 7050 – 7070 – 7120.*
- *Irreversible permanent closure*
- *Maximum filling level indicator*
- *Various supports available*
- *No emission of any harmful gasses during the inceneration process. Once filled, the containers have to be disposed through incineration, releasing no toxic emissions*
- *Suitable to autoclave at 134° for 18 min*

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*Item designed for low volumes of waste (care tray, trolley, domestic care, doctor's office, etc.)*

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*Item designed for frequent use (Hospitals, Helthcare Centres, Clinics, etc.)*

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Item designed for frequent use (Hospitals, Helthcare Centres, Clinics, etc.)

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**ASSEMBLY:** Firmly press on all the perimeter, until the lid clicks on the body

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### IMPORTANT!

Before use, for codes 7015 – 7020 – 7050 – 7070 – 7120 press on the anti-reflux fins in order to create a funnel-shaped opening. This device eases the insertion of the waste and creates a barrier to prevent from leakage.

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**TEMPORARY CLOSURE:** The container could be temporarily closed using the cap upside down.

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**PERMANENT CLOSURE:** The container could be closed permanently applying pressure on the cap until it clicks on the lid

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*During use, open and close the cap in temporary mode.*

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*Close the container permanently and irreversibly once full, positioning the lid on the permanent closure side.*

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*Use the “V” removal device to separate needle and syringe*

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**Insulin pens removal:** Use the “marguerite” device and pull the pen bottom-up (leverage effect) before unscrewing

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**Hole for introduction of waste:** large orifice to allow the introduction among others, of tubes, scalpels, etc..

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*Do not reopen the closed container and sealed permanently*

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*Do not attempt to rescue objects already in the container*

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*Respect the filling level indicated: 80% of real capacity*

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*Containers are tested to not being perforated by sharp objects allowing safety to the operators, as long as the container is used properly according to instructions*

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*Keep out of flame or heat sources which might damage the product*

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*Do not use after expiry date or if packing is opened*

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*Do not re-use: Disposable device*

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*Do not vary the intended purpose of the product*

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*Store in dry place, Temperature range: min -10°C max +50°C*

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*Disposal: use appropriate personal protective equipment and act according to applicable regulations*

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*Before use with particular substances check the resistance / compatibility chart on our catalogue*

## IMBALLO / PACKING

<b>CODICE CODE</b>	<b>Quantità (pz): Quantity (pcs):</b>	<b>Misura esterna scatola (cm): External box dimensions (cm):</b>	<b>Volume (m<sup>3</sup>): Volume (m<sup>3</sup>):</b>
<b>7006</b>	100	62 x 41 x 34 h.	0,086
<b>7008</b>	100	62 x 41 x 34 h.	0,086
<b>7015</b>	100	60 x 41 x 44 h.	0,108
<b>7020</b>	75	60 x 41 x 44 h.	0,108
<b>7050</b>	55	79 x 53 x 49 h.	0,205
<b>7070</b>	40	79 x 53 x 49 h.	0,205
<b>7120</b>	25	79 x 53 x 49 h.	0,205