Defibrinated Sheep Blood

[REF] SR0051B, SR0051C, SR0051D, SR0051E & SR0051R

Intended Use

A non-selective, general-purpose enrichment supplement that may be used in the preparation of many types of medium for the isolation and culture of pathogenic and non-pathogenic microorganisms from clinical samples (e.g. cerebral spinal fluid, pus and wound swabs, sputum, eye swabs and ear, nose and throat swabs, blood and faecal samples).

The device is for professional use only, is not automated nor is it a companion diagnostic.

Summary and Explanation

The addition of blood to culture media provides means of identification, as the ability to haemolyse blood can be used to classify certain microorganisms^{1,2}. The presence and pattern of haemolysis, the size of the zone of haemolysis, and colony morphology can be used together to identify isolates cultured using blood agar, such as CNA^{1,2}. Blood agar, such as CBA, are recommended for the primary culture of a number of human pathogens, including Gram-positive microorganisms^{1,2}.

Principle of Method

Defibrinated Sheep Blood is collected aseptically, then processed mechanically to remove the fibrin; this prevents the blood from clotting.

Materials Provided

SR0051B: is a liquid additive of dehydrated culture medium that is available in a quantity of 25 mL

SR0051C: is a liquid additive of dehydrated culture medium that is available in a quantity of 100 mL

SR0051D: is a liquid additive of dehydrated culture medium that is available in a quantity of 50 mL

SR0051E: is a liquid additive of dehydrated culture medium that is available in a quantity of 500 mL

SR0051R: is a liquid additive of dehydrated culture medium that is available in a quantity of 500 mL

Materials Required but Not Supplied

- Inoculating loops
- Swabs
- Collection containers
- Incubators
- Dehydrated culture media (CM691, CM972, CM331, CM271)

Storage

- Store product in its original packaging between 2°C and 8°C.
- Keep container tightly closed.
- The product may be used until the expiry date stated on the label.
- Store away from light.
- Allow reconstituted product to equilibrate to room temperature before use.

Warnings and Precautions

- DO NOT USE SUPPLEMENTS IF YOU ARE PREGNANT OR CONTEMPLATING PREGNANCY.
- Rehydrate all supplements in Safety Cabinet, wear disposable gloves.
- Supplements may be dispensed on the bench after rehydration, wear disposable gloves and safety glasses.
- Disposable gloves must be worn for rehydrating, dispensing and disposing of ALL Supplements (i.e. wear disposable gloves whilst handling supplements).
- All waste (i.e. rehydrated supplement) must be emptied into the HAZARDOUS WASTE container situated in the Microbiology laboratory.
- The vial and rubber bung must be rinsed into the HAZARDOUS WASTE container.
- The metal lid and rinsed rubber bung should be placed in the waste bin.
- The rinsed glass vial should then be placed in the glass bin.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

Specimen should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI).

Procedure

1. Allow supplement to equilibrate to room temperature before use.
2. Refer to the IFU for CM0691B, CM0972B, CM0331B, CM0271B. IFUs can be found at www.thermofisher.com
3. Mix well and aseptically dispense into sterile containers.

Additional Information

Defibrinated Sheep Blood devices are liquid additives of dehydrated culture medium. Given below is the list of dehydrated culture medium (DCM) for use with defibrinated sheep blood devices.

DCM	Product Code	Clinical Pathogens Isolated
Brucella Agar Base	CM691	<i>Brucella</i> species
Anaerobe Basal Agar (Dehydrated).	CM972	<i>Bacteroides</i> spp
Columbia Blood Agar Base (Dehydrated)	CM331	<i>Haemophilus</i> species and the HACEK group of organisms
Blood Agar Base No. 2 (Dehydrated)	CM271	<i>Staphylococcus</i> species

Bibliography

1. Public Health England. 2014a. 'Identification of Streptococcus Species, Enterococcus Species and Morphologically Similar Organisms'. UK Standards for Microbiology Investigations. ID 4 (3). <https://www.gov.uk/government/publications/smi>

-id-4-identification-of-streptococcus-species-
enterococcus-species-and-morphologically-
similar-organisms.

2. Public Health England 2020b. 'Identification of Staphylococcus Species, Micrococcus Species and Rothia Species'. UK Standards for Microbiology Investigations. ID 7 (4).
<https://www.gov.uk/government/publications/smi-id-7-identification-of-staphylococcus-species-micrococcus-species-and-rothia-species>.

Oxoid Limited, Wade Road, Basingstoke,
Hampshire, RG24 8PW, UK

For technical assistance please contact your local distributor.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Do not re-use
	Consult instructions for use or consult electronic instructions for use
	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier

Revision information

Version	Date of issue and modifications introduced
2.0	2022-12-07

©2022 Thermo Fisher Scientific Inc. All rights reserved.
All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

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Bibliograafia

1. Public Health England. 2014a. 'Identification of Streptococcus Species, Enterococcus Species and Morphologically Similar Organisms'. UK Standards for Microbiology Investigations. ID 4 (3) . <https://www.gov.uk/government/publications/smi-id-4-identification-of-streptococcus-species-enterococcus-species-and-morphologically-similar-organisms>.

2. Public Health England 2020b. 'Identification of Staphylococcus Species, Micrococcus Species and Rothia Species'. UK Standards for Microbiology Investigations. ID 7 (4). <https://www.gov.uk/government/publications/smi-id-7-identification-of-staphylococcus-species-micrococcus-species-and-rothia-species>.

Oxoid Limited, Wade Road, Basingstoke,
Hampshire, RG24 8PW, ÜK

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Bibliographie

1. Public Health England. 2014a. 'Identification of Streptococcus Species, Enterococcus Species and Morphologically Similar Organisms'. UK Standards for Microbiology Investigations. ID 4 (3).
<https://www.gov.uk/government/publications/smi-id-4-identification-of-streptococcus-species-enterococcus-species-and-morphologically-similar-organisms>.
2. Public Health England 2020b. 'Identification of Staphylococcus Species, Micrococcus Species and Rothia Species'. UK Standards for Microbiology Investigations. ID 7 (4).
<https://www.gov.uk/government/publications/smi-id-7-identification-of-staphylococcus-species-micrococcus-species-and-rothia-species>.

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Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, Royaume-Uni

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Bibliografi

1. Public Health England. 2014a. 'Identification of Streptococcus Species, Enterococcus Species and Morphologically Similar Organisms'. UK Standards for Microbiology Investigations. ID 4 (3).
<https://www.gov.uk/government/publications/smi>

-id-4-identification-of-streptococcus-species-enterococcus-species-and-morphologically-similar-organisms.

- Public Health England 2020b. 'Identification of Staphylococcus Species, Micrococcus Species and Rothia Species'. UK Standards for Microbiology Investigations. ID 7 (4). <https://www.gov.uk/government/publications/smi-id-7-identification-of-staphylococcus-species-micrococcus-species-and-rothia-species>.

Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, UK

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