

HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.

HLX/ZS-10ml-002

Revision No.

A/0

Date of Revision

2022.02.01

TECHNICAL DATASHEET OF STERILE HYPODERMIC SYRINGES FOR SINGLE USE




PRODUCT DETAILS

Description:	Disposable 3-Part Syringe 10ml, with needle 22GX1 1/2"	
Type:	Luer Slip	
Graduation:	0.5 ml	
Shelf life:	5 years from date of manufacturing	
REF No:	H10LS22/112	
Composition:	<i>Disposable 3-Part Syringe with needle</i> is a medical device composed of:	
	-Barrel;	-Luer Slip precision ensures a secure connection of the needle; -Transparent cylinder allows easier and precise dosing, observation and elimination of air bubbles; -Clear and precise scale allows accurate and simple dosing; -Equipped with stopping design, which prevents the piston from accidentally leaves.
	-Piston;	-Anatomically designed piston, which facilitates handling of the syringe.
	-Gasket;	-Latex-free, with double ring; -Ensures good seal and high precision; -Allows a smooth and regular piston progress when in contact with high quality lubricant.
	-Needle;	-Provides sharp and smooth puncture.
	-Protective cap.	-Tight closure ensures high level of safety against accidental needle stick injury when handling the syringe.

Materials: PP, Rubber (Latex Free), Stainless Steel, Lubricant

Sterilization: EO (Ethylene Oxide)

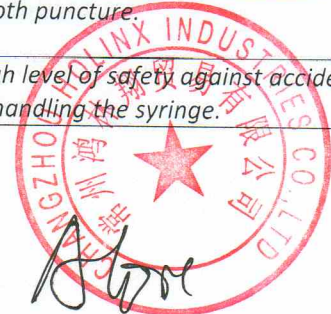

HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.	HLX/ZS-10ml-002	Revision No.	A/0	Date of Revision	2022.02.01
--------------	-----------------	--------------	-----	------------------	------------

Product standards: ISO 7886-1:2017, ISO 7864-2016**Certificates:** CE, ISO13485**Packaging:** Single Blister packaging with needle at side, 100 pcs per box, 16 boxes per carton**Intended Use:**

Its purpose is to suck and introduce liquids. Not suitable for high pressures or cancer medicines.

Cautions:

- Read instructions before use.
- Before usage confirm the sterility validity, packing completeness, the LOT number and expiry date.
- Use only if package is undamaged and validity not expired;
- The syringes should be used immediately after unit package opened.
- To be used by qualified doctor or paramedic only.
- Check the tightness of needle before the use.
- For single use. The re-sterilization and/or use by more than one patient may cause a cross-infection.
- Destroy after use.

Instructions for use:

1. Select the appropriate syringe with hypodermic needle for your procedure.
2. Examine individual package for integrity of packaging prior to use. Do not use if packaging has been torn.
3. Before usage, open the unit package using aseptic technique.
4. Remove protective needle cap.
5. Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.
6. Apply protective cap when procedure is finished.
7. Any used syringe is considered contaminated and should be disposed of in an approved container immediately after use.
8. Immediately dispose of safety after single use to avoid risk of infection.

Features:

- Non-toxic;
- Non-pyrogenic;
- PTH and DEHP free;
- Latex free;
- Sterile unit packages.



HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.	HLX/ZS-10ml-002	Revision No.	A/0	Date of Revision	2022.02.01
--------------	-----------------	--------------	-----	------------------	------------

Storage:

Keep away from sunlight. Protect from humidity and heat sources. Not store with organic or chemical solvents. Not store with corrosive gases. Store in dry, cool, good ventilated and clean room, where the relative humidity is less than 80%.

Sample blister packaging design:

 Seringă de unică folosință	<p>Precauție:</p> <ul style="list-style-type: none">• Nu reesterilizați• Steril dacă ambalajul nu a fost deschis sau deteriorat• Nu utilizați dacă ambalajul este deteriorat sau a fost deschis anterior• Aruncați după o singură utilizare• Nu depozitați la lumina directă a soarelui, la temperaturi extreme sau la umiditate ridicată	 <p>Shanghai International Holding Corp. GmbH(Europe) Add.: Eiffelstrasse 80, 20537 Hamburg, Germany</p> <p>Changzhou Holinx Industries Co., Ltd. No.1 Dongtang Rd., Zhenglu Town, Changzhou, Jiangsu, China Repr. Aut. in Republica Moldova: S.C. "MOLDOVA TRADE" SRL Adr.: No 42, Gh. Asachi str., Chișinău, Republica Moldova</p>	<ul style="list-style-type: none">• Netoxic• Apirogen• De unică folosință• Fără latex	10 ML 22GX1 1/2"	REF: HTDLS22-112 LOT: 20220816 PROCT: 2022 08 16 EXP: 2027.08.15
--	--	---	--	----------------------------	---

Editor:

Audit:

Approve:

