

orthoBAND FAST plaster of Paris bandage

TD-04-I.2.c-1.1

edition I

date: 2021-04-22

amendment of: 2021-12-22

MANUFACTURER: ZARYS International Group sp. z o.o. sp.k.

ADDRESS: ul. Pod Borem 18, 41-808 Zabrze, Poland; T: +48 32 376 07 00-02,06 F: +48 32 370 38 08; www.zarys.com

CLASSIFICATION

Class I medical device compliant with Medical Devices Regulation (EU) 2017/745 (MDR)

INTENDED PURPOSE

 disposable device intended for use as a full or partial dressing for stiffening/immobilisation in orthopaedic, ambulatory and surgical procedures

FEATURES

- gypsum substance carrier made of 100 % cotton gauze
- min. 94% white natural gypsum, carrier sprayed twice with gypsum mass - carrier material covering both sides
- carrier is wound on a "X" shaped plastic core
- prior to application requires soaking in a container of water at the recommended temperature of 20°C for approximately 5 seconds
- initial setting time 3 minutes
- complete stiffening of the plaster of Paris bandage takes place within 24 hours
- available with a straight edge (6 cm, 8 cm) or serrated edge (10 cm, 12 cm, 14 cm, 15 cm, 20 cm)
- does not absorb X-rays
- white colour
- single-use
- non-sterile
- shelf life: 5 years



	CATALOGUE NUMBER	DIMENSIONS	UNIT PACKAGING (SALES PACKAGING) FOIL BAG	BULK PACKAGING CARTON
_	OB06300-F	6 cm x 3 m	1 pc.	48 x 1 pc.
	OB08300-F	8 cm x 3 m	1 pc.	48 x 1 pc.
	OB10300-F	10 cm x 3 m	1 pc.	48 x 1 pc.
	OB12300-F	12 cm x 3 m	1 pc.	48 x 1 pc.
	OB14300-F	14 cm x 3 m	1 pc.	36 x 1 pc.
	OB15300-F	15 cm x 3 m	1 pc.	36 x 1 pc.
	OB20300-F	20 cm x 3 m	1 pc.	24 x 1 pc.