


2

**The Remask window appears.**

The current mask frame number is shown in the [Current Mask Frame Number] box.

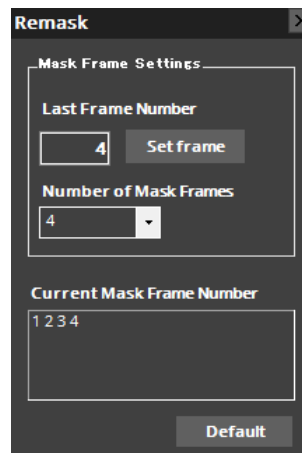

**NOTE** To remask with an image that was acquired with Flex-APS (optional) and position adjusted, uncheck the check box of [Flex-APS] in the Pixel Shift screen before start remask.



3

To change from the automatically-selected mask frames, set the last frame to be used as a new mask in [Last Frame Number] and click [Set Frame] button on the [Remask] window.

**Hint** We recommend to mask the frame where the patient's body movement is not affected or just before the contrast agent enters.

4

To change the number of frames used to create the mask, select a different value from the drop-down list of [Number of Mask Frames].

4