For use on Mindray Chemistry Analyzer

# CD80 DETERGENT

**mindray**

# CD80 DETERGENT

## Order Information

Cat. No.	Specification
105-000748-00	1 × 1 L
105-035958-00	2 × 1 L
105-000106-00	2 × 2 L
105-000107-00	6 × 2 L
105-011705-00	1 × 1 L

## Intended Purpose

CD80 DETERGENT is used to clean Sample and Reagent Probes, Mixers and Cuvettes of the Chemistry Analyzer manufactured by Mindray.

## Product Characteristics

CD80 DETERGENT is an improved phosphate-free, biodegradable alkaline concentrated liquid cleaning agent containing surfactants. It's effective for removing proteins, lipids, ions and other chemistry reaction residues on surfaces of Sample and Reagent Probes, Mixers and Cuvettes of the Chemistry Analyzers.

## Ingredients

CD80 DETERGENT is an alkaline agent with 2% potassium hydroxide, it also contains EDTA disodium salt dihydrate, surfactants.

## Storage and Stability

CD80 DETERGENT is stable up to expiration date indicated on the label when stored in unopened container at 2-35°C.

Once opened in used, CD80 DETERGENT is stable for 3 months when the operating environment is at 2-35°C.

## Usage

This product shall be used at 2-35°C. Please shake before using.

## Warnings

1. The product contains potassium hydroxide which is an alkaline.
2. Contains severe eye irritant, mild skin irritant. Please wear eye protection and latex gloves when handling.
3. For in vitro diagnostic use only. For laboratory professional use.
4. Do not inhale and/or ingest. In case of inhaling or ingestion, immediately seek help from physician.
5. Avoid eye and skin contact. In case of contact, immediately flush the affected area with plenty of water and go to see a doctor if necessary.

## Precaution

1. Do not use product beyond the expiration date and the in-use date.  
Do not mix fresh reagents with in-use reagents.  
Avoid the formation of foam.
2. Please confirm the integrity of the package before use. Do not use the CD80 DETERGENT with damaged packages.
3. If unintentionally opened before used, store the CD80 DETERGENT tightly capped at 2-35°C, and the stability is equally to in-use stability.
4. If product has been frozen or been transported at high temperature, allow to warm at room temperature and shake sufficiently before using.
5. Safety data sheet is available for professional user on request.
6. Dispose of waste product, unused product and contaminated packaging in accordance with local or national guidelines for biohazard waste disposal.
7. CD80's capacity of cleaning and compatibility of materials must be verified before applying to other medical devices.
8. CD80's performance is influenced by expiration date, air pollution, or mixing with other reagents.

9. All identified risks have been reduced as far as possible without adversely affecting the benefit-risk ratio, and the overall residual risk is acceptable.
10. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.
11. This product contains components classified as follows in accordance with the Regulation (EC) No 1272/2008:

<b>Danger</b>	
H314	Causes severe skin burns and eye damage.
<b>Prevention:</b>	
P260	Do not breathe mist/vapours/spray.
P264	Wash all exposed external body areas thoroughly after handling.
P280	Wear protective gloves, protective clothing, eye protection and face protection.
<b>Response:</b>	
P301+P330+P331	IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
P303+P361+P353	IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/or shower.
P305+P351+P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor/physician/first aider.
P363	Wash contaminated clothing before reuse.
P304+P340	IF INHALED: Remove person to fresh air and keep comfortable for breathing.
<b>Disposal:</b>	
P501	Dispose of contents/container to authorised hazardous or special waste collection point in accordance with any local regulation.

### Graphical symbols

In Vitro Diagnostic  
medical device

Unique device  
identifier

European  
Conformity

Consult Instructions  
For use

Use-by  
date

Authorized representative in  
the European Community

Batch Code

Temperature  
limit

Manufacturer

Catalogue  
number

© 2022-2024 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved

**Manufacturer:** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

**Address:** Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

**E-mail Address:** service@mindray.com; **Website:** www.mindray.com

**Tel:** +86-755-81888998; **Fax:** +86-755-26582680

**EC-Representative:** Shanghai International Holding Corp. GmbH (Europe)

**Address:** Eiffestraße 80, Hamburg 20537, Germany

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© 2022-2024 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. Tous droits réservés
**Fabricant :** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
**Représentant en Europe :** Shanghai International Holding Corp. GmbH (Europe)