

	• Include Dose SR ▫ You can set whether to include Dose SR. ▫ Activated only when the DICOM option is checked. • Include Text File ▫ You can set whether to include Text File. ▫ Activated only when the JPEG option is checked.
Compression	Sets the way of JPEG2000 compression. If the DICOM server does not support JPEG2000, this function cannot be used. • No Compression: Not compressing JPEG2000 • JPEG2000 Lossless: Compressing JPEG2000 without a loss • JPEG2000 Lossy: Compressing JPEG2000 with a loss
Compression Quality	Sets JPEG2000 compressibility of DICOM images. It activates when checking the option JPEG2000 Lossy Compression. Sets compressibility by adjusting the slider.
Image Bits	Sets the number of Bit that expresses a pixel in DICOM image. 12 bits: 14->12 / 14 bits: 14->14(Default)
Burned in Annotation	Sets whether to include annotation information in the image during an External Save or not. • Yes: Included / No: Not included
Burned in Information	Sets whether to include the Overlay information in the image during an External Save or not. • Yes: Included / No: Not included
Software Collimation	Sets the image crop status. • NONE: Original image without cropping • Cut: Cropped area of the image • Fill: The outside area of the cropped image is filled with black. • Fill & Center: The image cropped from the original image is displayed at the center, and the area other than the cropped image is filled with black.
Image Modality	Sets modality information. • Current: Uses the modality information without change. • DX / CR / DR: Uses the modality information by fixing it among DX/CR/DR.
Language	Sets the language for DICOM encoding.
DAP Unit Type	The DAP value to be transferred to the DICOM image is changed according to the set unit. • mGy*cm², uGy*m², dGy*cm²
 • The higher the compressibility, the image loss is happened severely. This may affect the image quality when performing loss image compression.	