


Operation Panel

Buttons

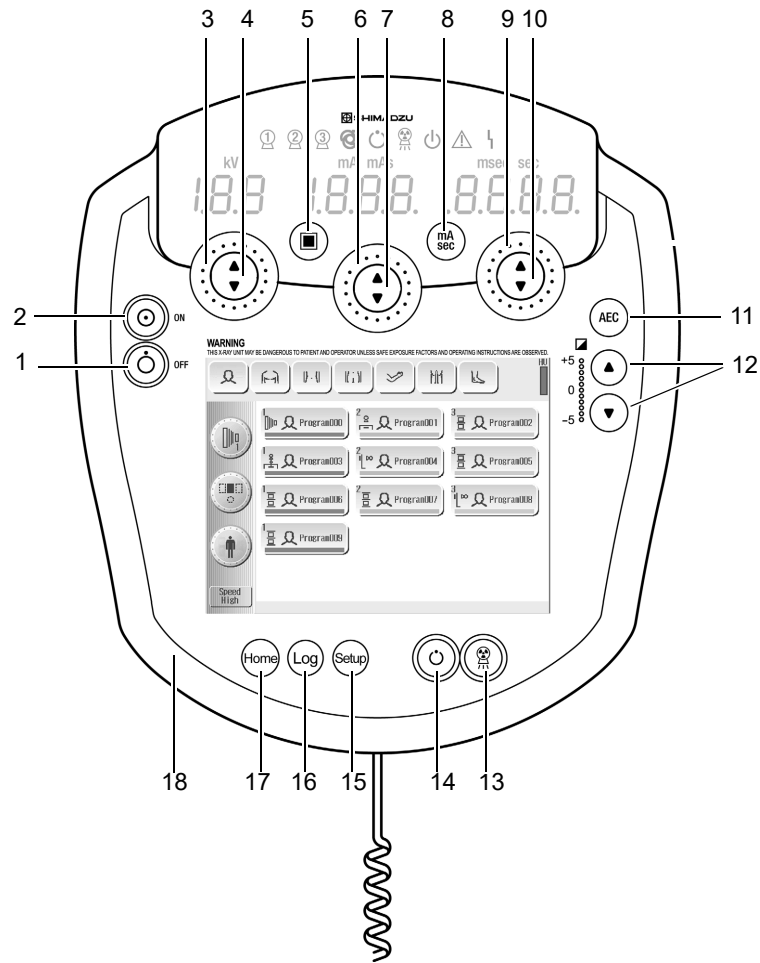

No.	Name	Function
1	Power OFF Button	Turns off the equipment power.
2	Power ON Button	Turns on the equipment power.
3	kV Shuttle	Sets the radiography tube voltage. Turn clockwise to increase the displayed value. Turn counterclockwise to decrease the displayed value. Effective for making large changes to the value.
4	kV-up Button/ kV-down Button	Sets the radiography tube voltage. Press ▲ to increase the displayed value. Press ▼ to decrease the displayed value. Effective for making small changes to the value.
5	Focus Switch Button	Switches the X-ray tube focus. Toggles between large focus and small focus each time this button is pressed. The switch lights when small focus is selected.
6	mA/mAs Shuttle	Sets the radiography tube current or the radiography tube current time product. Turn clockwise to increase the displayed value. Turn counterclockwise to decrease the displayed value. Effective for making large changes to the value.