


Endocervical Sterile Histology Cervical Brush PE Head

Item Number: 8302

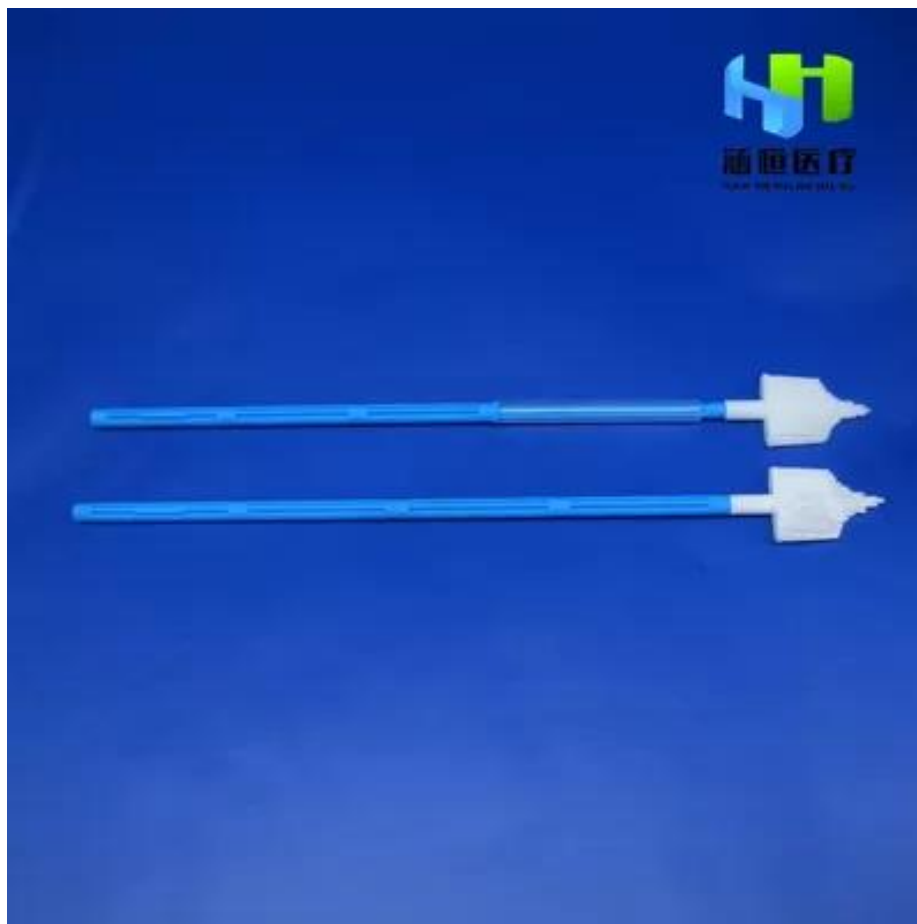

Properties:	General Examination Apparatus	Size:	206MM
Stock:	Yes	Material:	PE HEAD, PE Head, PP Rod
Quality Certification:	Ce	Instrument Classification:	Class II
Safety Standard:	EN 149 -2001+A1-2009	Product Name:	Disposable Sterile Cervical Sampler
Application:	Sampling For Testing	MOQ:	10000
Packing:	Individual Package	Sample:	Can Be Offered
Certificate:	CE ISO	Port:	Shanghai
High Light:	Endocervical Histology Cervical Brush, PE Head Histology Cervical Brush, Endocervical sterile cytology brush		

8302 Disposable cervical sampler HPV gynecological examination woman brush

The Cervical Brush enables simultaneous collection of ectocervical, endocervical and transformation-zone cells with a single device. The brush is made up of well-defined semi-circular soft, flexible hairs for the optimal sample collection. The Cervical Brush can be used for HPV testing, conventional cytology and liquid-based cytology. The hydrophobic material of the brush facilitates the release of the cell material into the fluid or on to a glass slide.

Item Number	8302
Production Description:	Disposable cervical sampler brush HPV gynecological examination woman PP LDPE cleaning head plastic handle ---head material: LDPE, handle material: PP
Product Application :	For clinical use of cervical cells for sampling,HPV examination
Logo Print:	printing on the package
Production Packing:	Sterilization packaging,paper-plastic
Regular Carton Size:	57*30*50cm,100PC/box,2000pcs/carton
N.W./G.W.	10/11kg
Production Time:	According to customer`s quantity and PO date &Rush order
Sample Delivery:	20-30 days
Shipping :	Air or sea -Fedex,DHL,USP,DHL
Usage	Hospital using,Medical using, Home nursing,Clinic nursing etc

Instructions

1. This product is sterilized by ethylene oxide, sterile and non-toxic.
2. After the vagina is dilated by the vaginal speculum, insert the tip end of the cervical sampler into the cervix until the bristles on both sides are in close contact with the two sides of the cervix.
3. Hold the handle of cervical sampler with thumb and forefinger and rotate the cervical sampler for 2-5 turns in one direction, then pull out the cervical sampler.
4. Disassemble the brush head and put it into the cell preservation solution for sample inspection.

Disposable Endocervical Spatula

(Folkman spoon)

Item Number: 8401



Disinfecting Type:	EO Sterilization	Properties:	For Ectocervix And Posterior Fornix Cells In Cervical Cancer Screening
Stock:	NO	Shelf Life:	5 Years
Material:	ABS	Quality Certification:	CE/ISO13485
Instrument Classification:	Class Is	Safety Standard:	None
High Light:	endocervical brush and spatula, endocervical brush and spatula, pap smear spatula and brush		

Disposable Cervical Sampler (Spatula - Folkman spoon) 8401

Brand	Hanheng
Supplier and Manufacturer	Jiangsu Hanheng Medical Technology Co., Ltd
Product Name	Disposable Cervical Sampler (Folkman spoon)
Indication for Use	Used to sample ectocervix and posterior fornix cells in cervical cancer screening procedures.
Product Number of suppliers	8401
Storage temperature	-10°C~40°C
Shelf life	5 years
Instruction for Use	<p>Obtain an adequate sampling from the ectocervix using a spatula by placing the rounded tip of the spatula into the cervix until the curved edge of the spatula rests against the ectocervix.</p> <p>Rotate the spatula 360 degrees.</p> <p>Immediately rinse the spatula in the vial by swirling the spatula vigorously in the vial 10 times.</p> <p>Discard the spatula. Do not leave the spatula in the vial.</p>
Sterilization	Sterilized by EO