



HBsAg Card Test

For the detection of Hepatitis B surface antigen in human whole blood / serum / plasma

One-step Hepatitis B Card Test for Whole blood / Serum / Plasma



- Clear background.
- Self contained test procedure.
- Specifically engineered for whole blood specimen, serum & plasma.
- > 100% sensitivity and 100% specificity.
- Storage temperature 4°C 30°C.
- Detects all subtypes of HBsAg.
- Long shelf life up to 24 months.

TEST PROCEDURE

Add 2 drops of whole

blood or serum or plasma

to the specimen well.



- Unmistakable results at 20 minutes.
- Simple and easy to use, does not require advanced medical training.
- Cost effective, can be used effectively in resource limited conditions.
- Accurate and reliable.
- Does not require cold chain & convenient for small volume users.


(3)


Ideal for laboratory / field conditions.





RESULTS INTERPRETATION

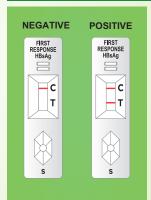

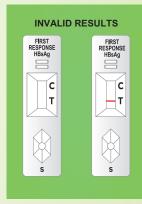

Specimen

transfer

device

Do not interpret after 25 min.

Result at 20 min.

KIT CONTENTS

- · Test device with desiccant
- · Assay buffer bottle
- · Specimen transfer device
- Sterile lancet
- Alcohol swab
- Instructions for use

Place of Evaluation	Year of	Sensitivity	Specificity
Trace of Evaluation	Testing	Gensiavity	Opecinicity
Biosynex, France	2018	100%	100%
Virus Research Institute, Uganda	2018	99.1%	100%

EXTERNAL LABORATORY EVALUATION REPORTS

Add 1 drop of assay

buffer to the

specimen well.

ORDERING INFORMATION

info@premiermedcorp.com


Type : Bulk Packing


Catalogue No.	Pack Size	
PI10FRC25	25 Tests / Kit	
PI10FRC30	30 Tests / Kit	
PI10FRC25CE	25 Tests / Kit	
PI10FRC30CE	30 Tests / Kit	

(E₁₄₃₄



ISO 13485 & EN ISO 13485 Certified Company

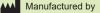

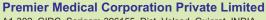

A1-302, GIDC, Sarigam 396155. Dist. Valsad, Gujarat, INDIA
• Customer support email: info@premiermedcorp.com • Tel.: +91 260 2780112/113,

• www.premiermedcorp.com