



OPERATION MANUAL

MEASUREMENTS VOLUME

DIAGNOSTIC ULTRASOUND SYSTEM

Aplio a CUS-AA000

Read and understand this manual before operating the equipment.
After reading, keep this manual in an easily accessible place.

Trademarks

Windows is a trademark of the Microsoft group of companies.

BI-RADS[®] is a registered trademark (in the United States) of the American College of Radiology and the American College of Radiology Association.

BI-RADS[™] is a trademark (in countries other than the United States) of the American College of Radiology and the American College of Radiology Association.

TI-RADS[™] and LI-RADS[™] are trademarks of the American College of Radiology and the American College of Radiology Association.

This manual may include trademarks or registered trademarks of other companies.

Note that the [™] mark and the [®] mark may or may not be used in this manual.

IMPORTANT!

- (1) No part of this manual may be copied or reprinted, in whole or in part, without written permission.
- (2) The contents of this manual are subject to change without prior notice and without our legal obligation.
- (3) The contents of this manual are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

©CANON MEDICAL SYSTEMS CORPORATION 2023-2024
ALL RIGHTS RESERVED

CONTENTS

	Page
1. Outline of Measurement Functions.....	9
1.1 Switches Used for Measurement.....	10
1.2 Setting the Measurement.....	12
1.3 Modifying a Measurement.....	14
1.4 Deleting a Measurement	17
1.5 Starting Next Measurement After Current Measurement	18
1.5.1 During basic measurements.....	18
1.5.2 During application measurements	18
1.6 Measurement Result Display Area	19
1.6.1 Display type	19
1.6.2 Customization of the Measurement Result Display Area.....	19
1.6.3 Change of the format	21
1.7 Worksheet Screen.....	22
1.8 Setting Measurement Preset.....	24
1.9 Procedure for Applying Application Measurement Labels to Basic Measurement	25
1.10 Recovering Measured Values during an Exam.....	26
2. 2D-Mode Measurements.....	27
2.1 Distance Measurement (Distance)	29
2.2 Curved Line Distance Measurement (Trace Length)	29
2.2.1 Continuous Trace.....	30
2.2.2 Spline Trace	31
2.3 Angle Measurement (Angle).....	32
2.4 Angle Measurement (3 Point Angle).....	33
2.5 Area/Circumference Measurement (Area)	33
2.5.1 Continuous Trace.....	34
2.5.2 Spline Trace	34
2.5.3 Ellipse	35
2.5.4 Cross.....	37
2.6 Volume Measurement (Volume).....	39

	Page
2.7	Joint Measurement..... 40
2.8	MeanIMT Measurement..... 41
2.9	%Stenosis (Diam) Measurement..... 43
2.10	%Stenosis (Area) Measurement 44
2.11	IMT-C10 Measurement..... 45
3.	M-Mode Measurements 47
3.1	M-Mode Distance Measurement (Distance) 49
3.2	Slope Measurement (Slope) 50
3.3	Time Measurement (Time) 51
3.4	Heart Rate Measurement (HR)..... 52
4.	Doppler-Mode Measurements..... 53
4.1	Velocity Measurement (Velocity)..... 54
4.2	Acceleration Measurement (Accel) 55
4.3	Time Measurement (Time) 56
4.4	Heart Rate Measurement (HR)..... 56
4.5	Velocity Trace Measurement (Vel Trace) 56
4.5.1	Continuous Trace..... 57
4.5.2	Spline Trace 59
4.5.3	Line Trace..... 60
4.5.4	Range 61
4.5.5	Auto Range 64
4.5.6	RI Only 65
4.6	PI Measurement (PI Trace) 66
4.7	RI Measurement 67
4.8	S/D Measurement..... 68
4.9	Flow Volume Trace Measurement (Flow Volume) 69
4.9.1	Flow volume trace measurement using two switches..... 69
4.9.2	Flow volume trace measurement using one switch..... 72
5.	Application Measurement (Cardiac)..... 75
5.1	Introduction 76
5.1.1	Starting the application measurement menu..... 76
5.1.2	Displaying the measurement menu on the monitor 77
5.1.3	Heart rate acquisition methods (Cardiac) 78
5.2	2D-Mode Measurements (Cardiac)..... 80
5.2.1	LV measurement (MOD Simpson method)..... 80

	Page	
5.2.2	LV measurement (calculation using a three-chamber cross-sectional image).....	90
5.2.3	LV measurement (Teichholz method)	94
5.2.4	LV measurement (Cube method).....	99
5.2.5	LV measurement (Gibson method)	99
5.2.6	LV measurement (Single-plane (aka Area-Length) method).....	100
5.2.7	LV measurement (Biplane method).....	102
5.2.8	LV measurement (Bullet method)	104
5.2.9	LA measurement (MOD Simpson method).....	106
5.2.10	LA measurement (3-Axis method).....	109
5.2.11	LA measurement (Measurement using the MOD Simpson method in 3 cardiac phases)	111
5.2.12	LV Mass measurement.....	118
5.2.13	MPI measurement.....	120
5.2.14	Auto EF measurement	124
5.3	M-Mode Measurement (Cardiac).....	137
5.3.1	LV measurement (Teichholz method)	137
5.3.2	LV measurement (Cube method).....	141
5.3.3	LV measurement (Gibson method)	141
5.3.4	Aortic valve measurement	142
5.3.5	Mitral valve measurement.....	144
5.3.6	Tricuspid valve measurement.....	146
5.4	Doppler-Mode Measurement (Cardiac).....	147
5.4.1	Aortic valve measurement	147
5.4.2	Mitral valve measurement.....	156
5.4.3	Pulmonary valve measurement.....	169
5.4.4	Tricuspid valve measurement.....	177
5.4.5	Pulmonary vein blood flow measurement.....	185
5.4.6	PISA.....	188
5.4.7	Coronary	190
5.4.8	IVC.....	193
6.	Application Measurement (Vascular).....	195
6.1	Starting the Application Measurement Menu	196
6.1.1	2D-mode measurement.....	196
6.1.2	Doppler-mode measurement.....	198
6.1.3	Measurement value display on the Worksheet screen.....	201

	Page
6.2	Auto-IMT Measurement..... 202
6.2.1	Touch panel menu 202
6.2.2	Auto-IMT measurement method 206
6.2.3	IMT-C10 measurement method 209
6.3	Auto Measurement 210
6.4	Worksheet Screen..... 211
7.	Application Measurement (Radiology)..... 213
7.1	Starting the Application Measurement Menu 214
7.1.1	2D-mode measurement..... 214
7.2	Worksheet Screen..... 218
8.	Multi Parametric Report (Radiology) 219
8.1	Outline of Multi Parametric Report 220
8.2	Abdominal Measurement Applications..... 220
8.2.1	SWE/SWD 220
8.2.2	ATI 220
8.3	Details of Multi Parametric Report..... 221
8.3.1	Multi Parametric Report display 222
8.3.2	Mini-Worksheet display 226
8.3.3	Changing the graph display settings..... 228
8.4	Displaying Previous Exam Data in the Multi Parametric Report..... 244
8.5	Storing a Multi Parametric Report 245
9.	RADS (Reporting and Data Systems) 247
9.1	Outline for Diagnostic Framework Display 248
9.1.1	Starting the RADS..... 249
9.1.2	RADS Screen..... 250
9.2	BI-RADS 251
9.2.1	Findings Entry Area..... 252
9.2.2	Summary tab 260
9.2.3	Creating the Worksheet..... 272
9.2.4	Report 277
9.3	TI-RADS..... 278
9.3.1	Findings Entry Area..... 279
9.3.2	Creating the Worksheet..... 284
9.3.3	Summary worksheet..... 286
9.3.4	Report 287

	Page
9.4 CEUS LI-RADS.....	288
9.4.1 Findings Entry Area.....	289
9.4.2 Creating the Worksheet.....	295
9.4.3 Report.....	297
9.5 IOTA.....	298
9.5.1 Findings Entry Area.....	300
9.5.2 Analysis results.....	303
9.5.3 Creating the Worksheet.....	306
10. Workflow Navigator.....	309
10.1 Outline of Workflow Navigator.....	310
10.1.1 Workflow Navigator setting.....	310
10.2 Dialog Details.....	311
10.2.1 Diastolic Dysfunction.....	311
10.2.2 Left Ventricular Hypertrophy.....	313
10.2.3 Pulmonary Hypertension.....	314
11. Measurements on a Stored Image.....	317
11.1 Measurements on ER Images.....	320
11.2 Measurements on ER Images for Which Manual Calibration Is Required.....	321
11.3 Manual Calibration Required Before the Start of Each Measurement.....	326
11.4 Worksheet Display.....	327
11.5 Accuracy of Measurement When Manual Calibration Is Used.....	328
12. Measurement Data Storage and Review Function.....	329
12.1 Storing Measurement Data in the DICOM SR Format.....	330
12.1.1 Automatic data storage at the time of termination of exam.....	330
12.1.2 Data storage using the Worksheet screen.....	331
12.2 Reviewing Measurement Data in the DICOM SR Format.....	333
12.3 Reproduction of Caliper on the Stored Image.....	334
13. Presetting.....	335
13.1 Setting Presets.....	337
13.2 Details of Each Preset.....	339
13.2.1 Common Preset.....	339
13.2.2 Labeled Preset.....	346
13.2.3 Basic Preset.....	381
13.2.4 Labeled (Details of registration functions in Measure Setup).....	383

	Page
13.2.5 Import/Export function	394
13.2.6 Initial Setting function	398
13.2.7 Copy function.....	399
13.3 Confirmation of the Automatic Measurement Start Item.....	400
13.3.1 Measurement to be started in basic measurement mode.....	400
13.3.2 Measurement to be started in application measurement mode.....	402
14. Registration of User-Programmed Measurements	405


5.3.6 Tricuspid valve measurement

Tricuspid valve measurement is performed on M-mode images.

<<Menu display on the touch panel>>




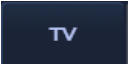

<< Measurement positions>>

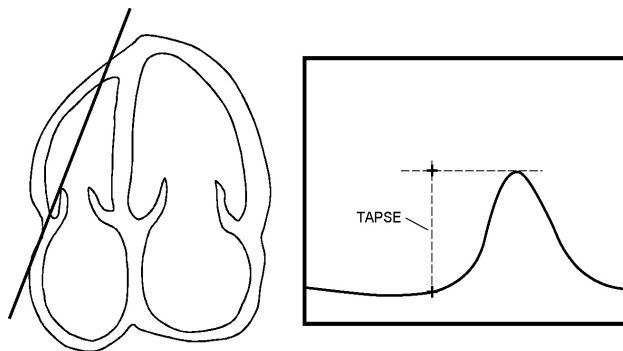

<<Measurement item>>

Switch	Item to be measured	Result display
TAPSE	Tricuspid annular plane systolic excursion	TAPSE (mm)

<<Measurement procedure>>

- (1) Press  .
- (2) Display the  page of the application measurement menu.
- (3) Press  .

Measure the Tricuspid annular plane systolic excursion using the M-mode distance measurement procedure.



LEGAL MANUFACTURER

CANON MEDICAL SYSTEMS CORPORATION

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN