
INTENDED USE

Ophthalmic Microsurgical Knives are used during ophthalmic surgery operations in order to make precise incisions, which are among the preconditions for application in delicate ophthalmic surgical interventions.

FURTHER INFORMATION

1. Construction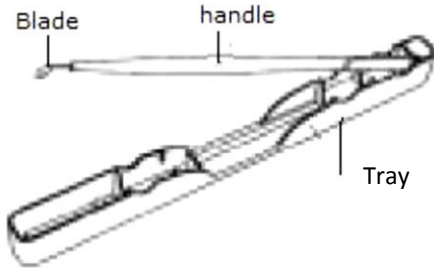

2. Materials

- Blade : Stainless steel
- Handle : Polycarbonate
- Tray : Polycarbonate

3. Packaging

Each Ophthalmic knife is individually packed in a pouch to ensure sterility. These products are sold in boxes.

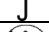
WARNINGS / PRECAUTIONS FOR SAFE HANDLING

- Read, follow and keep these Instructions for Use.
- See the intended use and use the product only in accordance with it.
- Store any new products in a dry, clean and safe place.
- Avoid high temperature, high humidity, direct sunlight, soaking and preserve at room temperature.
- Do not preserve at a place with chemical products and gas occurrence.
- Preserve package material at a non-break and non-pinhole status
- It is necessary to confirm the use period on each package and box
- By confirming the use period on each package and box, dispose the products whose use period is expired.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- If product is damaged or defective, do not use it and set it aside. Avoid contact of the delicate blades at tips with hard surfaces or other instruments.

CONTRAINDICATIONS - PROHIBITIONS

- Re-use is prohibited.
- Re-sterilization is prohibited.
- Do not use it after expiry date.
- Do not use it if product's aseptic condition is considered spoiled by broken package.

SYMBOLS AND EXPLANATIONS

	CE marking and notified-body No.		Manufactured by
	Manufacture date		Expiry date
	Sterilization by ethylene oxide		Do not re-use
	Do not re-sterilize		Lot Number
	Keep away from sunlight		Product should be kept dry
	Caution! Refer to accompanying document		Do not use if the package is damaged

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