

Anti-nRNP/Sm ELISA (IgG)

















Test instruction

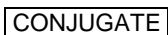ORDER NO.	ANTIBODIES AGAINST	IG CLASS	SUBSTRATE	FORMAT
EA 1591-9601 G	nRNP/Sm	IgG	Ag-coated microplate wells	96 x 01 (96)

Indications: Sharp's syndrome (MCTD), systemic lupus erythematosus.

Principles of the test: The ELISA test kit provides a semiquantitative or quantitative in vitro assay for human autoantibodies of the IgG class against nRNP/Sm in serum or plasma. The test kit contains microtiter strips each with 8 break-off reagent wells coated with nRNP/Sm. In the first reaction step, diluted patient samples are incubated in the wells. In the case of positive samples, specific IgG antibodies (also IgA and IgM) will bind to the antigens. To detect the bound antibodies, a second incubation is carried out using an enzyme-labelled anti-human IgG (enzyme conjugate) catalysing a colour reaction.

Contents of the test kit:

Component	Colour	Format	Symbol
1. Microplate wells, coated with antigens: 12 microplate strips each containing 8 individual break-off wells in a frame, ready for use	---	12 x 8	
2. Calibrator 1 200 RU/ml (IgG, human), ready for use	dark red	1 x 2.0 ml	
3. Calibrator 2 20 RU/ml (IgG, human), ready for use	red	1 x 2.0 ml	
4. Calibrator 3 2 RU/ml (IgG, human), ready for use	light red	1 x 2.0 ml	
5. Positive control (IgG, human), ready for use	blue	1 x 2.0 ml	
6. Negative control (IgG, human), ready for use	green	1 x 2.0 ml	
7. Enzyme conjugate peroxidase-labelled anti-human IgG (rabbit), ready for use	green	1 x 12 ml	
8. Sample buffer ready for use	light blue	1 x 100 ml	
9. Wash buffer 10x concentrate	colourless	1 x 100 ml	
10. Chromogen/substrate solution TMB/H ₂ O ₂ , ready for use	colourless	1 x 12 ml	
11. Stop solution 0.5 M sulphuric acid, ready for use	colourless	1 x 12 ml	
12. Test instruction	---	1 booklet	
13. Quality control certificate	---	1 protocol	
<div style="display: flex; justify-content: space-between; align-items: center;"> <div>  Lot  In vitro determination </div> <div style="text-align: center;">  </div> <div>  Storage temperature  Unopened usable until </div> </div>			

Storage and stability: The test kit has to be stored at a temperature between +2°C to +8°C. Do not freeze. Unopened, all test kit components are stable until the indicated expiry date.

Waste disposal: Patient samples, calibrators, controls and incubated microplate strips should be handled as infectious waste. All reagents must be disposed of in accordance with local disposal regulations.



Preparation and stability of the reagents

Note: All reagents must be brought to room temperature (+18°C to +25°C) approx. 30 minutes before use. After first use, the reagents are stable until the indicated expiry date if stored at +2°C to +8°C and protected from contamination, unless stated otherwise below.

- **Coated wells:** Ready for use. Tear open the resealable protective wrapping of the microplate at the recesses above the grip seam. Do not open until the microplate has reached room temperature to prevent the individual strips from moistening. Immediately replace the remaining wells of a partly used microplate in the protective wrapping and tightly seal with the integrated grip seam (Do not remove the desiccant bag).
Once the protective wrapping has been opened for the first time, the wells coated with antigens can be stored in a dry place and at a temperature between +2°C and +8°C for 4 months.
- **Calibrators and controls:** Ready for use. The reagents must be mixed thoroughly before use.
- **Enzyme conjugate:** Ready for use. The enzyme conjugate must be mixed thoroughly before use.
- **Sample buffer:** Ready for use.
- **Wash buffer:** The wash buffer is a 10x concentrate. If crystallization occurs in the concentrated buffer, warm it to 37°C and mix well before diluting. The quantity required should be removed from the bottle using a clean pipette and diluted with deionized or distilled water (1 part reagent plus 9 parts distilled water).
For example: For 1 microplate strip, 5 ml concentrate plus 45 ml water.
The working strength wash buffer is stable for 4 weeks when stored at +2°C to +8°C and handled properly.
- **Chromogen/substrate solution:** Ready for use. Close the bottle immediately after use, as the contents are sensitive to light ☀. The chromogen/substrate solution must be clear on use. Do not use the solution if it is blue coloured.
- **Stop solution:** Ready for use.

Warning: Calibration and controls used have been tested negative for HBsAg, anti-HCV, anti-HIV-1 and anti-HIV-2 using enzyme immunoassays or indirect immunofluorescence methods. Nonetheless, all materials should be treated as being a potential infection hazard and should be handled with care. Some of the reagents contain the agent sodium azide in a non-declarable concentration. Avoid skin contact.

Preparation and stability of the patient samples

Sample material: Human serum or EDTA, heparin or citrate plasma.

Stability: Patient samples to be investigated can generally be stored at +2°C to +8°C for up to 14 days. Diluted samples should be incubated within one working day.

Sample dilution: Patient samples are diluted **1:201** in sample buffer. For example: dilute 5 µl serum in 1.0 ml sample buffer and mix well by vortexing (sample pipettes are not suitable for mixing).

NOTE: Calibrators and controls are prediluted and ready for use, do not dilute them.



Incubation

For **semiquantitative analysis** incubate **calibrator 2** along with the positive and negative controls and patient samples. For **quantitative analysis** incubate **calibrators 1, 2 and 3** along with the positive and negative controls and patient samples.

(Partly) manual test performance

Sample incubation: (1st step)

Transfer 100 µl of the calibrators, positive and negative controls or diluted patient samples into the individual microplate wells according to the pipetting protocol. Incubate for **30 minutes** at room temperature (+18°C to +25°C).

Washing:

Manual: Empty the wells and subsequently wash 3 times using 300 µl of working strength wash buffer for each wash.

Automatic: Wash reagent wells 3 times with 450 µl of working strength wash buffer (program setting: e.g. TECAN Columbus Washer "Overflow Modus").

Leave the wash buffer in each well for 30 to 60 seconds per washing cycle, then empty the wells. After washing (manual and automated tests), thoroughly dispose of all liquid from the microplate by tapping it on absorbent paper with the openings facing downwards to remove all residual wash buffer.

Note: Residual liquid (> 10 µl) remaining in the reagent wells after washing can interfere with the substrate and lead to false low extinction values. Insufficient washing (e.g., less than 3 wash cycles, too small wash buffer volumes, or too short residence times) can lead to false high extinction values.

Free positions on the microplate strip should be filled with blank wells of the same plate format as that of the parameter to be investigated.

Conjugate incubation: (2nd step)

Pipette 100 µl of enzyme conjugate (peroxidase-labelled anti-human IgG) into each of the microplate wells. Incubate for **30 minutes** at room temperature (+18°C to +25°C).

Washing:

Empty the wells. Wash as described above.

Substrate incubation: (3rd step)

Pipette 100 µl of chromogen/substrate solution into each of the microplate wells. Incubate for **15 minutes** at room temperature (+18°C to +25°C) protect from direct sunlight.

Stopping the reaction:

Pipette 100 µl of stop solution into each of the microplate wells in the same order and at the same speed as the chromogen/substrate solution was introduced.

Measurement:

Photometric measurement of the colour intensity should be made at a wavelength of 450 nm and a reference wavelength between 620 nm and 650 nm **within 30 minutes of adding the stop solution**. Prior to measuring, slightly shake the microplate to ensure a homogeneous distribution of the solution.



Test performance using fully automated analysis devices

Sample dilution and test performance are carried out fully automatically using the analysis device. The incubation conditions programmed in the respective software authorised by EUROIMMUN may deviate slightly from the specifications given in the ELISA test instruction. However, these conditions were validated in respect of the combination of the EUROIMMUN Analyzer I, Analyzer I-2P or the DSX from Dynex and this EUROIMMUN ELISA. Validation documents are available on inquiry.

Automated test performance using other fully automated, open system analysis devices is possible, however, the combination should be validated by the user.

Pipetting protocol

	1	2	3	4	5	6	7	8	9	10	11	12
A	C 2	P 6	P 14	P 22			C 1	P 4	P 12	P 20		
B	pos.	P 7	P 15	P 23			C 2	P 5	P 13	P 21		
C	neg.	P 8	P 16	P 24			C 3	P 6	P 14	P 22		
D	P 1	P 9	P 17				pos.	P 7	P 15	P 23		
E	P 2	P 10	P 18				neg.	P 8	P 16	P 24		
F	P 3	P 11	P 19				P 1	P 9	P 17			
G	P 4	P 12	P 20				P 2	P 10	P 18			
H	P 5	P 13	P 21				P 3	P 11	P 19			

The pipetting protocol for microtiter strips 1-4 is an example for the **semiquantitative analysis** of 24 patient samples (P 1 to P 24).

The pipetting protocol for microtiter strips 7-10 is an example for the **quantitative analysis** of 24 patient samples (P 1 to P 24).

The calibrators (C 1 to C 3), the positive (pos.) and negative (neg.) controls, and the patient samples have each been incubated in one well. The reliability of the ELISA test can be improved by duplicate determinations for each sample.

The wells can be broken off individually from the strips. Therefore, the number of tests performed can be matched to the number of samples, minimizing reagent wastage.

Both positive and negative controls serve as internal controls for the reliability of the test procedure. They should be assayed with each test run.

Calculation of results

Semiquantitative: Results can be evaluated semiquantitatively by calculating a ratio of the extinction value of the control or patient sample over the extinction value of calibrator 2. Calculate the ratio according to the following formula:


$$\frac{\text{Extinction of the control or patient sample}}{\text{Extinction of calibrator 2}} = \text{Ratio}$$

EUROIMMUN recommends interpreting results as follows:

Ratio <1.0:	negative
Ratio ≥1.0:	positive



Quantitative: The standard curve from which the concentration of antibodies in the serum samples can be taken is obtained by point-to-point plotting of the extinction values measured for the 3 calibration sera against the corresponding units (linear/linear). Use "point-to-point" plotting for calculation of the standard curve by computer. The following plot is an example of a typical calibration curve. Please do not use this curve for the determination of antibody concentrations in patient samples.

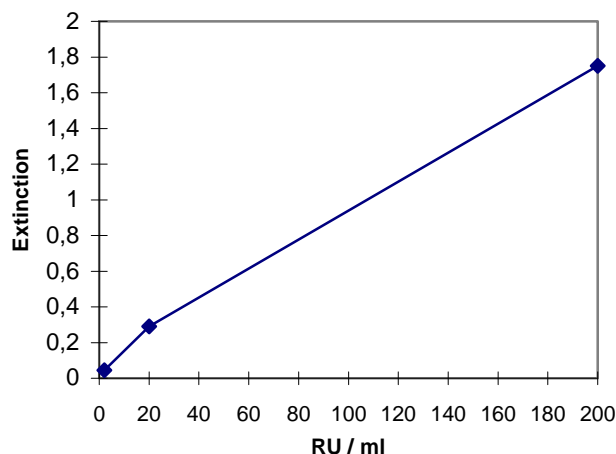

If the extinction of a serum sample lies above the value of calibrator 1 (200 RU/ml). The result should be given as ">200 RU/ml". It is recommended that the sample be re-tested at a dilution of 1:800. The result in RU/ml read from the calibration curve for this sample must then be multiplied by a factor of 4.

The upper limit of the normal range (**cut-off**) recommended by EUROIMMUN is 20 relative units (RU)/ml. EUROIMMUN recommends interpreting results as follows:

<20 RU/ml:	negative
≥20 RU/ml:	positive

For duplicate determinations the mean of the two values should be taken. If the two values deviate substantially from one another the sample should be retested.

For diagnosis, the clinical symptoms of the patient should always be taken into account along with the serological results.

Test characteristics

Calibration: As no international reference serum exists for the quantitative detection of antibodies against nRNP/Sm, the calibration is performed in relative units (RU). The reactivity of the Anti-nRNP/Sm ELISA was verified using the human reference serum CDC-ANA #4 of the "Centers for Disease Control" (Atlanta, USA).

For every group of tests performed, the extinction values of the calibrators and the relative units and/or ratio determined for the positive and negative controls must lie within the limits stated for the relevant test kit lot. A quality control certificate containing these reference values is included. If the values specified for the controls are not achieved, the test results may be inaccurate and the test should be repeated.

The activity of the enzyme used is temperature-dependent and the extinction values may vary if a thermostat is not used. The higher the room temperature (+18°C to +25°C) during substrate incubation, the greater will be the extinction values. Corresponding variations apply also to the incubation times. However, the calibrators are subject to the same influences, with the result that such variations will be largely compensated in the calculation of the result.



Antigen: The microplate wells were coated with U1-nRNP purified by affinity chromatography from calf thymus. U1-nRNP contains RNP as well as Sm reactive proteins.

Sharp and co-workers described in 1972 a new group of autoantibodies in patients with a disease described as a "mixed collagenosis" ("Mixed Connective Tissue Disease, MCTD, Sharp syndrome). All that was at first known of the associated antigen was that it contained RNA and protein (ribonucleoproteins, RNP) and that it was localised in the cell nucleus. Antibodies were then found in systemic lupus erythematosus which were directed against biochemically similar antigens (anti-Sm). These antigens belong to a group of smaller ribonucleoproteins (snRNP, small nuclear ribonucleoproteins) consisting of low molecular weight RNA with a high uridine content (U-RNA) and various proteins (molecular weights 9-70 kDa). The RNA components are described as U1 to U6 depending on their chromatographic behaviour. Apart from the individual RNA component, each U-n(nuclear)RNP particle displays six various core proteins (B, B', D, E, F, G), furthermore, U1-nRNP and U2-nRNP contain particle-specific proteins (U1-nRNP: 70 K, A, C; U2-nRNP: A', B"). The antibodies are directed exclusively against epitopes on the proteins.

Protein component		RNA component				
		U1	U2	U4	U5	U6
70K	70 kDa	●				
A	32 kDa	●				
A'	31 kDa		★			
B	26 kDa	+	+	+	+	+
B'	27 kDa	+	+	+	+	+
B''	26.5 kDa		★			
C	18.5 kDa	●				
D	13 kDa	+	+	+	+	+
E	11 kDa	+	+	+	+	+
F	11 kDa	+	+	+	+	+
G	<10 kDa	+	+	+	+	+

● = U1-nRNP reactive

★ = U2-nRNP reactive

+ = Sm reactive

Table: Composition of the various snRNP particles which react with antibodies against RNP and Sm. Antibodies against U1-nRNP are directed against one or more of the particle-specific proteins 70K, A and C. In contrast, antibodies against Sm can detect one or more of the core proteins B, B', D, E, F and G of the listed U-nRNP particles.

Linearity: The linearity of the Anti-nRNP/Sm ELISA (IgG) was determined by assaying 4 serial dilutions of different patient samples. The coefficient of determination R^2 for all sera was > 0.95 . The Anti-nRNP/Sm ELISA (IgG) is linear at least in the tested concentration range (17 RU/ml to 193 RU/ml).

Detection limit: The lower detection limit is defined as the mean value of an analyte-free sample plus three times the standard deviation and is the smallest detectable antibody titer. The lower detection limit of the Anti-nRNP/Sm ELISA (IgG) is 1 RU/ml.

Cross reactivity: This ELISA showed no cross reactivity.



Interference: Haemolytic, lipaemic and icteric samples showed no influence on the result up to a concentration of 10 mg/ml for hemoglobin, 20 mg/ml for triglycerides and 0.4 mg/ml for bilirubin in this ELISA.

Reproducibility: The reproducibility of the test was investigated by determining the intra- and inter-assay coefficients of variation using 3 sera. The intra-assay (CVs) are based on 20 determinations and the inter-assay CVs on 4 determinations performed in 6 different test runs.

<i>Intra-assay variation, n = 20</i>		
Serum	Mean value (RU/ml)	CV (%)
1	48	4.4
2	71	3.9
3	106	2.4

<i>Inter-assay variation, n = 4 x 6</i>		
Serum	Mean value (RU/ml)	CV (%)
1	41	6.3
2	66	4.8
3	105	1.9

Reference range: The levels of the anti-nRNP/Sm antibodies (IgG) were analyzed with this EUROIMMUN ELISA in a panel of 206 healthy blood donors. With a cut-off of 20 RU/ml, all blood donors were anti-nRNP/Sm negative.

Clinical significance

Antibodies (AAb) against nuclear antigens (ANA) are directed against various cell nuclear components. Among the most important nuclear antigens, including cytoplasmic antigens, are nRNP/Sm, Sm, SS-A (Ro), SS-B (La), Scl-70, PM-Scl, Jo-1, centromeres, PCNA, dsDNA, nucleosomes, histones and ribosomal P-proteins. They are mainly components of functional nuclear particles, are bound to nucleic acids or fulfil functions in the cell cycle, e.g. in transcription or translation.

The investigation of ANA and subsequent differentiation within the ANA (or ENA) spectrum contributes greatly to establishing a diagnosis, particularly in the following rheumatic diseases:

- systemic lupus erythematosus (SLE),
- Sharp syndrome (mixed connective tissue disease = MCTD),
- Sjögren's syndrome (SS),
- systemic sclerosis (SSc), and
- poly-/dermatomyositis (PM/DM).

Sharp syndrome (mixed connective tissue disease, MCTD) is a multi-symptomatic and multiform mixed connective tissue disease combining clinical symptoms of rheumatoid arthritis, SLE, systemic sclerosis, CREST syndrome (calcinosis cutis, Raynaud's phenomenon, oesophagus motility disorders, sclerodactyly, teleangiectasia) and vasculitides.

High **anti-nRNP/Sm** titers are characteristic for Sharp syndrome, whereby the titer correlates with the disease activity. Anti-nRNP/Sm antibodies are also detected in patients with SLE, SSc and PM/DM.

Antibodies against	Disease	Prevalence
nRNP/Sm	Sharp syndrome (MCTD)	95% -100%
	Systemic lupus erythematosus (SLE)	3% - 47%
	Systemic sclerosis (SSc)	2% - 14%
	Polymyositis/dermatomyositis (PM/DM)	12% - 16%
	Overlapping polymyositis/SSc	approx. 24%

Literature references

1. Benito-Garcia E, Schur PH, Lahita R; American College of Rheumatology Ad Hoc Committee on Immunologic Testing Guidelines. **Guidelines for immunologic laboratory testing in the rheumatic diseases: anti-Sm and anti-RNP antibody tests.** Arthritis Rheum 51 (2004) 1030-1044.
2. EUROIMMUN AG. Stöcker W, Schlumberger W, Krüger C. **Alle Beiträge zum Thema Autoimmundiagnostik.** In: Gressner A, Arndt T (Hrsg.) Lexikon der Medizinischen Laboratoriumsdiagnostik. 2. Auflage. Springer Medizin Verlag, Heidelberg (2012).
3. Hartung K, Seelig HP. **Laboratory diagnostics of systemic autoimmune diseases. Part 1. Collagenoses.** [Article in German] Z Rheumatol 65 (2006) 709-724.
4. Houtman PM, Kallenberg CG, Limburg PC, Huitema MG, van Rijswijk MH, The TH. **Quantitation of antibodies to nucleoribonucleoprotein by ELISA: relation between antibody levels and disease activity in patients with connective tissue disease.** Clin Exp Immunol 62 (1985) 696-704.
5. Houtman PM, Kallenberg CG, Limburg PC, van Leeuwen MA, van Rijswijk MH, The TH. **Fluctuations in anti-nRNP levels in patients with mixed connective tissue disease are related to disease activity as part of a polyclonal B cell response.** Ann Rheum Dis 45 (1986) 800-808.
6. Phan TG, Wong RC, Adelstein S. **Autoantibodies to extractable nuclear antigens: making detection and interpretation more meaningful.** Clin Diagn Lab Immunol 9 (2002) 1-7.
7. Satoh M, Langdon JJ, Hamilton KJ, Richards HB, Panka D, Eisenberg RA, Reeves WH. **Distinctive immune response patterns of human and murine autoimmune sera to U1 small nuclear ribonucleoprotein C protein.** J Clin Invest (1996) 2619-2626.
8. Tan EM, Chan EK, Sullivan KF, Rubin RL. **Antinuclear antibodies (ANAs): diagnostically specific immune markers and clues toward the understanding of systemic autoimmunity.** Clin Immunol Immunopathol 47 (1988) 121-141.
9. Varga J. **Systemic sclerosis: an update.** Bull NYU Hosp Jt Dis 66 (2008) 198-202.