
CONAXX 2

CONAXX 2 Veterinarian Version

CE 0297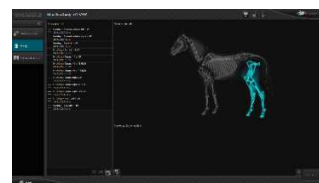

**made
in
Germany**

Please note:

The CE marking is an EU directive compliance mark and may only be affixed to products for which a directive applies, which provides the CE marking.

At PROTEC these are medical devices that are intended for use in human medicine. For the identical products which are intended for use in veterinary medicine, the CE marking is excluded.

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Product Features

Automatic image optimization

The appearance of incorrect exposures can be noticeably reduced through automatic image optimization. This is achieved through automatic adjustment of over and under exposed images and the optimal display of body part specific structures. This is further improved by an increase in contrast to a reduction in noise.

Furthermore the Professional Image Tuning (PIT) function enables the technician and the physician the most effective individual image adjustment. All kinds of body parts can be modified according to the taste of the physicians. This fine tuning function is easy and comfortable to use and all modified adjustments – e.g. grey tone values, contrast, sharpness - can be automatically processed on the corresponding body parts.

Three clicks only to get your X-ray image

Due to a well-structured layout of the automatic transfer of patient and assignment data as well as context oriented adaption of the surface, the user in different operation methods of CONAXX 2 will always end up at the X-ray image fast and intuitive.

Image diagnose possible directly in CONAXX 2

With the optional CONAXX 2 Diagnostic Viewer the image processing and the image archiving is optimal and completely integrated into the single workplace solution. No additional archiving system (PACS) is necessary and consequential costs for additional software, hardware or service can be saved.

Compatible with all DICOM PACS

By the use of the defined DICOM standard it is easily possible to define CONAXX 2 as the software for the image acquisition and the image processing with an already existing PACS-solution.

Extraordinary efficiency saves time and money

The well-structured user surface is adapting itself to various operation methods and functions for an efficient use. Due to the modular structure of CONAXX 2 it can be adjusted to changing terms of a complex radiologic network anytime. The advantage of the single workplace solution is that no additional archiving system (PACS) is necessary.

Technical Product Specifications

<i>Supported languages</i>	English, German, French, Czech, Hungarian, Spanish, Polish, Turkish, Chinese, Italian, Slovenian, Lithuanian
<i>Interfaces</i>	DICOM Media Storage DICOM MPPS (SCU) DICOM Print Management (SCU) DICOM Query/Retrieve (SCU) DICOM Storage (SCU) DICOM Storage Commitment (SCU) DICOM Verification DICOM-Worklist (SCU) GDT/BDT
<i>DICOM Conformance</i>	Standard 3.0
<i>Standard modules</i>	Acquisition AIP (advanced image progressing) DICOM store Email GDT/BDT interface (not in veterinarian version)
<i>Additional modules</i>	DICOM Worklist DICOM Query DICOM Print Dual Panel GDT/BDT/EMR Gridline Suppression Patient CD X-ray Generator Connection Automatic Collimator X-ray Journal Stitching / Stitching Autotracking
<i>Basic functions</i>	Windowing, Magnifier, Rotating, Free Rotating, Flipping, Cropping, Annotations, ...
<i>Special features</i>	Intuitive user interface, intelligent automatic backup function, intelligent database purging function, automatic image optimization steps adjustable for each body part, program in window or full screen mode, varying filter levels selectable
<i>Number of organ programs (APR)</i>	No limitation
<i>Single workstation PACS option</i>	CONAXX 2 Diagnostic Viewer
<i>Additional functions Diagnostic Viewer</i>	Region windowing, measuring, electronic shutter, additional filters, tiling, full screen, zooming, quick access bar, histogram, ...

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