	ONECARE Incontinence Products Instruction For Use <i>ONECARE İnkontinans Ürünleri Kullanım Kılavuzu</i>			
	Document Number	Publication date	Revision No	Revision date
	TD.11.121	06.05.2024	00	.../.../...

ONECARE INCONTINANCE PRODUCTS INSTRUCTION FOR USE

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Description and Purpose of Use

ONECARE Incontinence Products are used to create a barrier layer to protect the incontinence related skin of patients with special needs, such as those with reduced mobility, bedridden and intensive care patients, by cleaning urine and feces contact from external factors. It prevents skin damage that may occur due to pH change and irritation in the body.

Mechanism Of Action

There is a barrier layer on the upper surface of healthy skin that protects the skin. However, the use of excessively basic products damages this layer. As a result, the skin becomes open to all threats. It has been determined that using products close to the pH level of the skin does not cause any irritation on the skin. In order for the skin in these areas not to become irritated and open to wounds, products specially designed for this purpose should be used. Thanks to its pH compatible with the skin, our products ensure the maintenance of the protective barrier layer on the skin.

Indication

Intensive care units, inpatients or people who cannot care for themselves

Contraindication

The products have no known contraindications.

Patient Population

It can be used in adult, infant and pediatric patients.

Product Life

Products are for single use and product shelf life is 2 years.

Manufacturer-Identified Warnings and Precautions

Do not use products with damaged packaging. Do not use as a tampon. Do not reuse the used product. It should be used for 0-6 months babies, breastfeeding and pregnant women under the supervision of a doctor. No other cleaning liquid or substance should be used before and after using the product. Avoid contact with eyes. If it does, wash it off with plenty of water. Keep out of reach of children. It is for external use only. Not intended for skin disinfection. It is suitable for use by a single patient only.

PRECAUTIONS: Do not get wet. Close the package tightly after use. If the pack or gloves are soiled, discard the remaining pad wipes.

	ONECARE Incontinence Products Instruction For Use <i>ONECARE İnkontinans Ürünleri Kullanım Kılavuzu</i>			
	Document Number	Publication date	Revision No	Revision date
	TD.11.121	06.05.2024	00	.../.../...

Instruction manual

Open the package without removing the label. Take out the dirty laundry. Clean the contaminated area using a pad wipe, and the perineal area using a perineal pad. Repeat this action using a new tissue until the affected area is clean.

Use with microwave:

- Place the package in the microwave oven and heat at 700 Watt for 20 seconds.
- Use it by controlling the temperature.
- Do not use at temperatures above 35-40°C.

Sterility

Our Products Are Offered For Sale As Non-Sterile.

Radioactivity

Our Products Do Not Have Radioactive Properties.

Condition of Product Containing Any Drug, Animal Origin, or Blood

Our product groups do not contain any drugs, animal origin or blood.

User Profile

Person or persons who will use our product;

- a) Persons who are competent to read the information in the user manual and use the product
- b) Patient caregivers
- c) Nurses

Precautions and Warnings

- Be sure to read the instructions for use before using the product.
- Avoid contact with eyes.
- IF SWALLOWED: Rinse your mouth, do not try to vomit.
- In case of skin irritation with the contact, wash with plenty of soap and water. Get medical attention if skin irritation occurs.
- Do not reuse the packaging of the finished product.
- IF IN EYES: Rinse carefully with water for a few minutes. In case of eye irritation, consult a doctor.
- Avoid spreading to the environment.
- Store in its original container.
- Dispose of waste and used packaging in accordance with official regulations

	ONECARE Incontinence Products Instruction For Use ONECARE İnkontinans Ürünleri Kullanım Kılavuzu			
	Document Number	Publication date	Revision No	Revision date
	TD.11.121	06.05.2024	00	.../.../...

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KAF GRUP SAĞLIK HİZMETLERİ İNŞ. SAN. VE TİC. LTD. ŞTİ.