

Complete Bathing Solution™
SHAMPOO



PRE-MOISTENED SHAMPOO WASH GLOVE



Hair care has equal importance to skin care in bathing a patient and from the patient's perspective may even have more significance. Cleanis® developed the AQUA® pre-moistened Shampoo Glove eliminating the need for basin- or bowl- shampooing. Studies have shown when hospitals switched from bath basins to disposable washing products, the risk of nosocomial infections substantially declined.^{1,2,3,4}

Using the AQUA® Shampoo Wash Gloves also removes the need for any handling of water.^{1,2} The AQUA® Shampoo Wash Glove is a two-sided, non-woven, soap-free and rinse-free glove containing a protective washing solution that is gentle to the hair and scalp. The scientific formula is impregnated with a hypoallergenic cleansing agent that removes excess sebum and buildup. One size fits most hands.

- Pre-moistened single-use shampoo glove
- Ready-to-use, waterless shampooing
- Protective plastic liner inside glove
- Gentle and non-irritating to skin or eyes
- Eliminates soap, shampoo, water, basins, rinsing and towels
- Saves time and reduces costs associated with traditional bathing and shampooing
- Eliminates clean-up and disinfection of "accessories" after shampooing
- Hypoallergenic and dermatologically tested (clinical tests available on request)
- Does not pill or shed particulates
- Reduces water consumption
- Reduces laundry costs

Ideal for Use In:

- Nursing homes
- Home health care
- Patients who are reluctant to shampooing
- Patients who have trouble bending over basins or sinks
- Geriatrics: when a patient is too weak for standard shampooing methods
- Oncology: avoids contaminated linen for patients having chemotherapy
- Intensive or palliative care: when other bathing methods are impractical

1. Emmerson A. Emerging waterborne infection in health-care settings. *Emerg Infect Dis.* 2001; 7:272-276.

2. Clark AP, John LD. Nosocomial infections and bath water: any cause for concern? *Clin Nurse Spec.* 2006; 20:119-123.

3. Skewes SM. Skin care rituals that do more harm than good. *Am J. Nurs.* 1996; 96(10):33-35.

4. Cervia JS, Canonica F, Ortolano G. Water as a source of healthcare-associated infections [letter]. *Arch Intern Med.* 2007; 167:92.

ated practice parameter. *Ann Allergy Asthma Immunol.* 2004; 93: S1-S21.



AQUA® SHAMPOO

Pre-Moistened Wash Glove

Product Description

Gentle, rinse-less shampoo! A ready-to-use, two-sided pre-moistened, non-woven, rinse-free glove impregnated with a protective washing solution that is hypoallergenic and dermatologically tested. The AQUA® Shampoo Wash Glove is alcohol free, contains no soap and requires no rinsing or drying. It is equipped with a polyethylene liner on the inside. The AQUA® Shampoo Wash Glove is ideal for bedridden, incontinent and dependent patients and can be used anywhere.

Benefits

- Single-use, prevents cross-contamination
- Saves time and reduces costs associated with traditional shampooing; significant reduction of cleansing time required for patients
- Avoids the cleanup/disinfection of conventional shampoo accessories
- Avoids wet bed linens and wet patient gowns
- Hypoallergenic shampoo solution is dermatologically tested
- Safe for all skin types including newborns and geriatric patients
- Allows easy shampooing of bedridden patients
- No water or towel required

Instructions for Use

1. Open the pack by peeling down from the top of label.
2. Remove one AQUA® Shampoo Wash Glove and reseal the label.
3. Place the AQUA® Shampoo Wash Glove on hand.
4. Shampoo the patient following your local SOP for proper hygiene. Both sides of the glove may be used.

Disposal

- When shampooing is complete, dispose of used gloves according to standard institutional operating procedures.
- Do NOT flush AQUA® Shampoo Wash Glove. Discard in regular waste container according to facility infection control guidelines.

Precautions

Single-use product. Do NOT reuse (ⓧ)
Do not use on open wounds
DO NOT FLUSH the AQUA® Wash Glove

Product Composition

Internal protection film: Polyethylene/PET
Glove: 60% PET, 40% Viscose
Pre-moistened with Glycerin, Aloe Barbadensis Leaf Extract, Cocam Idopropyl-Betaine, Phenoxyethanol, Ethylhexylglycerine, Benzoic Acid, Dehydroacetic Acid, Sodium Benzoate, Citric Acid.

Identification & Expiration Date

Each pack is identified by a reference/lot number and expiration date (3 years)

Safety Data

- This product is not considered dangerous according to the European Union regulation
- MSDS sheets available upon request

Conformity

- Compliant with the 93/42/CEE regulation for medical device – Class I
- Customs code: 3401190000
- Company certification: ISO 9001:2008 and ISO 13485:2003

Reorder Number: 9929200

Unit	Pouch of 12 gloves	0.68 lbs (309 gr)
Case	12 boxes of 12 glove pouches (total of 144 gloves)	11 ³ / ₄ " (L) x 9 ³ / ₈ " (W) x 7 ⁹ / ₁₆ " (H); 9.04 lbs
Pallet	104 cases (1,248 pouches, total of 14,976 gloves)	31 ¹ / ₂ " (L) x 47 ¹ / ₄ " (W) x 74 ¹³ / ₁₆ " (H); 992 lbs

Also available in a pack of 2 gloves: Reorder Number 9929217



1375 Broadway Avenue, Floor 11, Suite 1100, New York, NY 10018
Website: www.Cleanis.com • Phone: 646-278-5627 • Fax: 646-253-1258