



FemoSeal[™]

Vascular Closure Device


ACTIVE
**VASCULAR
CLOSURE**

Now, by  **TERUMO**
INTERVENTIONAL
SYSTEMS

FemoSeal™

Vascular Closure Device

HOW TO USE

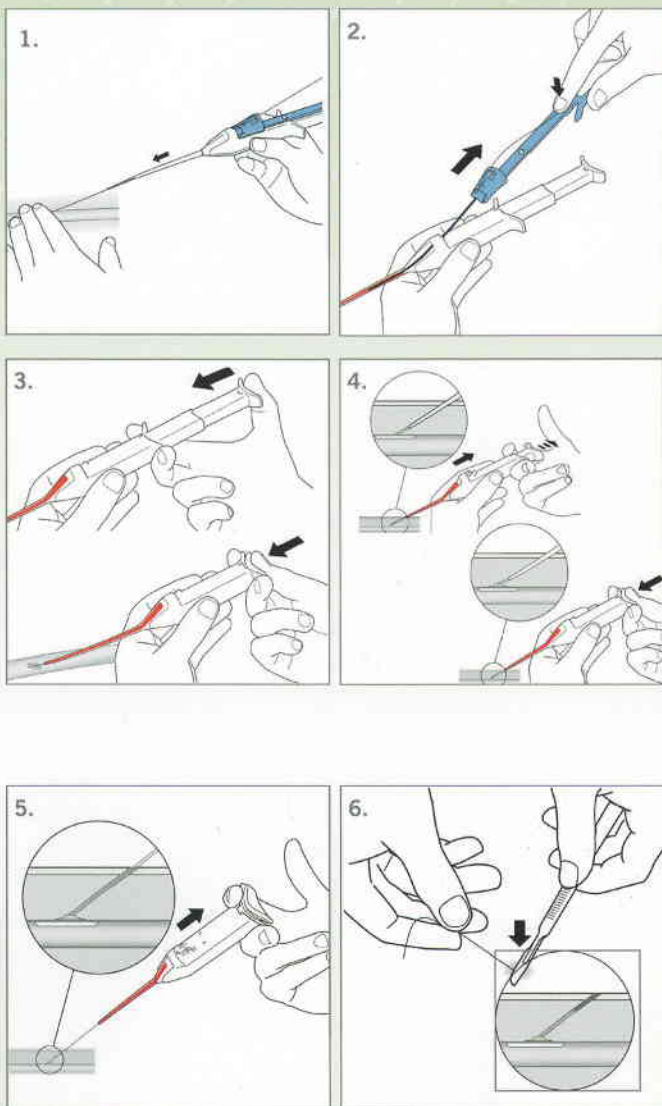

LOCATE THE ARTERY¹

- Thread the FemoSeal device over the guidewire and insert the sheath and dilator into the artery
- Detach the safety catch with dilator.

When the FemoSeal **safety catch** with dilator has been retracted a few centimeters (2-4cm), blood should appear in the proximal part of the sheath, verifying that the sheath is in the artery.

SEALING THE FEMORAL ARTERY

- Deploy the **inner disc** by pressing down the button completely in one continuous movement. Pull back on the FemoSeal device until the depressed button springs back.
- The **inner disc** establishes hemostasis. Maintain suture tension while deploying the outer locking disc. Deploy the disc by completely pressing down the button again.

VERIFYING HEMOSTASIS

- Remove the FemoSeal device. Verify hemostasis.
- After verification, cut the suture below skin level using a sterile instrument. Clean puncture site and apply a sterile dressing so that it can easily be observed during recovery.

ORDERING INFORMATION

Product code	French size	Guidewire diameter (in)
C11202	7	0.038

¹ Assess puncture site location by fluoroscopy.

TERUMO
INTERVENTIONAL
SYSTEMS

Terumo Corporation
+81333748111

Terumo Europe NV
+3216381211

Terumo
Interventional Systems EMEA
+33147160930

EMEA Sales Offices

Terumo Europe NV
Africa Business Division
+3216381308

Terumo Europe NV
Benelux Sales Division
Belgium:
+3216392580
The Netherlands:
0800 0220396

Terumo Europe NV
Emerging Market Division
+3216381211

Terumo Deutschland GmbH
+49 6196 80 230

Terumo Deutschland GmbH
Österreich
+43 2236 378020

Terumo Deutschland GmbH
Switzerland
+41 56 419 10 10

Terumo Europe España SL
+34 902 10 12 98

Terumo France S.A.S.
+33 130 96 13 00

Terumo Italia S.r.l.
+39 06 94 80 28 00

Terumo Russia LLC
+7 495 988 4740

Terumo Sweden AB
+46 3174 85 880

Terumo Middle East FZE
+971 4 292 0200

Terumo UK Ltd
+44 1276 480 440

Terumo BCT Tıbbi Cihazlar Dağıtım
ve Hizmetleri A.Ş.
+90 216 590 0074



© Registered Trademark.
Published by Terumo Europe NV

www.terumo-europe.com