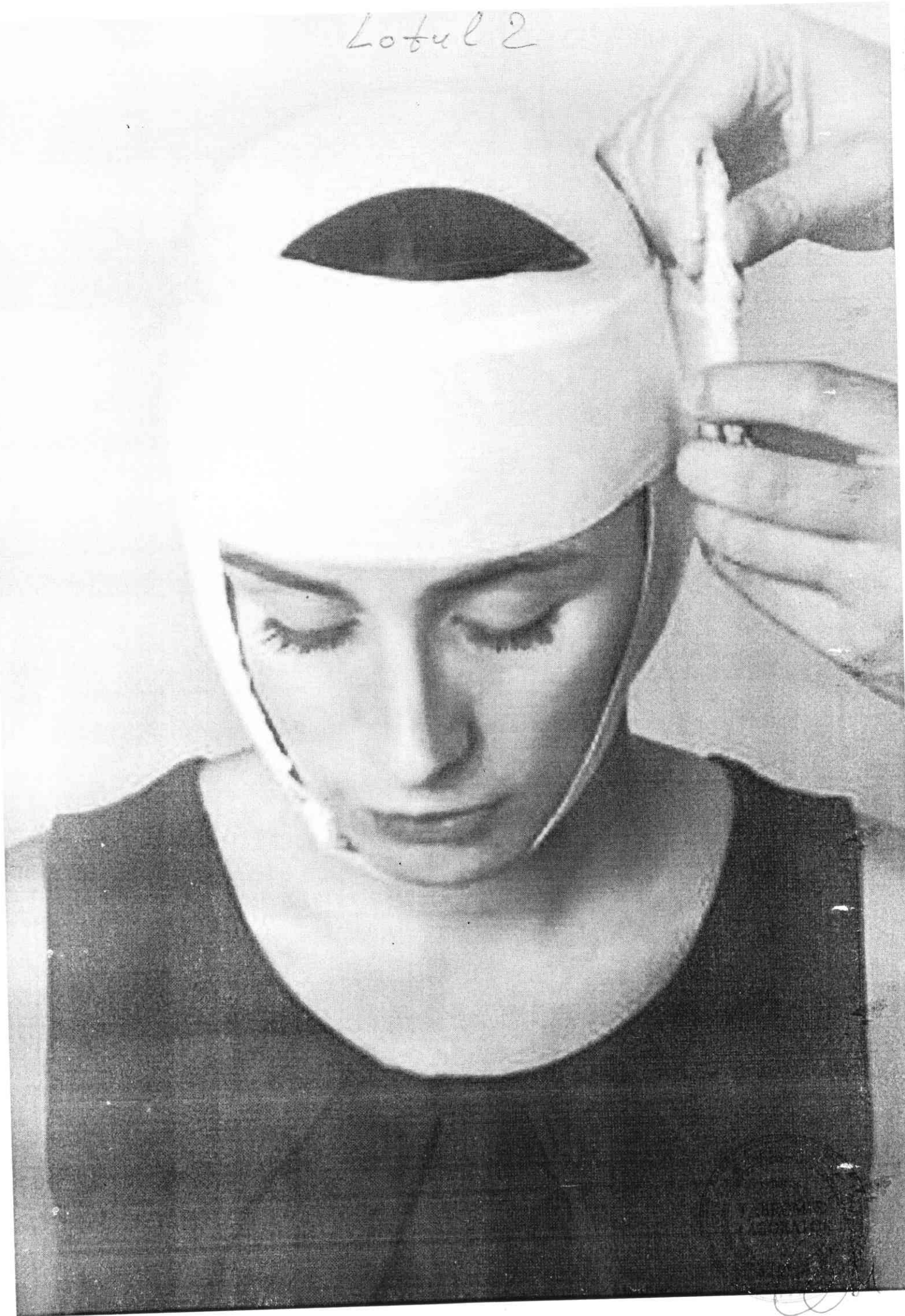
Lotul 2

lotul 2

First Care Products is active in the development and marketing of innovative products for pre-hospital emergency medical situations.
 The company's mission is to bring to market products that improve treatment while providing significant savings in time and resources to armed forces, homeland security, police, fire departments, first aid providers, EMS, and civilians worldwide.

www.firstcareproducts.com

First Care Products Ltd.
 10 Amal St., Afek Industrial Park,
 Rosh Ha Ayin Zip 4809234, Israel
 Tel: +972 9 9517444
 info@firstcareproducts.com

Blizzard Protection Systems Ltd.
 Unit 2, Coed-y-Parc Industrial Estate
 Bethesda, Bangor, Gwynedd, LL57 4YV,
 United Kingdom
 Tel: +44 (0) 1248 600666

304000022 Rev.3 01.2016

First Care WoundStop First-Aid Trauma Bandage

EN

Description of the product:

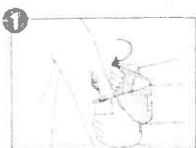
First Care WoundStop Bandage is an all-in-one easy-to-use sterile first-aid dressing that stops bleeding fast. The bandage is designed for use at home, at work, outdoors and anywhere accidents can occur.
 First Care WoundStop Bandage controls bleeding and reduces the possibility of the onset of shock by applying focused and safe direct pressure to the wound.
 First Care WoundStop Bandage is designed for flexibility & ease of use. It can be applied by anyone on all parts of the body to control light to moderate bleeding and to dress wounds and abrasions.

Warnings:

- Make sure that the person injured always receives medical attention as soon as possible.
- Do not touch the pad of the bandage which comes in direct contact with the wound.
- Avoid contact with blood and other bodily fluids.
- For external use only.
- WoundStop is not a tourniquet!
- If bleeding persists, exert further direct pressure on the wound and call for assistance and professional medical care.

Directions of use:

Storage:


Place pad on wound & wrap the elastic bandage around limb or body part.

Tighten elastic bandage and...

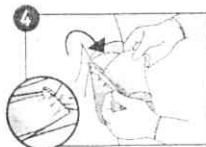Wrap elastic bandage tightly over all edges of dressing pad.

Secure hooking ends of closure bar into elastic bandage.

Keep First Care WoundStop Bandage in original package. Store at room temperature, in a dry environment.

If you have any questions or comments, please contact First Care Products at: info@firstcareproducts.com

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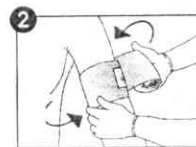
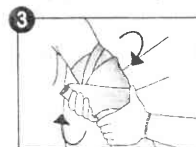
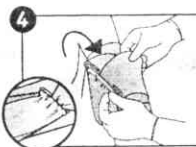
Lotus 2

FirstCare
A Persys Medical Company

WoundStop Care1


10cm / 4" Trauma Dressing

www.firstcaremedical.com

10cm / 4" Eerste Hulp Traumawachtel	NL
10cm / 4" Trauma Bandage	D
10cm / 4" Pansement compressif Trauma	F
10cm / 4" Banda per ferite Traumatische	I
10cm / 4" Opstruuk anatomisch Trauma Dressing	PL
10cm / 4" Vendaje para Trauma	E


First Care Products Ltd, 11 Gungah Hill Road, St. Helens, Merseyside, L35 9DF, UK
 Tel: +44 (0) 151 342211 | info@firstcareproducts.com | www.firstcareproducts.com
 Advanced Protection Systems Ltd, Unit 2, Lead & Pine Industrial Estate, Bethesda, Bangor,
 Gwynedd, LL57 4TY, United Kingdom | Tel: +44 (0) 1248 500046 | Fax: +44 (0) 1248 500044

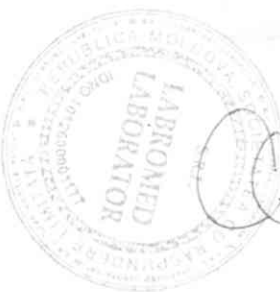Handwritten signature


L.S.

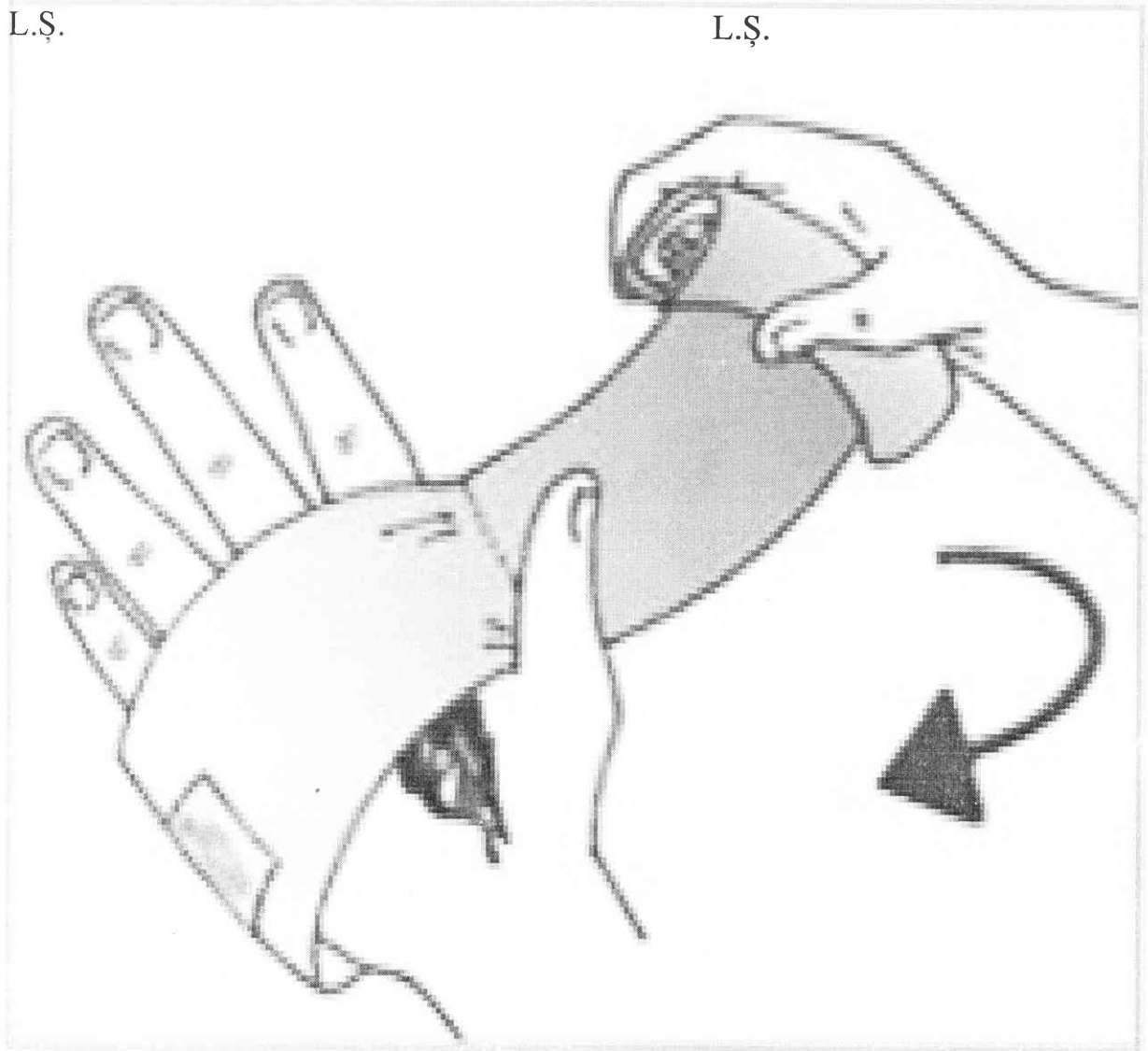L.S.

the wounded hand inside the loop of the bandage to stabilize bandage. Position it so that the bandage will rest on the wound area. Use your other hand to wrap the elastic bandage.

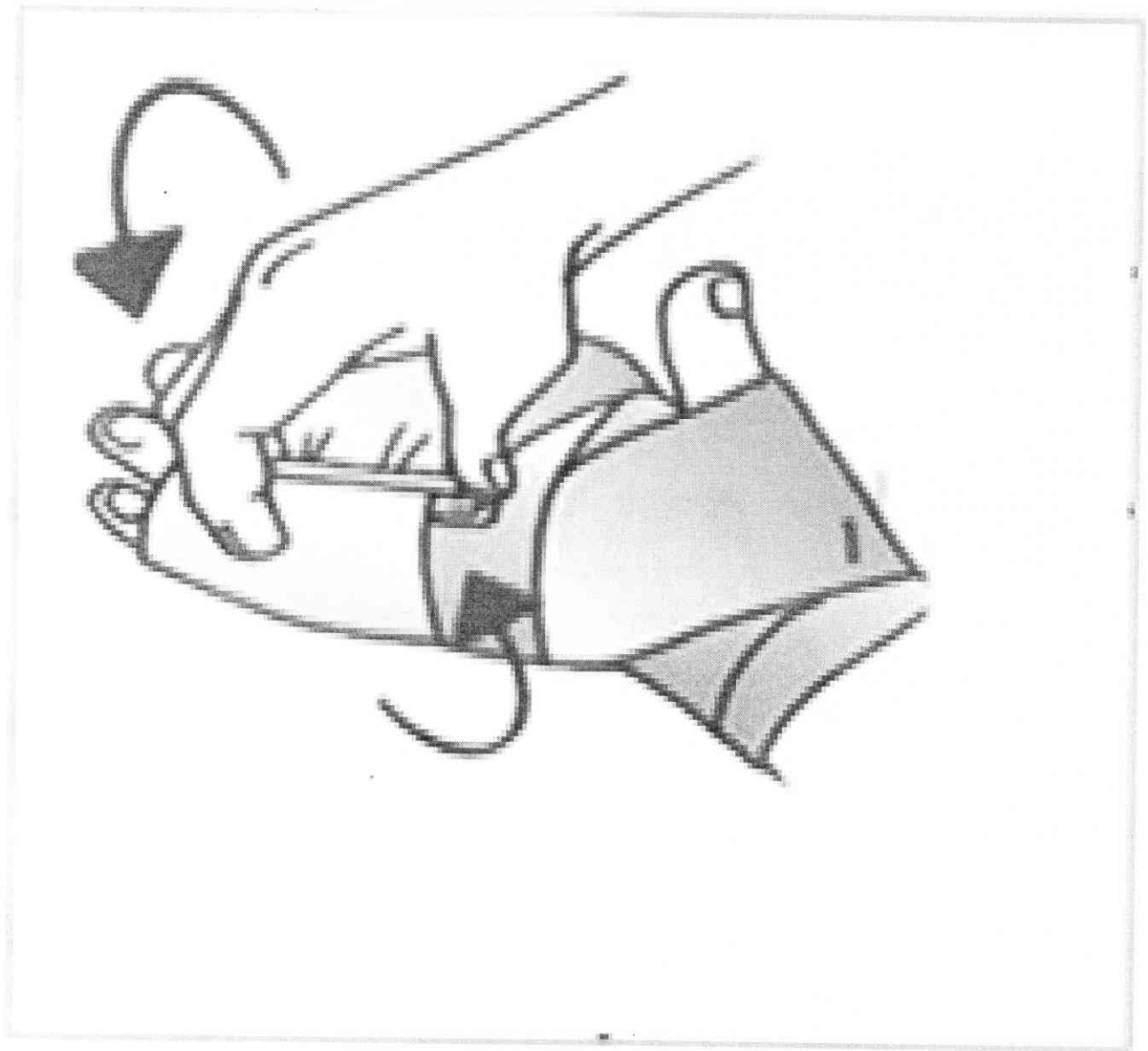Lotul 2

hooking ends of closure bar into elastic
le.