Iris Diluent

REF 800-3202

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	22
中文 (ZH-CN)	24
Lietuviškai (LT)	26
Magyar (HU)	28
Polski (PL)	30
Čeština (CZ)	32
Slovenčina (SK)	34
한국어 (KO)	36
Türkçe (TR)	38
Русский (RU)	40
eesti keel (ET)	43
Hrvatski (HR)	45
Български (BG)	47
中文 (ZH-TW)	49
Română (RO)	51
Србија (SR)	53
Latviešu (LV)	55
Українська (UK)	57
Português Brasil (PT-BR)	59
Tiếng Việt (VI)	61

Iris Diluent

REF 800-3202

Rx Only in the U.S.A.

INTENDED USE

Iris Diluent is intended for *in vitro* Diagnostic Use with the iQ200 or DxU 840/850m Series of Automated Urine Microscopy Analyzers.

INTENDED USER

The device is intended to be used by a laboratory professional.

PRINCIPLE

Iris Diluent is a particle free aqueous salt solution which is used for diluting urine and body fluid samples. It is also used to rinse the system to prevent carryover of system cleanser and samples.

WARNING AND PRECAUTIONS

- Dispensed material should not be re-used.
- Iris Diluent has been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910.1200(d).
- Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly.
- Wear appropriate personal protective equipment when handling.
- Product should present as a clear, colorless, odorless liquid. Product that differs from these attributes should be discarded as they may have deteriorated.
- Consult local and state authorities with regard to recommended method of disposal

GHS HAZARD CLASSIFICATION

Not classified as hazardous

SDS Safety Data Sheet is available at beckmancoulter.com/techdocs

NON-REACTIVE INGREDIENTS

- Water
- Salts
- Preservatives

STORAGE AND STABILITY

- Store at 20°C to 28°C (68°F to 82.4°F).
- Iris Diluent is stable for twelve months from date of manufacture.
- Open expiration: Unused solutions that remain in the original capped container are stable until the expiration date.
- Do not freeze.
- Do not store in direct sunlight.
- Do not use solutions after the expiration date printed on the bottle label.

MATERIALS PROVIDED

- 4 X 475 mL Iris Diluent
- 4 Flip-Top Caps

MATERIALS REQUIRED, BUT NOT PROVIDED

- Approved tubes listed in the iQ200 or DxU 840/850m Instructions for Use. Do not use preservative tubes.
- Personal Protective Equipment.

PROCEDURE

Follow the directions in the iQ200 or DxU 840/850m Instructions for Use.

TECHNICAL SUPPORT

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-854-3633 (USA or Canada) or contact your local Beckman Coulter Representative.

A Glossary of Symbols (PN C27648) is available at beckmancoulter.com/techdocs.

NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see www.beckmancoulter.com/patents.

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Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100