

Illustration

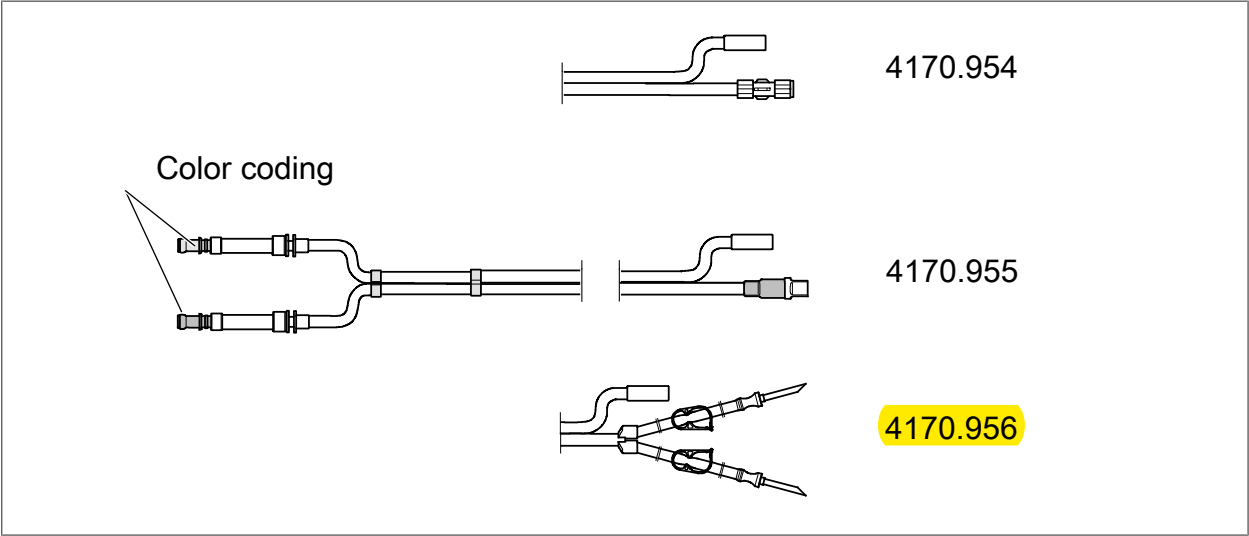














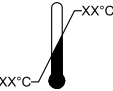
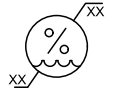





Fig. 1

Item number	Description
4170.954	<b>Irrigation tube:</b> With Luer-Lock, for connection to the irrigation tube of a suction and irrigation tube set, e.g., irrigation tube of the Endo-Irrigator 2211 <b>Suction tube:</b> For connection to a collection or secretion container and/or vacuum pump. <b>Tube length:</b> 3 meters
4170.955	<b>Irrigation tube:</b> With SAFE-LOCK®, for connection to an irrigation container. <b>Suction tube:</b> For connection to a collection or secretion container and/or vacuum pump. <b>Tube length:</b> 3 meters
4170.956	<b>Irrigation tube:</b> With 2 piercing spikes, for connection to an irrigation container. <b>Suction tube:</b> For connection to a collection or secretion container and/or vacuum pump. <b>Tube length:</b> 3 meters

8.1 Labeling

Symbols	Designation
	Follow the manual
	Caution
	Product number
	Lot code
	Manufacturer
	Manufacturing date
	Number, amount

Symbols	Designation
	Expiry date
	Sterilized with ethylene oxide
	Single sterile barrier system with protective packaging inside (sterilized using ethylene oxide)
	Do not reuse!
	Do not resterilize
	Keep away from sunlight!
	Store in a dry place!
	Temperature, limitation
	Humidity, limitation
	Do not use if package is damaged
	Contains phthalates or phthalates are present: diethylhexylphthalate (DEHP)
	CE marking in accordance with Medical Devices Directive 93/42/EEC. <b>Applies only</b> if the <b>product and/or packaging bears this marking</b> . Products in Class IIa and higher and sterile products or products with a measuring function in Class I are also marked with the four-digit identification number of the notified body.

## Operating, storage, transport, and shipping conditions

Disposable items, sterile products	Follow the instructions on the packaging!
------------------------------------	---

### ATTENTION

Store sterile products in their original packaging until use.  
Improper storage can lead to loss of sterility.

### ATTENTION

The relevant regulations and laws valid in the country of use must be observed when disposing of the product.  
▷ For further information, please contact the manufacturer.

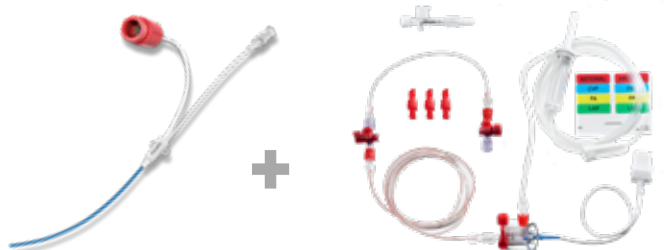
## 3.5 PiCCO Kits

PiCCO Kits consists of:

PiCCO Catheter



Monitoring Kit



Additional information about the PiCCO Catheter see chapter 3.1; page 11

Additional information about the Monitoring Kits see chapter 3.2; page 12

PiCCO Catheter		Monitoring Kit	REF	Getinge order #
<b>PV2015L20-A</b> <b>6885049</b> Ø: 5 French Usable length: 20 cm	+	<b>PV8215 / 6882817</b> Pressure transducer & 150 cm Arterial Pressure Line; Injectate Temperature Sensor Housing (6882773)	PVPK2015L20-A 5 pieces	<b>6885060</b> 1 purchase unit
<b>PV2013L07-A</b> <b>6885044</b> Ø: 3 French Usable length: 7 cm	+	<b>PV8215 / 6882817</b> Pressure transducer & 150 cm Arterial Pressure Line; Injectate Temperature Sensor Housing (6882773)	PVPK2013L07-A 5 pieces	6885055 1 purchase unit
<b>PV2014L08-A</b> <b>6885045</b> Ø: 4 French Usable length: 8 cm	+	<b>PV8215 / 6882817</b> Pressure transducer & 150 cm Arterial Pressure Line; Injectate Temperature Sensor Housing (6882773)	PVPK2014L08-A 5 pieces	6885056 1 purchase unit
<b>PV2014L16-A</b> <b>6885046</b> Ø: 4 French Usable length: 16 cm	+	<b>PV8215 / 6882817</b> Pressure transducer & 150 cm Arterial Pressure Line; Injectate Temperature Sensor Housing (6882773)	PVPK2014L16-A 5 pieces	6885057 1 purchase unit
<b>PV2014L22-A</b> <b>6885047</b> Ø: 4 French Usable length: 22 cm	+	<b>PV8215 / 6882817</b> Pressure transducer & 150 cm Arterial Pressure Line; Injectate Temperature Sensor Housing (6882773)	PVPK2014L22-A 5 pieces	6885058 1 purchase unit

