



## INSTRUMENT WASHER- DISINFECTORS AND TUNNELS

### + PRODUCT

DETERGENT  
PERFORMANCE

CONCENTRATED  
FORMULA

FROM  
4 MIN.

### GOOD PRACTICES



Product file available on  
[www.exeol.fr/en](http://www.exeol.fr/en)

# exeol clean matic

## Non-foaming alkaline detergent

► Concentrated solution for cleaning of surgical instruments and accessories in Washer-Disinfector or tunnel washer before sterilisation.

- Validated detergency performance on the main cleaning indicators of the market (e.g. TOSI® and Browne®).
- EDTA-free, NTA-free and phosphate free.
- Compatible with stainless steels according to standard NF S94-402-1 (May 2004) and materials that are not sensitive to slightly alkaline pH levels (pH 10).



### Instructions for use

**exeol clean matic** is a concentrated solution.

When using for the first time, before unscrewing the cap, remove the tamper-proof ring. Place the container in or near the washer-disinfector and insert the sampling tube into the container for the washing phase. If changing the product on the washer-disinfector, first drain and rinse the hoses and dosing system using clean water. Do not recycle the washing solution.

**exeol clean matic** should be injected during the washing phase.

Dilution between 0.3% (3mL/L) and 0.5% (5mL/L) at a temperature of 55°C from 4 minutes. The optimal dose is dependent on site specificities (water quality, type of washer-disinfector and equipment to be treated, level of soiling).

Then use **exeol rinse matic** for the thermal disinfection phase (final rinsing).

Follow the instructions given by the manufacturers of the medical devices to be treated and the washer-disinfector.

Composition	Characteristics	Cautionary notes
Anionic and non-ionic surfactants, sequestering agents and excipients.	<ul style="list-style-type: none"><li>▪ Pure pH: 9.7 - 10.5</li><li>▪ pH at 0.5%: 9.5 - 10.0</li><li>▪ Fragrance: Unscented</li><li>▪ Colour: Colourless to yellow-brown</li></ul>	Always read the label and product information before use. UFI : HWVG-SAD7-KV26-MM4Q



DANGER

### Commercial presentations

- Box 2 x 5L: EXS0068
- Container 10L: EXS0069

### PROFESSIONAL USE ONLY

MADE IN FRANCE

MD class I



Sodel  
190 rue René Barthélemy  
14100 Lisieux, France  
TEL: +33 (0)2 31 31 10 50  
[www.exeol.fr](http://www.exeol.fr)

SANTÉ  
**exeol**

A SODEL DIVISION