ANEROID SPHYGMOMANOMETERS

INSTRUCTION MANUAL

Table of contents

2. Introduction
2. Description
3. Operating instructions
3. Conditions for an accurate measurement
3. Applying the cuff
4. Taking a measurement
5. Choosing the size of the cuff
5. Washing instructions
5. Important information
6. Maintenance and care
6. Servicing and gauging
7. Technical features
8. Guarantee

Moretti S.p.A.
Via Bruxelles 3 - Melegnano
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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2

INTRODUCTION

Buying this sphygmomanometer you have acquired a quality product. We are grateful for your trust in the **MORETTI** brand and thank you for it.

Valuable materials as well as a highly sensitive and long life mechanism assure you precise measurements.

MORETTI sphygmomanometers are manufactured in compliance with both the EN 1060-1-2 standard and the EEC directive 93/42 dated 14 June 1993 regarding medical devices and its following modifications.

The CE mark shows proof of this and so does the 0123 Notified Body (TUV Sddeutschland Group). Each sphygmomanometer is given a lot number under which it is registered.

GUARANTEE

MORETTI sphygmomanometers are guaranteed for a 24 month period (you will find the guarantee coupon on the back of the cover). The guarantee covers neither parts subject to wear and tear nor damages due to improper use.

DESCRIPTION

MORETTI aneroid sphygmomanometers work according to the Riva Rocci/Korotkoff auscultating method. 3 different types of aneroid sphygmomanometers are available:

- ONE HAND TYPE WITH LARGE DIAL
- ONE HAND TYPE WITH SMALL DIAL
- STANDARD TYPE WITH LATEX BULB APART

All models are also available with a stethoscope integrated into the cuff for self-measurement.

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The one hand aneroid sphygmomanometer with a large dial enables an easy reading. The one hand aneroid sphygmomanometer with a small dial is less cumbersome. Both instruments can be easily used with just one hand owing to the PVC bulb which is assembled directly on the manometer. The model with the PVC bulb apart is a classic aneroid instrument to measure blood pressure. This system makes the appliance compact but keeps its technical features and reliability unchanged.

OPERATING INSTRUCTIONS

Sphygmomanometers work according to the Riva Rocci/Korotkoff auscultating method. The PVC bulb enables you to generate the necessary air pressure inside the cuff. The release valve makes it possible to precisely regulate the air outlet from the cuff, which allows you to measure blood pressure. Each manometer has a reading scale ranging from 0 to 300 mmHg (mmHg=millimeters of mercury). Minor notches are traced every 2mmHg and so are larger notches every 10mmHg for an easy reading. The measurement precision is more or less 3mmHg. Each sphygmomanometer is gauged and undergoes a final, complete checking.

CONDITIONS FOR AN ACCURATE MEASUREMENT

The measurement must be carried out with the patient sitting. Place the patient's elbow on the table so as to get the appliance at the same height as his/her heart. The left arm must be free. If the clothes are rolled up they must not oversqueeze the arm. The patient must turn the palm of his/her hand upwards, sit still and he/she should not speak while the pressure is being measured.

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3

To choose the right size of cuff you must know the circumference of the patient's arm. The instrument is provided with a cuff for adults of the adult 1 type with D-ring. This cuff is appropriate for a circumference ranging from 28 to 35cm. For other sizes you should use cuffs according to the table which is on page 5 in this manual.

APPLYING THE CUFF

Before applying the cuff, completely remove the air in it. Properly fasten the cuff on the upper part of the left arm. The lower edge of the cuff must be at least 2 centimeters above the bend of the arm, at heart's height. You should leave just enough room between cuff and arm to fit a finger in it. The label on the cuff must be placed on the artery.

HOW TO APPLY THE CUFF WITH D-RING IN CASE OF APPLIANCES FOR SELF MEASUREMENT

Place the cuff on the upper part of your left arm and make sure that the tubes are turned downwards (towards your hand). Grab the end of the cuff, pull the metal bracket leftwards and fix the cuff pushing on the Velcro adhesive. Get the tube of the stethoscope connected.

TAKING A MEASUREMENT

Take the bulb in your right hand and make sure that the pressure-release valve is closed (turn it clockwise between your thumb and your forefinger).

Squeezing the bulb inflates the cuff. The pointer of the instrument moves and shows the corresponding pressure in the cuff. Keep pumping until you reach 20-30mmHg beyond the highest pressure that you expect. Wait for 4 to 5 pulses and open the release valve a bit so as to get the pointer go down by 2 to 3mmHg per second. Watch the pointer slowly go down.

As soon as you hear the pulse, read the value indicated by the pointer on the manometer. This value corresponds to the systolic

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4

5

or highest pressure. You then hear several pulses which gradually lose intensity. When you hear the last perceptible pulse read the value indicated by the pointer. This corresponds to your diastolic or low-est pressure. The measurement is over. Open the screw on the valve so as to let the air in the cuff completely out.

CHOOSING THE SIZE OF THE CUFF

The following table gives you some information on how to properly use the cuff. Our cuffs comply with the following classification:

Cuff type	Circumference of the limb in cm	Cuff size in cm
Newborn baby	07-13	22x4,5
Child	13-26	38x6,5
Teenager	24-32	48x10
Adult 1 with D-ring	26-36	50x14
Adult 2	26-38	51x14
Overweight	32-47	60x18
Thigh (leg)	42-57	80x21

The adult 1 model is provided with a D-ring, that is a steel ring which makes a double fastening of the cuff possible. It is very convenient for self-measurement. The adult 2 model is a classic Velcro model. All bulbs are in pvc latex free and each model is available with 1 or 2 tubes.

WASHING INSTRUCTIONS

The cuffs must be washed at 30°C with a programme and a detergent for delicate clothes. DO NOT IRON. You can also clean the cuff with a sponge or a soft brush and a detergent for delicate clothes. Rinse with water.

IMPORTANT INFORMATION

- Never let in air beyond 300mmHg.

6

- The time used to measure blood pressure must not exceed 2minutes.
- Never make a second measurement before at least 5 minutes after the first one.
- Protect the instrument from violent jerks and shocks.
- Keep the instrument in a clean, dry place.
- Beware of damaging the rubber parts with sharp, cutting objects.
- Keep the instrument at a temperature between 10 and 40°C with a relative humidity not exceeding 85%.
- Keep the instrument and the rubber parts in particular away from UV rays.

MAINTENANCE AND CARE

1) Manometer

The manometer requires no maintenance. If necessary, address your own dealer. We advise you to clean the manometer with a soft, dry material.

2) Pvc latex free bulb

If you have to replace the latex bulb make sure that the connecting parts are completely dry and clean so that they can firmly hold together. You will thus avoid any air leak.

3) Cuff

Before cleaning the cuff, remove the pvc latex free inner tube and close the Velcro adhesive (placing the metal ring inside if you have a D-ring model).

SERVICING AND GAUGING

When put on the market, the sphygmomanometers are gauged and provided with the CE mark. For the doctor and the patient's safety, we advise you to have your sphygmomanometer checked at least every 2 years by the manufacturer or by your own dealer. They will make sure that it is in good working condition and if necessary they will regauge it. Appliances with a tolerance exceeding 3mmHg must be regauged. In case of repair, original spare parts and accessories must be used.

TECHNICAL FEATURES		Standard aneroid		Small one hand		Large one hand	
Model	Scale: Diameter in mm	50	86	120g	+/-3mmHg	From 0mmHg to 300mmHg	1-tube cuff
Scale: Diameter in mm	50	65	170	180g	+/-3mmHg	From 0mmHg to 300mmHg	1-tube cuff
Length in mm	86	160	170g	180g	+/-3mmHg	From 0mmHg to 300mmHg	1-tube cuff
Weight	120g	170g	180g	180g	+/-3mmHg	From 0mmHg to 300mmHg	1-tube cuff
Precision	+/-3mmHg	+/-3mmHg	+/-3mmHg	+/-3mmHg	+/-3mmHg	From 0mmHg to 300mmHg	1-tube cuff
Measurement range	From 0mmHg to 300mmHg	From 0mmHg to 300mmHg	From 0mmHg to 300mmHg	From 0mmHg to 300mmHg	From 0mmHg to 300mmHg	From 0mmHg to 300mmHg	1-tube cuff
Accessories	2-tube cuff	2-tube cuff	2-tube cuff	2-tube cuff	2-tube cuff	2-tube cuff	1-tube cuff
Material of the case	Enamelled zinc alloy	Enamelled zinc alloy	Enamelled zinc alloy	Enamelled zinc alloy	Enamelled zinc alloy	Enamelled zinc alloy	Chromium-plated steel
Valve	With hand screw	With hand screw	With hand screw	With hand screw	With hand screw	With hand screw	With hand screw
Manufacturer	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC	Wenzhou Kangliu Medical Instrument Co., Ltd. 256 Yongqiang Road Yongzhong Town 325024 Wenzhou, Zhejiang, PRC

7

WARRANTY

This device has 2-year warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In the case you need to get your device repaired and you fulfil the conditions of warranty, send the device to the nearest retailer together with the purchase receipt.

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