

## 2.2 Operation panel

The function of each button found on the operation panel is explained. The buttons on the rear operation panel located on the back surface of the equipment perform the identical functions.

Their functions are explained as per the standard arrangement.

Arrangement of buttons ① to ⑧ can be changed by installing.

A function of the button ⑨ can be selected.

Should the arrangement be changed, read the explanation with respect to that new arrangement.

To change the arrangement, please contact your Shimadzu service representative.

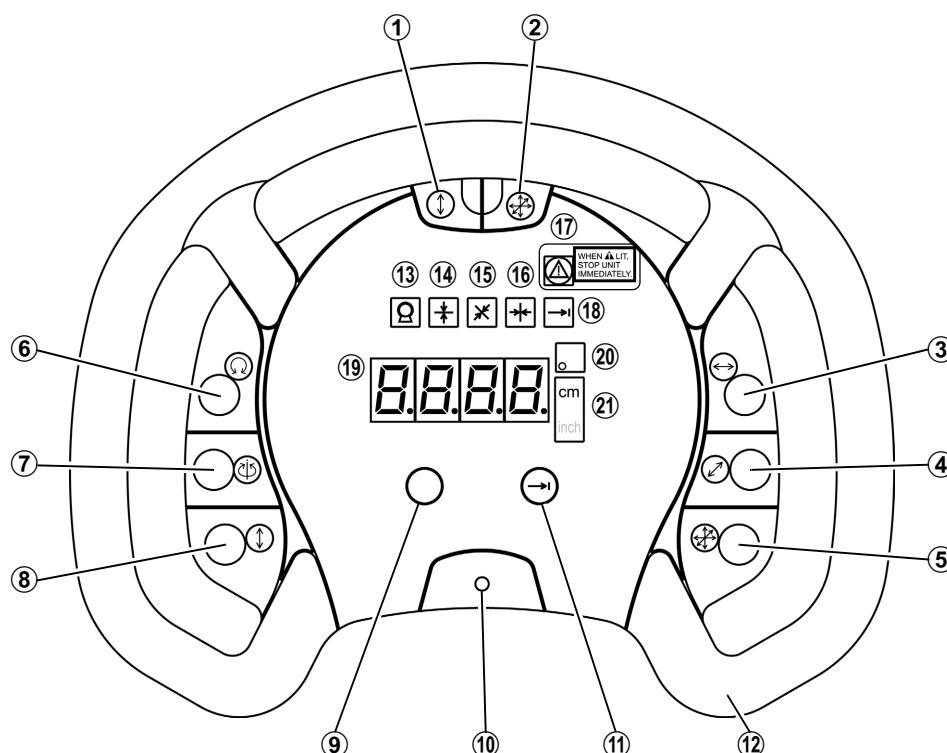


Figure 2.3 Operation panel

### ① [Vertical release] button

This button releases the lock when moving the X-ray tube assembly vertically.

### ② [3-directional release] button

This button simultaneously releases the locks for the vertical, longitudinal and transversal directions.

### ③ [Longitudinal release] button

This button is to release the lock when moving the X-ray tube assembly longitudinally.

### ④ [Transversal release] button

This button is to release the lock when moving the X-ray tube assembly transversally.

### ⑤ [3-directional release] button

This button performs the same function as ②.

### ⑥ [H-axis rotation release] button

This button is to release the lock when rotating the X-ray tube assembly around the horizontal axis.