




	<b>Technical Data Sheet</b>  <b>Sterile Hypodermic Syringes for Single Use</b>	Revision : 5
		Date : 2019-07-01
		Page 1 of 2

Manufacturer: Changzhou Jiafeng Medical Equipment Co., Ltd	1ML Insulin Syringe For Insulin U-100 Use (with integrated needle)	
---	--	--

Product variants	Ref	Description	Luer type
	SI321012913	1ML Insulin Syringe with needle 29G * 1/2"	Integrated needle
Intended use		Single-use hypodermic 1ml Insulin syringe for Insulin injection only	
Materials		PP, NR, stainless steel, silicone oil (Syringes are PVC- and DEHP-free)	
Needle		Designed with optimal point geometry for maximum comfort and reliability, thin-walled stainless steel, trilateral sharpened and ultrasound polished. Needle surface coated with special grease.	
Sterilization		Ethylene oxide (E.O)	
Shelf life		60months	
Product standards		EN ISO8537:2016	
Classification		MDD 93/42/EEC, Class IIa	
Certificates		CE,ISO13485, CFDA, FSC	
Packaging		Single PE bag, 150pcs per middle bag, 20middle bags per carton	
Storage		Store in relative humidity less than 80%, no corrosive gas, cool, dry, well ventilated, clean environment.	



	<b>Technical Data Sheet</b>  <b>Sterile Hypodermic Syringes for Single Use</b>	Revision : 5
		Date : 2019-07-01
		Page 1 of 2

Operation Instruction	1.Open the individual packing and take out the syringe 2.Take off the needle cap, avoid touching with the needle tube 3.Extract the solution and exhaust the air 4.Push the plunger to the top of the barrel to finish fluid injection 5.Throw the used syringe in to to the medical waste box
Cautions	*Check the batch number and validity before use. *It is strictly prohibited to use if the single packing damaged or needle cap drop off. *For single use only, use immediately after unsealed, destroy after use. It is strictly prohibited to reuse.
Marks	  
Packing	* Primary PE bag -Sample PE bag design  