## CELLETTA™ Standard brush cell collector

### CELLETTA™ brush cell collector for Pap Smears

Materials: Handle PP, Brush head: Stainless steel wire nylon bristles

Art.-No.: 39173

Contents: 20 x 100 pcs.

For SINGLE use only. Disposable cell collector. Latex-free

#### WARNING

Only for medical application.

DO NOT use CELLETTA™ brush cell collector for endometrial sampling. Use for cervical sampling only.

If used on pregnant patients sample with care.

Check tight fitting of brush head before use!

CELLETTA™ is a registered trademark of Ranir Germany.  
Made in India.

#### INSTRUCTIONS FOR USE

1. Mark with a pencil on a clean microscope slide name and identification number.
2. Use a spatula for sampling from vaginal fornix and portio.
3. Smear on 2/3 of the slide.
4. Check tight fitting of brush head before use!
5. Insert CELLETTA™ gently into the endocervical canal.
6. Rotate the brush ONLY ONE HALF TO ONE FULL TURN.
7. Smear the cell material from the brush on the rest of the slide.
8. Fix immediately to avoid air-drying which destroys the cells. Send the sample to the laboratory.

### CELLETTA™ Standard brush cell collector

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Engelbrecht Medizin- und Labortechnik GmbH