
CALCEMEX®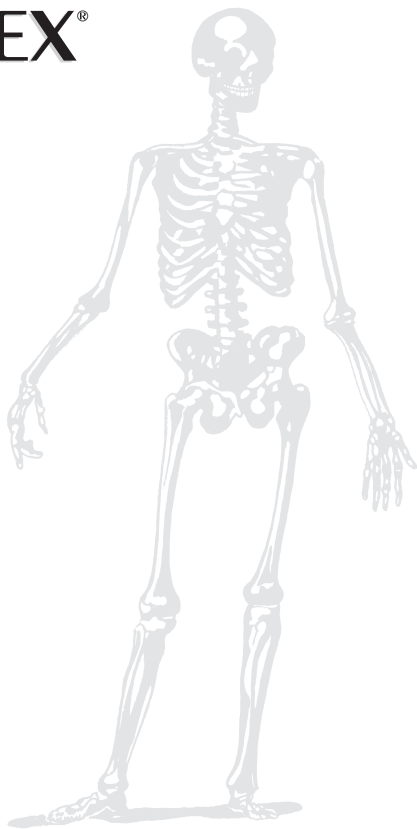

TECRES®
ADVANCING HIGH TECHNOLOGY

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CALCEMEX®

INSTRUCTION BOOKLET

To the personal attention of the surgeon

DESCRIPTION

Calcemex is a radiopaque injectable, mouldable and biocompatible bone void filler.

Calcemex is a hybrid device, organic-inorganic, consisting of PMMA (polymethylmethacrylate) and TCP (tricalcium phosphate).

Calcemex is a sterile and disposable medical device, composed of an hermetically sealed sachet and a phial. The liquid is sterilised by means of filtration and the powder by means of gamma irradiation.

Composition

Powder (10 g)	%
PMMA	40.4
TCP	41.6
Benzoyl Peroxide	1.3
Saline solution	16.7

Liquid (3,1 g)	%
MMA	99.5
N,N Dimethyl-p-Toluidine	0.5
Hydroquinone	75 ppm

INDICATIONS FOR USE

Calcemex bone void filler is intended for bony void or defects that are not intrinsic to the stability of the bony structure. The device is intended to be placed or injected into bony void or gaps of the skeletal system. These defects may be surgically created osseous defects or osseous defects created from traumatic injury to the bone. The product provides a bone void filler that partially resorbs and is replaced with bone during the healing process.

The device is not intended for use in the spine and should not be used in the presence of active or suspected infection.

Contraindications

Calcemex must not be used under the following conditions:

- Existing acute or chronic infections, particularly at the implant site
- Existing inflammatory bone diseases such as osteomyelitis
- Existing malignant tumour diseases
- Pronounced impairment of renal function
- Patients who are allergic to the monomer or to any of its components.
- To fix joint prosthesis (knee, hip)
- Maxillo-facial application

In principle, particular caution is warranted in patients with the following conditions:

- Improperly controlled endocrinopathies
- Disorders of calcium metabolism
- Immunosuppressive therapy
- Therapy with drugs affecting bone metabolism, e.g. bisphosphonates or calcitonin

PRECAUTIONS/USE

Surgical use of **Calcemex** must be restricted to qualified professional with expertise in the surgical technique. Pre-surgical evaluations, adequate selection of materials, use of compatible surgical techniques and procedures as well as post-surgical control are crucial to produce the desired result.

Store at a temperature below 25°C in a dry place away from all sources of light.

Do not re-sterilize and/or re-use the device. It is designed for single-use on a single patient. Avoid dividing the product into two or more portions, in order to use it other times. This would be a re-utilization and may lead to an error in the proportion of the powder and liquid components. It can also cause a sterility loss.

Re-sterilization should not be carried out since it can cause infection risks for the patient. Re-sterilization can also alter the device's morphology and the mechanical features, causing a malfunction of the same with serious risks for the patient's health.

All residues must be considered waste surgical material and therefore should be eliminated at the end of the operation. Because of the volatility and flammability of the liquid monomer of this product, the liquid monomer should be evaporated in a well-ventilated hood or absorbed by an inert material and transferred into a suitable container for disposal. The polymer component may be disposed in an authorized waste facility.

USER PRECAUTIONS

Caution should be exercised during the mixing of the liquid and powder components of **Calcemex** to prevent excessive exposure to the concentrated vapours of liquid monomer, which may produce irritation of the respiratory tract, eyes, and possibly the liver.

Avoid monomer contact with the skin and mucous membranes. Cases of contact dermatitis have been observed in susceptible subjects. It is therefore advisable to wear a second pair of surgical gloves and scrupulously observe the instructions for mixing the components in order to reduce the possibility of reactions caused by hypersensitivity.

Personnel wearing contact lenses should not mix bone void filler or be near its mixing.

The liquid component is a powerful lipid solvent. It should not contact rubber or latex gloves. The product should not contact the gloved hand until it has acquired the consistency of dough, about 2 min and 1/2 after mixing.

The liquid component is flammable and volatile and for this reason the operating theatre must be correctly ventilated. The liquid component and/or its vapours must never be directly exposed to naked flames or heated materials. Ignition of monomer vapours caused by the use of electrocautery devices in surgical sites near freshly implanted bone cement has been reported.

Once the two components are mixed, the consistency of the dough changes in just a few minutes: viscosity increases rapidly to form a solid mass. The attainment of this state is easily ascertained by the increase in temperature of the dough itself. After a few minutes, the paste cools spontaneously, indicating the end of the reaction.

CAUTIONS:

Avoid to disturb the device during polymerization.

Attention to the extra-filling of the device of the defect site

Eventual excess material must be removed as soon as possible.

In order to reduce fat embolization and/or embolization of the device material into the bloodstream, it is recommended to avoid overfilling of the bone site.

CALCEMEX PREPARATION PRECAUTIONS

Sterility is assured only if the unit container is not damaged or opened.

Do not use the product after the expiration date because the effectiveness of the device may be compromised.

Make sure that the inner package is undamaged and that the components are undamaged. The liquid in the ampoule should not appear syrupy.

Temperature has a major effect on the preparation characteristics of **Calcemex**. Temperatures of more than 23°C for the product, the preparation accessories and the environment accelerate the various stages in the preparation procedure. Lower temperatures retard the preparation stages.

Make sure that the **Calcemex** preparation accessories are specifically compatible with the product.

Do not open the phial of liquid over the mixing bowl to avoid the risk of glass fragments entering the dough.

Do not mix the paste in a flow of air since this will promote rapid evaporation of the liquid component and consequent variation in the performances.

CALCEMEX APPLICATION PRECAUTIONS

Clinical study data demonstrate the need to maintain strictly aseptic surgical techniques. It must be borne in mind that any deep infection of a surgical wound is a serious risk for the successful outcome of the implant. Such infection may begin in a latent manner without clinical evidence even some years after surgery.

To minimise the risk of inclusion of blood and debris in **Calcemex**, and of marrow content in the vascular system, prior to the application of the bone void filler, the bone cavity should be thoroughly irrigated with Ringer or saline solutions and dried.

SPECIAL PRECAUTIONS

The surgeon must inform the patient about the correct behaviour to follow after the operation. Following surgery, if any form of infection should arise, patients must immediately consult their doctors to reduce the risks of infection.

NEVER add other substances or foreign bodies to the paste.

Caution: the paste reaches temperatures higher than physiological temperatures during the exothermic polymerisation reaction. Polymerisation is an exothermic reaction that occurs while the paste is hardening *in situ*. The released heat may damage adjacent bone or tissue.

USE DURING PREGNANCY OR LACTATION

No investigations have been carried out in pregnant or nursing patients.

USE IN CHILDREN OR ELDERLY PATIENTS

No particular precautions are known for elderly patients. No investigations have been carried out in children.

ADVERSE EFFECTS

No adverse reactions attributable to the material are expected due to the composition of the material.

INTERACTIONS

No interactions with other medicinal products or drugs are expected due to the composition of the material.

GENERAL RISKS

Note that like in any surgical intervention, surgery-related risks may occur such as rejection reactions, pseudarthrosis, lack of healing, swelling, seroma and haematoma formation, pain, wound healing disorders, and dehiscence. As a consequence, repeated operations may become necessary.

APPLICATION INSTRUCTIONS

Open the powder sachet and the liquid phial on a sterile shelf in the operating room, and pour them inside a mixing bowl. Mix with a spatula till to obtain a dough.

The dough may be used manually or injected with a delivery device.

Note: all liquid and powder shall be used. **Caution:** never arbitrarily modify the ratios between the liquid and solid components.

Note: for syringe use is advisable to use a 5ml syringe. Remove the plunger of the syringe and use a spatula to load **Calcemex**.

For manual application

Duration	Step / Phase	Notes
0 - 30 secs	Mixing phase	Break open the phial and pour the liquid into the mixing bowl and mix with a spatula
30sec - 2min30sec	Waiting phase	
2min30sec – 7 mins	Working phase	The paste can be used/applied
> 7 mins	Hardening phase	Polymerization reaction – the paste cannot be used anymore

For syringe application

Duration	Step / Phase	Notes
0 - 30 secs	Mixing phase	Break open the phial and pour the liquid into the mixing bowl and mix with a spatula
30 secs – 1min and 15 secs	Transfer to delivery device	
1 min and 15 secs – 7 mins	Working phase	The paste can be injected
> 7 mins	Hardening phase	Polymerization reaction – the paste cannot be used anymore

Laboratory testing has demonstrated that temperature may affect the timing of preparation / application / hardening of **Calcemex**.

A graph on setting time according to temperature is provided for ease of reference (data were obtained in controlled environmental and storage conditions subjected to standard deviation).

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TECRES S.p.A.