


CPAP/BIPAP Masks


EaseFit FMI-NV

Make breath naturally happen!

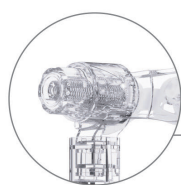

Forehead Stepless Knob

- Independent innovation
- Stepless knob design to fit forehead easily
- Perfectly suit individual facial contour

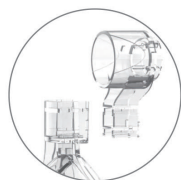

Adjustable Height

- 2 levels of height are available



Elbow type 1

- Flexible rotation 360°
- For common tube (single limb)



Elbow type 2

- Flexible rotation 360°
- For professional medical breathing circuit (dual limb)



Double Cushions

- Medical liquid silicon
- A comfortable seal is achieved by dual-wall design
- Easy to clean and reassemble



Arc-buckle

- Curve buckle structure enables flexible rotation
- Easy to attach, easy to adjust



Dimension Details



EaseFit FM I -NV
Elbow type 1



EaseFit FM I -NV
Elbow type 2



Dimension Size	L(mm)		W(mm)	T(mm)
	Elbow Type 1	Elbow Type 2		
Large	194.9	180.4	122	114.5
Medium	183.9	169.4	118	114.5
Small	175.9	156.9	110	112.5



Wearing Instructions



Put the headgear through the mask from both sides.



Put the mask on the head in good position.



Adjust the headgear and forehead pad to ensure fixation and comfortability.



Cleaning Guide

Daily Use

Slightly wash the mask accessories (except for headgear) with soap in warm water; After cleaning, dry them in natural way. Use a soft brush to clean the vent hole.

Weekly Washing

Clean the headgear with hand. First soak it in warm soap water for less than 10 min; And then dry it in natural way. Avoid expose it to direct sunlight.



Accessory Replacement

Silicon cushion »» Every 6 months


Headgear »» Every 6 months

Mask frame »» Every 12-18 months

CPAP/BIPAP Masks

EaseFit NMI-NV

Make breath naturally happen!

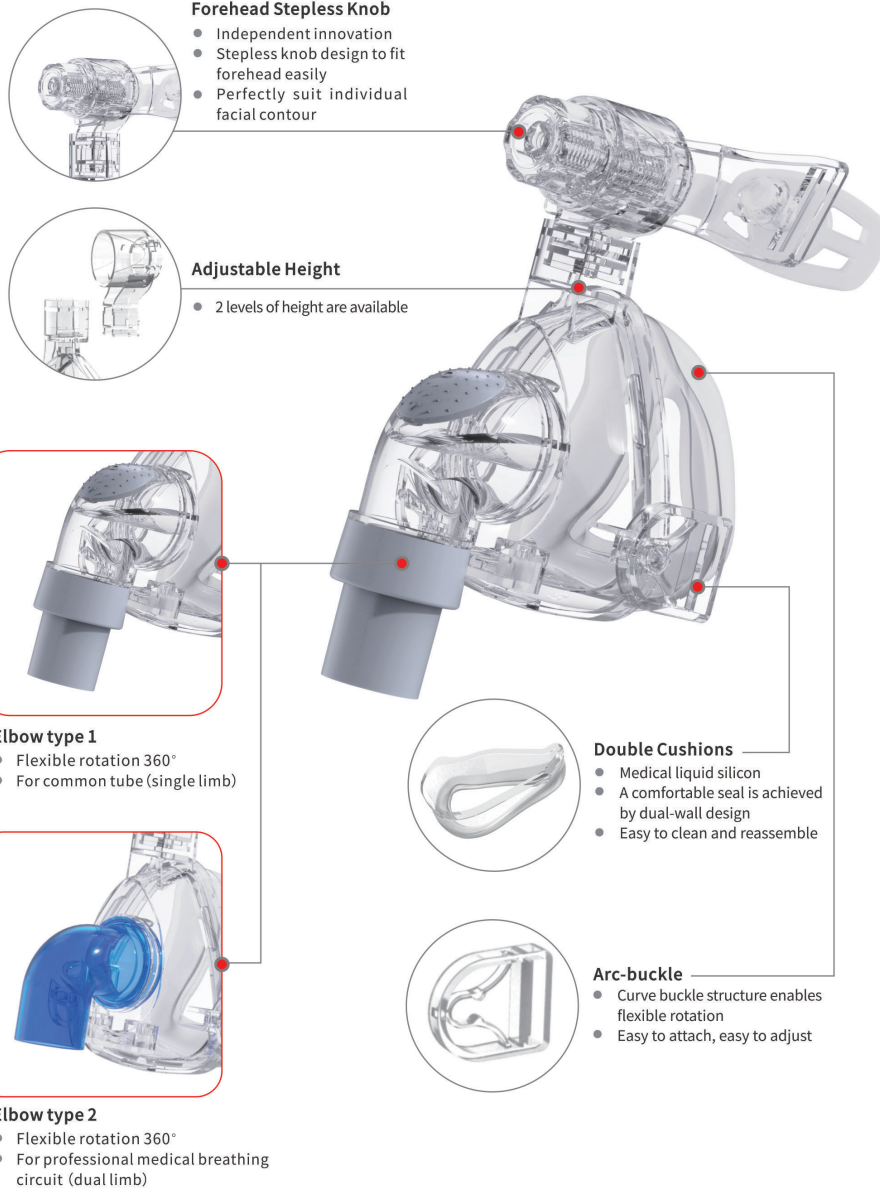


Dimension Details



EaseFit NMI-NV
Elbow type 1



EaseFit NMI-NV
Elbow type 2

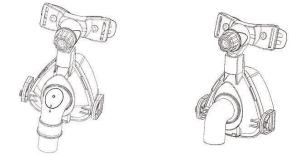

Dimension Size	L(mm)		W(mm)	T(mm)
	Elbow Type 1	Elbow Type 2		
Large	150.6	130.5	95.6	96.5
Medium	150.6	130.5	95.6	94
Small	138.6	114.9	88.9	93.3



Wearing Instructions



Put the headgear through the mask from both sides.



Put the mask on the head in good position.



Adjust the headgear and forehead pad to ensure fixation and comfortability.



Cleaning Guide

Daily Use

Slightly wash the mask accessories (except for headgear) with soap in warm water; After cleaning, dry them in natural way. Use a soft brush to clean the vent hole.

Weekly Washing

Clean the headgear with hand. First soak it in warm soap water for less than 10 min; And then dry it in natural way. Avoid expose it to direct sunlight.



Accessory Replacement

Silicon cushion >>> Every 6 months

Headgear >>> Every 6 months

Mask frame >>> Every 12-18 months