EtOH WH 0,3 g/L – In vitro diagnosticum

Ethanolkontrollen im Vollblut

Anwendung

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Ethanol control in whole blood

Application

This material should be used in accordance with the laboratory's operating procedures for instrument calibration or as a control material.

User-guide

This ACQ Science EtOH WH requires no additional preparation and is ready for use.

Assigned Value

The assigned ethanol concentration was determined by 3 independent laboratories each accredited to DIN EN 17025. Measurements were carried out daily, in duplicate, on 5 separate days using Gas Chromatography.

Storage and stability

Storage: 2 ° to 8 ° C

Shelf Life:

- Unopened Vial, stored in the dark: see expiration date on the package.
- Stored in the dark tightly capped: see expiration date on package

Precautions

All materials of human origine should be considered as potentially infectious and treated with the same care as patient specimens.

Each individual original blood unit used for the production of the control was tested for following antigens and antibodies: HBsAG, anti-HIV-1, anti-HIV-2, anti-HBc and anti-HCV and found to be negative.

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GI_EtOHWH_03_402101056_De_En_20110205.doc

¹ Confidence ranges – Measured values

The confidence interval indicates the range in which the target value is located with a significance level of 95%.

² Confidence ranges – German forensic directives

[EtOH] ≥ 1,0 g/L → ± 5% from the target value
[EtOH] < 1,0 g/L → ± 0,05 g/L from the target value

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ACQ Science GmbH