

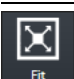


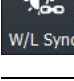
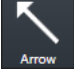

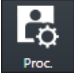
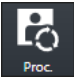

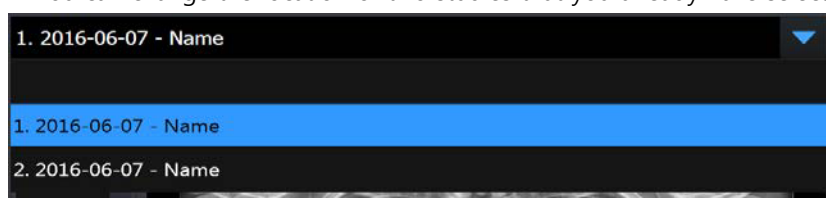
	W/L (Window Level)	Adjusts the brightness and contrast of the image.
	CW (Clockwise)	Rotates an image by 90° clockwise.
	CCW (Counter Clockwise)	Rotates an image by 90° counterclockwise.
	Fit	Fits an image to the main image window size. If the image is cropped, the cropped area is fit to the window size.
	Sync	When you zoom, pan, rotate the selected image after activation, you can it automatically applies to all remaining images displayed on the screen simultaneously.
	W/L Sync	If you adjust the W/L of the selected image after activation, it automatically applies to all remaining images displayed on the screen simultaneously.
	Arrow	Draws an arrow to indicate the specific region of the image.
	Text	Adds text to the image to indicate the information.
	Image Process	Processes the image with various tools from XIPL , the image processing program connected with VXvue .
	Process Control	Processes the image conveniently.
	Save	Saves the processed image.
	Close	Finishes comparing studies.

9.7.3 Selecting Studies

- Select the combo box to select studies you want to view.
 - A study without images can be selected.
- You can change the location of two studies that you already have selected.



9.7.4 Selecting Images

- Click the thumbnail to select images.
- You can compare the selected images with those of another study.